# STETHOSCOPES

**seca s60, s50, s40**  
**seca s20, s22, s30, s32**  
**seca s10**

# **USER MANUAL**

## **STETHOSCOPES**

<b>English</b> .....	<b>6–9</b>
<b>Français</b> .....	<b>10–13</b>
<b>Deutsch</b> .....	<b>14–17</b>
<b>Español</b> .....	<b>18–21</b>
<b>Suomi</b> .....	<b>22–25</b>
<b>Polski</b> .....	<b>26–29</b>

# EXPLANATION OF SYMBOLS | LÈGENDE | ERKLÄRUNG DER SYMBOLE | EXPLICACIÒN DE SÌMBOLOS | SYMBOLIEN SELITYKSET | OBJAŚNIENIE SYMBOLI:

Follow Instructions for Use.  
Observer les consignes  
du manuel de l'utilisateur.  
Gebrauchsanweisung beachten.  
Tener en cuenta las  
instrucciones de uso.  
Noudata käyttöohjetta.  
Przestrzegać instrukcji obsługi

Fragile: Handle with care!  
Fragile, à manipuler avec  
precaution  
Zerbrechlich, mit Sorgfalt  
handhaben  
Frágil, manipular con cuidado  
Helposti särkyvä,  
käsiteltävä huolella  
Delikatny instrument, zachować  
ostrożność

Manufacturer  
Fabricant  
Hersteller  
Fabricante  
Valmistaja  
Producent

Store in a dry place  
À stocker dans un  
endroit sec  
Trocken aufbewahren  
Conservar en seco  
Säilytettävä kuivana  
Przechowywać w suchym miejscu

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This product complies with the Council Directive 93/42/EEC concerning medical devices.

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## TABLE OF CONTENTS

1. Description of device .....	6
2. Safety precautions .....	6
3. Operation .....	7
4. Hygiene treatment .....	8
5. Servicing .....	9
6. Disposal .....	9
7. Warranty .....	9
8. Declaration of conformity .....	9

## 1. DEVICE DESCRIPTION

---

### 1.1 Congratulations!

With this stethoscope, you have just purchased a precise and simultaneously robust device.

seca has used its technical expertise in the service of health care for more than 170 years and, as a market leader, has always set standards worldwide with its innovative developments.

### 1.2 Intended use

seca stethoscopes are used primarily by doctors and trained staff to auscultate various organs such as the heart, lungs and intestines, together with the arteries, to assess sounds. When measuring blood pressure, the stethoscope is used to detect Korotkoff sounds in order to determine systolic and diastolic blood pressure in adults or children.

### 1.4 Contraindications

No contraindications are known to date.

## 2. SAFETY PRECAUTIONS

---

### 2.1 Safety precautions in these Instructions for Use

#### **CAUTION!**

Identifies a hazardous situation. If you fail to take note of this information, minor to moderate injury may result.

#### **NOTICE!**

Indicates that the product may have been operated incorrectly. If you fail to take note of this information, the device may be damaged or the measured results may be incorrect.

**NOTE**

Additional information on how to use this device.

**2.2 Basic safety precautions on handling the device**

- Please follow the precautions in these Instructions for Use
- Keep the instructions for use in a safe place.

**CAUTION! Avoiding injuries**

- Never use stethoscopes without ear tips.
- Do not use stethoscopes without diaphragms.
- Do not push ear plugs too far into the auditory canal.

**NOTICE!**

- Check that the product is working perfectly before using it each time.
- Do not use the product if it is visibly damaged.
- Avoid damaging the rubber parts with sharp or pointed objects.
- These products may not be taken into strong magnetic fields, for example MRI, and used there.
- Only use genuine seca spare parts.
- Avoid subjecting the device to shocks.
- Follow the care instructions.
- Check regularly for soiling and clean if required.
- Do not sterilize using heat.
- Do not immerse in cleaning liquid and protect from dirt to prevent cleaning liquids or solid bodies getting into the system of hollow spaces inside the stethoscope.
- Do not expose the stethoscope to any direct UV radiation, this applies to all rubber parts in particular.

**NOTE:**

- Only qualified persons may interpret the results.
- seca stethoscopes are free of DEHP and latex.

**2.3 Ambient conditions**

- Storage at room temperature between -10 °C and 40 °C
- Relative humidity below 85 %

**3. OPERATION**

---

**3.1 Before use**

Before any use involving direct contact with the body (patient/examiner), ensure that you have cleaned and disinfected the item.

### 3.2 Handling the stethoscope

Please note the points below to obtain perfect hearing quality.

- Put the stethoscope on your head so that the ear tips are pointing in your direction of view. This ensures that the ear tips are put into the ear from the back to suit the anatomy of the auditory canal.
- You can reduce the initial tension of the stethoscope and thus the pressure on the auditory canal by taking the headset between your thumb and forefinger and slowly pulling the limbs apart.
- If the initial tension is not enough to ensure that the stethoscope is firmly located in the ear, you can increase it by taking the headset between your thumb and forefinger and carefully pushing the limbs together.
- Check regularly that the ear tips are firmly attached to the ends of the headset.

#### **NOTE: Switching over stethoscopes which have a double chest piece**

- When using a stethoscope with a double chest piece, you need to ensure that you have selected the right side for your intended purpose by turning the chest piece. Only one side is ever activated for auscultation.
- The switch is made by rotating the chest piece 180 ° until you hear a click.

## 4. HYGIENE TREATMENT

---

### 4.1 Cleaning and caring for stethoscopes

#### **NOTICE! Damage to device**

Unsuitable cleaning agents and disinfectants may damage the sensitive surfaces of the device.

- Do not use aggressive or abrasive cleaning agents.
- Do not use white spirit or petroleum spirit.
- Only use disinfectants suitable for sensitive surfaces. Suitable disinfectants are available from specialist dealers.
- For cleaning, we recommend a damp cotton or microfiber cloth wetted with disinfectant if appropriate.
- Prevent liquids from penetrating the housing.
- Do not immerse the stethoscope in liquids or autoclave it.

The stethoscope can be cleaned by spraying or wiping with disinfectant.

- **Spray disinfecting:** spray all parts with the cleaning product/disinfectant and leave the products to take effect in accordance with the manufacturer's instructions. If necessary, wipe over with a dry cotton cloth afterwards.
- **Disinfecting by wiping:** wet the stethoscope or a cloth with the cleaning product/disinfectant. Then wipe over the surfaces applying a gentle pressure.
- To support these cleaning methods, please use only clean, soft cotton cloths.

Cleaning by spraying or wiping with disinfectant	We recommend the following cleaning products and disinfectants: <ul style="list-style-type: none"> <li>• Fugaten® spray (Lysoform spray)</li> <li>• Esemfix® from Schülke (foam cleaner)</li> <li>• Promanum® pure from Braun</li> <li>• Meliseptol® Foam pure from Braun</li> </ul>
Care	To maintain the elasticity of the tubing and other non-metal components, we recommend using a care product designed for vinyl.

## 5. SERVICING

The stethoscope requires no servicing.

## 6. DISPOSAL

This device consists of high-quality raw materials which can be recycled. Dispose of the device in accordance with the local laws and regulations applicable in your country.

## 7. WARRANTY

There is a two-year warranty period from delivery for defects attributable to poor materials or workmanship. Defects which come under the warranty will be repaired for the customer free of charge against proof of purchase. Additional claims cannot be considered. Costs of transport to and from seca are the responsibility of the customer if the device is located somewhere other than the customer's headquarters. In the event of transport damage, warranty claims can only be made if the complete original packaging was used for transport and the device was secured and fastened therein according to its originally-packed condition. You should therefore keep all packaging parts.

The warranty will be voided if the device is opened by persons not expressly authorized by seca to do so.

## 8. DECLARATION OF CONFORMITY

This product complies with Council Directive 93/42/EEC pertaining to medical devices. For more information on the conformity of seca products, please go to [www.seca.com/products](http://www.seca.com/products).

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