# seca 651/650

## Instructions for use

17-10-07-653-100b1\_2023-09S

Firmwareversion: 1.7

<b>Deutsch – Gebrauchsanweisung .....</b>	<b>3</b>
<b>English – Instructions for use .....</b>	<b>78</b>
<b>Français – Mode d’emploi.....</b>	<b>150</b>
<b>Español – Instrucciones de uso .....</b>	<b>224</b>

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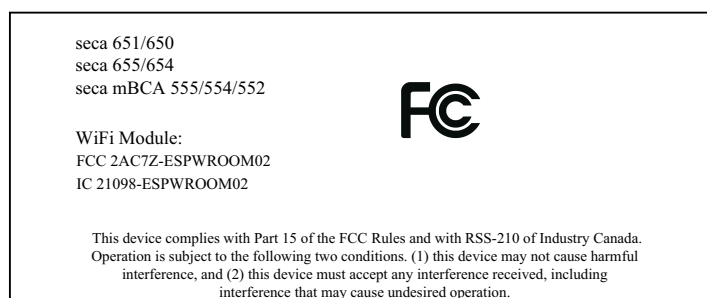
#### NOTE

This device complies with Part 15 of the FCC Rules and with RSS-210 of Industry Canada. Operation is subject to the following two conditions:

- This device may not cause harmful interference.
- This device must accept any interference received, including interference that may cause undesired operation.

#### NOTE

Changes or modifications made to this equipment not expressly approved by seca may void the FCC authorization to operate this equipment.

#### NOTE

Radiofrequency radiation exposure information: This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with minimum distance of 1 m between the radiator and your body. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

# TABLE OF CONTENTS

<b>1</b>	<b>About this document</b> .....	<b>80</b>	<b>5</b>	<b>Starting up device</b> .....	<b>94</b>
1.1	Representation in text .....	80	5.1	Scope of delivery.....	94
1.2	Representation in diagrams .....	80	5.2	Interfaces.....	95
1.3	PDF version .....	81	5.3	Setting up device .....	96
<b>2</b>	<b>Description of device</b> .....	<b>81</b>	5.4	Fitting the multifunctional display to the positioner.....	97
2.1	Intended use.....	81	5.5	Installing the multifunctional display on the wall .....	97
2.2	Clinical benefit.....	81	5.6	Connecting the multifunctional display to the scale .....	98
2.3	Contraindications .....	81	5.7	Connecting a barcode scanner (optional) .....	98
2.4	Patient target group.....	81	5.8	Establishing power supply .....	99
2.5	User qualification.....	81	5.9	Installing the drip guard.....	100
	Assembly .....	81	5.10	Adapting device settings.....	100
	Administration/network operation .....	81	5.11	Transporting device.....	101
	Measuring mode.....	81			
2.6	Functional description .....	82	<b>6</b>	<b>Operation</b> .....	<b>101</b>
	Measuring weight/entering height .....	82	6.1	Starting the measurement procedure .....	102
	Network functions .....	82		Activating the multifunctional display (stand-alone operation) .....	102
	Compatibility .....	82		Activating the multifunctional display (network connection) .....	102
				Using a reserved device (network connection).....	103
<b>3</b>	<b>Safety information</b> .....	<b>82</b>	6.2	Measuring weight.....	104
3.1	Safety information in these instructions for use.....	82	6.3	Entering height manually .....	104
3.2	Basic safety information .....	83	6.4	Using extended weighing functions .....	105
	Handling device.....	83		Taring additional weight (Tare) .....	105
	Preventing electric shock .....	84		Permanently displaying the weight (Hold) .	107
	Preventing injuries and infections.....	84		Permanently store additional weight (Pre- Tare) .....	108
	Preventing damage to device.....	85		Switching weighing range .....	109
	Handling measuring results .....	86		Automatic calculation of BMI or BSA.....	109
	Handling packaging material.....	86		Entering waist circumference .....	110
<b>4</b>	<b>Overview</b> .....	<b>87</b>	6.5	Completing the measurement.....	110
4.1	Controls .....	87		Stand-alone operation.....	110
4.2	Symbols on the ID display (main screen) .....	88		Devices with connection to an EMR System .....	111
4.3	Symbols on the ID display (menu).....	90		Devices with a connection to the seca analytics 125 software.....	112
4.4	Labels .....	91			

<b>7 Configuration</b> .....	<b>112</b>	<b>10 Troubleshooting</b> .....	<b>133</b>
7.1 Basic functions.....	112	10.1 General faults .....	133
Changing device mode .....	112	10.2 Measuring weight .....	134
Calling up/exiting a menu.....	113	10.3 Data transmission.....	134
Using the PDF version of the instructions for use (QR code) .....	114	10.4 Restarting the device .....	137
7.2 Measuring .....	114	Using the "Restart" menu function.....	137
Activating Autohold function .....	114	Interrupting and restoring the power supply (power supply operation) .....	138
Setting filter .....	115	10.5 Traffic light system for display messages .....	138
Selecting BMI/BSA/Waist circumf.....	116	10.6 Error codes.....	139
7.3 Adjusting display settings.....	117	<b>11 Servicing</b> .....	<b>140</b>
Setting display language .....	117	11.1 Verified scales .....	140
Setting date format.....	118	11.2 Non-verified scales.....	141
Setting name format.....	119	<b>12 Technical data</b> .....	<b>142</b>
Setting standby time .....	120	12.1 Menu structures .....	142
Setting display brightness .....	121	"Basic" device mode.....	142
Switching units .....	122	"Advanced" device mode.....	142
7.4 Setting up network functions .....	123	"Expert"/"Service" device modes .....	143
Entering server address.....	124	12.2 Functions/device mode.....	144
Activating/deactivating the WiFi function..	125	12.3 General technical data .....	145
Connecting the device to a WiFi network..	126	12.4 Dimensions and weights .....	146
Connecting the device to a WiFi network (WPS).....	127	12.5 Weight measurement .....	147
Permitting anonymous measurements.....	128	<b>13 Optional accessories and spare parts</b> .....	<b>148</b>
Automatically clearing measured values (Autoclear) .....	129	<b>14 Compatible seca products</b> .....	<b>148</b>
7.5 Factory settings.....	130	<b>15 Disposing of the device</b> .....	<b>148</b>
Overview of factory settings.....	130	<b>16 Warranty</b> .....	<b>148</b>
Restoring factory settings .....	130	<b>17 Declarations of conformity</b> .....	<b>149</b>
<b>8 Hygiene treatment</b> .....	<b>131</b>	17.1 Europe .....	149
8.1 Cleaning.....	132	17.2 USA and Canada.....	149
8.2 Disinfecting.....	132		
8.3 Sterilizing .....	132		
<b>9 Function check</b> .....	<b>132</b>		

# 1 ABOUT THIS DOCUMENT

## 1.1 Representation in text

Symbol	Description
✓	Requirement for actions
▶	Action
1. 2.	Actions with specified sequence
a) b)	Steps of an action with specified sequence
⇒	Result of an action
• •	First level of a list
– –	Second level of a list

## 1.2 Representation in diagrams

Symbol	Description
	Indicates relevant points on the device or on device components
	Indicates directions of movement of the device or of device components
	Correct action Correct result of action
	Incorrect action Incorrect result of action
	Points to the next step of a procedure
	Points to an element the user is clicking
	End of a procedure, e.g. the installation of a part

## 1.3 PDF version

---

The device menu contains a QR code which can be used to access the PDF version of these instructions for use and to load them e.g. onto your smartphone or tablet PC.

Further information is available here: → [Using the PDF version of the instructions for use \(QR code\), page 114](#)

## 2 DESCRIPTION OF DEVICE

### 2.1 Intended use

---

The electronic flat scale supports physicians in decision-making regarding diagnosis or therapy based on weight.

### 2.2 Clinical benefit

---

The electronic flat scale supports physicians in decision-making regarding diagnosis or therapy based on parameters measured and calculated (indirect clinical benefit).

### 2.3 Contraindications

---

No contraindications are known.

### 2.4 Patient target group

---

Persons of all ages who do not exceed the maximum capacity of the scale and who can stand independently on the scale.

### 2.5 User qualification

---

#### Assembly

Devices shipped partly assembled may only be assembled by sufficiently qualified persons (such as specialist dealers, hospital technicians or seca Service technicians).

#### Administration/network operation

The device may only be set up and incorporated in a network by experienced administrators or hospital technicians.

#### Measuring mode

The device may only be operated by persons with formal training in the health-care sector or in medicine.

## 2.6 Functional description

---

### Measuring weight/entering height

Weight calculation is performed by four load cells. The measured results are shown on the multifunctional display. Height is entered manually.

Body Mass Index (BMI) or Body Surface Area (BSA) are calculated automatically as soon as a height is entered.

### Network functions

The device can be integrated into a PC network via a LAN interface or via WiFi. The **seca connect 103** software is required to set up this integration.

The **seca connect 103** software receives measurement data from the device and forwards them to an EMR System or to the **seca analytics 125** software.

The **seca analytics 125** software receives measurement data from the **seca connect 103** software and processes them in graphical form. The software thus assists the attending physician in analyzing measured results and making a diagnosis.

### Compatibility

**seca connect 103** configuration software: Version 3.1 or higher, no downward compatibility

**seca analytics 125** analysis software: Version 1.8 or higher

## 3 SAFETY INFORMATION

### 3.1 Safety information in these instructions for use

---

#### **DANGER!**

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries will occur.

#### **WARNING!**

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries may result.

#### **CAUTION!**

Used to identify a hazardous situation. If you fail to take note of this information, minor to moderate injuries may result.

#### **NOTICE!**

Used to identify possible incorrect usage of the device. If you fail to take note of this information, you may damage the device, or the measuring results may be incorrect.

#### **NOTE**

Includes additional information about use of the device.

## 3.2 Basic safety information

### Handling device

- ▶ Please take note of the information in these instructions for use.
- ▶ Keep the instructions for use in a safe place. The instructions for use are a component of the device and must be available at all times.
- ▶ In the interest of patient safety, you and your patients are obliged to report serious events that occur in connection with this product to the manufacturer and to the authority responsible in your country.

#### **DANGER!** **Risk of explosion**

- ▶ Do not use the device in an environment in which one of the following gases has accumulated:
  - oxygen
  - flammable anesthetics
  - other flammable substances/air mixtures

#### **CAUTION!** **Patient hazard, damage to device**

- ▶ Additional devices connected to medical electrical devices must provide evidence of compliance with the relevant IEC or ISO standards (e.g. IEC 60950 for data-processing devices). Furthermore, all configurations must comply with the requirements of standards for medical systems (see IEC 60601-1-1 or Section 16 of edition 3.1 of IEC 60601-1 respectively). Anyone connecting additional devices to medical electrical devices is considered a system configurator and therefore responsible for ensuring that the system complies with the requirements of standards for systems. This also applies to additional devices recommended by seca. Your attention is drawn to the fact that local laws take precedence over the above-mentioned requirements of standards. In the event of any queries, please contact your local specialist dealer or Technical Service.

#### **CAUTION!** **Patient hazard, damage to device**

- ▶ Have servicing carried out regularly as described in the relevant section of this document.
- ▶ Technical modifications may not be made to the device. The device does not contain any user-serviceable parts. Only have servicing and repairs performed by an authorized seca service partner. You can find a service partner in your vicinity at [www.seca.com](http://www.seca.com) or by emailing [service@seca.com](mailto:service@seca.com).
- ▶ Use only seca original accessories and spare parts, otherwise seca will not grant any warranty.

#### **CAUTION!** **Patient hazard, malfunction**

- ▶ Keep other medical electrical devices, e.g. high-frequency surgical devices, a minimum distance of approx. 1 meter away to prevent faulty measurements or wireless transmission interference.
- ▶ Keep HF devices such as cellphones at a minimum distance of approx. 1 meter to prevent faulty measurements or wireless transmission interference.
- ▶ The actual transmission output of HF devices may require minimum distances of more than 1 meter. For details, go to [www.seca.com](http://www.seca.com).

## Preventing electric shock

### **WARNING!** **Electric shock**

- ▶ Set up devices which can be operated with a power supply unit so that the power supply socket is within easy reach and the power supply can be disconnected quickly.
- ▶ Ensure that your local electricity supply matches the details on the power supply unit.
- ▶ Never touch the power supply unit with wet hands.
- ▶ Do not use extension cables and multiple outlets.
- ▶ Ensure that cables are not crushed or damaged by sharp edges.
- ▶ Ensure that cables do not come into contact with hot objects.
- ▶ Do not operate the device at an altitude of more than 3000 m above sea level.
- ▶ Only devices that are approved as medical devices and that have no separate power supply may be connected to the USB interface.

## Preventing injuries and infections

### **WARNING!** **Injury from falling**

- ▶ Ensure that the device is steady and level.
- ▶ Route connector cables (if present) so that neither users nor the patient can trip over them.
- ▶ Assist people with limited mobility, e.g. when they are getting up from a wheelchair.
- ▶ Ensure that the patient does not step straight onto or off the edges of the patient standing area.
- ▶ Ensure that the patient steps onto and off the patient standing area slowly and safely.

### **WARNING!** **Risk of slipping**

- ▶ Ensure that the patient standing area is dry before the patient steps onto it.
- ▶ Ensure that the patient has dry feet before stepping onto the patient standing area.
- ▶ Ensure that the patient steps onto and off the patient standing area slowly and safely.

### **CAUTION!** **Injury, damage to device**

The patient standing area consists of a glass plate. Damage (e.g. scratches, cracks or chips) represent a risk of injury. Damage can also lead to the glass plate breaking.

- ▶ Do not put any sharp-edged objects on the glass plate.
- ▶ Before using each time, check the glass plate for scratches, cracks and chips. If you find damage of this kind, have the glass plate replaced with a new one.
- ▶ Do not use the device if the glass plate is damaged.

**WARNING!****Risk of infection**

- ▶ Before and after every measurement, wash your hands to reduce the risk of cross-contamination and nosocomial infections.
- ▶ Subject the device to a hygiene treatment at regular intervals as described in the relevant section of these instructions for use.
- ▶ Ensure that the patient does not have any infectious diseases.
- ▶ Ensure that the patient does not have any open wounds or infectious skin alterations which may come into contact with the device.

**Preventing damage to device****NOTICE!****Damage to device**

- ▶ Ensure that fluids never get inside the device. These can destroy the electronics.
- ▶ For devices with power supply operation: Switch off the device before you disconnect the power supply unit from the power supply socket.
- ▶ For devices with power supply operation: If the device is not to be used for an extended period, disconnect the power supply unit from the power supply socket. Only then is the device de-energized.
- ▶ For devices with battery or rechargeable battery operation: If you are not using the device for an extended period of time, remove batteries or rechargeable batteries. Only then is the device de-energized.
- ▶ Do not drop the device.
- ▶ Do not subject the device to shocks or vibrations.
- ▶ Perform a function check before each use as described in the corresponding section in this document. Do not operate the device if it is not working properly or is damaged.
- ▶ Do not place the device in direct sunlight and ensure that it is not placed in the direct proximity of a heat source. The excessive temperatures could damage the electronics.
- ▶ Avoid rapid temperature fluctuations. If the device is transported so that a temperature difference of over 20 °C occurs, the device must be left to stand for at least 2 hours before it is switched on, otherwise condensation may form; this may damage the electronics.
- ▶ Use the device only in the intended ambient conditions.
- ▶ Store the device only in the intended storage conditions.
- ▶ Use only cleaning agents and disinfectants which match the details in the section entitled "Hygiene treatment".
- ▶ For scales: Ensure that maximum capacity is not exceeded.

## Handling measuring results

### **CAUTION!** **Patient hazard**

To prevent misinterpretations, measuring results for medical purposes must only be displayed and used in SI units (kilograms/grams, meters/centimeters). Some devices have the option of displaying measuring results in different units. This is purely an additional function.

- ▶ Only use measuring results in SI units.
- ▶ The user takes sole responsibility for the use of measuring results in non-SI units.

### **NOTICE!** **Inconsistent measuring results**

- ▶ Before you save and continue using measured values determined using this device (e.g. in seca software or in an EMR system), ensure that the measured values are plausible.
- ▶ If measured values have been sent to seca software or to an EMR system, ensure before continuing to use them that the measured values are plausible and assigned to the correct patient.

## Handling packaging material

### **WARNING!** **Risk of suffocation**

Packaging materials made of plastic film (bags) present a risk of suffocation.

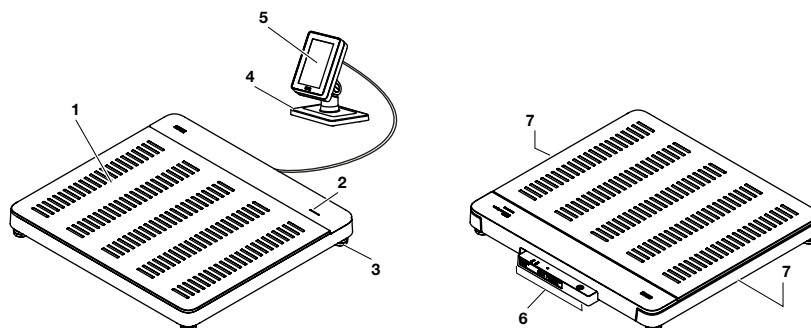
- ▶ Store packaging material out of the reach of children.
- ▶ If the original packaging material is no longer available, only use plastic bags with safety holes to reduce the risk of suffocation. Use recyclable materials if possible.

### **NOTE**

Store the original packaging material for future use (e.g. returning for servicing).

## 4 OVERVIEW

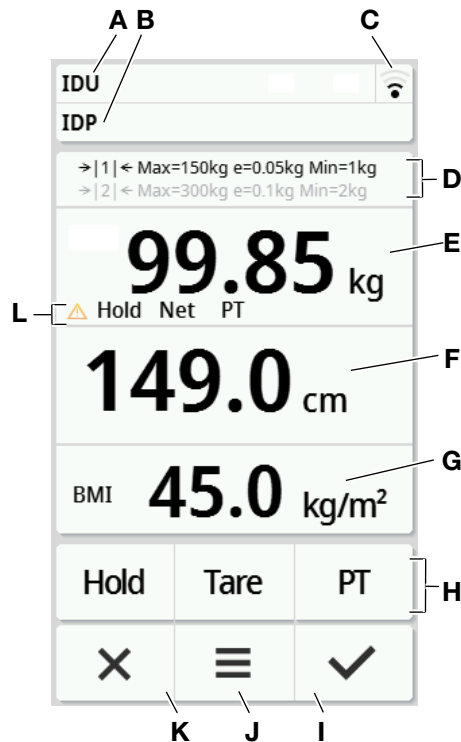
### 4.1 Controls

Item	Device component	Description
1	Weighing platform	Records the weight of the patient
2	Workflow LED	<p>Indicates the status of data recording and data transmission (requirement: Connection to the <b>seca connect 103</b> software)</p> <ul style="list-style-type: none"> <li>• Illuminated in green: Workflow active</li> <li>• Flashing green (approx. 5 seconds): Submitting measured results to the EMR System (depending on setting)</li> <li>• Illuminated in green (approx. 5 seconds): Measured results successfully submitted to the EMR System (depending on setting)</li> <li>• Illuminated in red (approx. 5 seconds): Error during data transmission or workflow</li> </ul> <p><b>NOTE</b></p> <p>The data recorded and transmitted are specified in the <b>seca connect 103</b> software. If you have any queries, contact your administrator or hospital technician.</p>
3	Foot screw	Used for precise alignment of the device (4 pcs)
4	Positioner	Used to position the multifunctional display on a table or to install it on a wall
5	Multifunctional display	<p>Central control and display element</p> <ul style="list-style-type: none"> <li>• → <a href="#">Symbols on the ID display (main screen), page 88</a></li> <li>• → <a href="#">Symbols on the ID display (menu), page 90</a></li> </ul>
6	Connection panel	Used for power supply and data transmission → <a href="#">Interfaces, page 95</a>
7	Recessed handle	Used for transporting over short distances (2 pcs)

## 4.2 Symbols on the ID display (main screen)

This section contains information about the display content in measuring mode. Information about the display content for configuration and administration is available here: → [Symbols on the ID display \(menu\)](#), page 90.

Item	Display element	Description
<b>A</b>	<b>IDU</b>	Only if connected to third-party EMR System (via <b>seca connect 103</b> ): <ul style="list-style-type: none"> <li>• <b>IDU</b>: User name</li> <li>• <b>IDP</b>: Name and date of birth of the patient</li> <li>• Press the area to display the IDs in enlarged form</li> </ul>
<b>B</b>	<b>IDP</b>	
<b>C</b>		WiFi connection status display: <ul style="list-style-type: none"> <li>•  WiFi unavailable</li> <li>•  WiFi activated, no signal</li> <li>•  Signal very weak</li> <li>•  Signal weak</li> <li>•  Signal good</li> <li>•  Signal optimal</li> </ul>

Item	Display element	Description
		<p>LAN connection status display:</p> <ul style="list-style-type: none"> <li> Not available</li> <li> Deactivated</li> <li> Activated</li> </ul>
<b>D</b>	Weighing technology data	<ul style="list-style-type: none"> <li>Weighing ranges: → 1 ←, → 2 ←</li> <li>Max: Maximum load per weighing range</li> <li>e: Graduations (verified models)</li> <li>d: Graduations (non-verified models)</li> <li>Min: Minimum load per weighing range</li> </ul>
<b>E</b>	<b>Weight</b> display field	<p>Body weight:</p> <p>Units:</p> <ul style="list-style-type: none"> <li>Kilograms</li> <li>Pounds (non-verified models)</li> </ul>
<b>F</b>	<b>Height</b> display field	<p>Height, press display for manual input:</p> <p>Units:</p> <ul style="list-style-type: none"> <li>Centimeters</li> <li>Feet/inch(es) (non-verified models)</li> </ul>
<b>G</b>	“Body Size Indicator” display field	<p>Body size indicator (press display field to switch):</p> <ul style="list-style-type: none"> <li><b>BMI</b>: Body Mass Index (kg/m<sup>2</sup>): Automatic calculation</li> <li><b>BSA (DuBois)</b>: Body Surface Area (m<sup>2</sup>): Automatic calculation</li> <li><b>BSA (Haycock)</b>: Body Surface Area (m<sup>2</sup>): Automatic calculation</li> <li><b>BSA (Mosteller)</b>: Body Surface Area (m<sup>2</sup>): Automatic calculation</li> </ul> <p>Body Size Indicator (activate/deactivate in menu Selecting BMI/BSA/Waist circumf.):</p> <ul style="list-style-type: none"> <li>: Enter waist circumference</li> </ul>
<b>H</b>	Additional functions	<ul style="list-style-type: none"> <li> Additional function activated (here: <b>Hold</b>)</li> <li> Additional function deactivated</li> </ul>
<b>I</b>		<ul style="list-style-type: none"> <li>Confirm measured results and send to EMR System</li> <li>Confirm manual input</li> </ul>
<b>J</b>		<ul style="list-style-type: none"> <li>Press briefly: Open menu (→ <a href="#">Configuration, page 112</a>)</li> <li>Press and hold (approx. 5 seconds): Changing device mode</li> </ul>

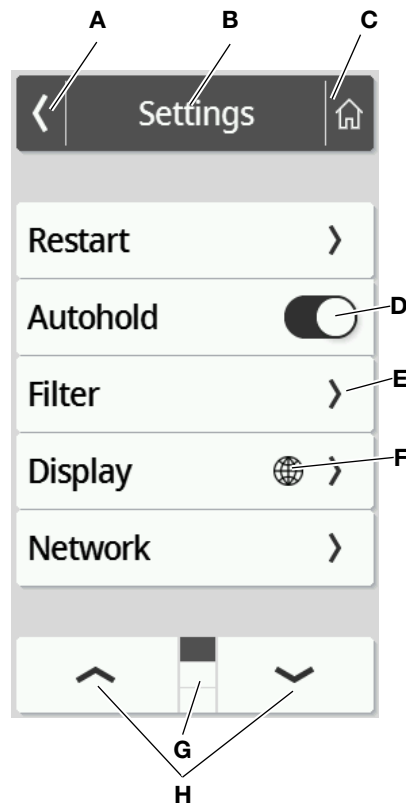
Item	Display element	Description
K	✕	<ul style="list-style-type: none"> <li>• Clear measured results</li> <li>• Clear manual input</li> <li>• Cancel automated procedures</li> </ul>
L	Extended weighing functions	<ul style="list-style-type: none"> <li>• : Non-verifiable function active</li> <li>• <b>Hold: Hold</b> function active</li> <li>• <b>NET: Tare</b> or <b>Pre-tare</b> function active</li> <li>• <b>PT: Pre-tare</b> function active</li> </ul>

### 4.3 Symbols on the ID display (menu)

This section contains information about the display content for configuration and administration. Information about the display content for measuring mode is available here: Symbols on the ID display (main screen).

The configuration options in the menu are dependent on the device mode selected:

- → [Changing device mode, page 112](#)
- → [Functions/device mode, page 144](#)

	Symbol	Description
A		Back to the higher menu level

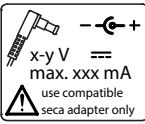
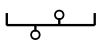
	Symbol	Description
B	<b>Header</b>	Indicates the current menu level
C		Back to the main screen
D		<ul style="list-style-type: none"> <li>Press: Activate/deactivate function</li> <li> Function activated</li> <li> Function deactivated</li> </ul>
E		<ul style="list-style-type: none"> <li>Submenu available</li> <li>Setting options available</li> </ul>
F		Keys with this symbol lead to the <b>Display\Language</b> menu item
G		<ul style="list-style-type: none"> <li>Pages per menu level; here: 3</li> <li> Current page is displayed; here: Page 1</li> </ul>
H		<ul style="list-style-type: none"> <li>Select page in the menu</li> </ul>

#### 4.4 Labels

Markings on the device and on the type plate	
Symbol	Meaning
	Name and address of manufacturer, date of manufacture
<b>UDI</b>	Unique Device Identifier (product identification number)
	Article number
	Serial number
<b>GAL</b>	Value in m/s <sup>2</sup> (model-dependent) <ul style="list-style-type: none"> <li>Gravitational acceleration on earth</li> <li>Depends on the intended location</li> </ul>
<b>ProdID</b>	Product identification number
<b>Approval Type</b>	Type designation of design approval
	Follow instructions for use
	Medical electrical device, Type BF

Markings on the device and on the type plate	
Symbol	Meaning
IP21	Type of protection to IEC 60529: <ul style="list-style-type: none"> <li>• Protection against ingress of solid foreign bodies with a diameter of over 12.5 mm</li> <li>• Protection against access with fingers</li> <li>• Protection against dripping water</li> </ul>
e	Value in units of mass (verified models) Used to classify and verify a scale
d	Value in units of mass (non-verified models) States the difference between two consecutive display values
→ x ←	Active weighing range
	Class III scale in accordance with 2014/31/EU
	Device complies with EU directives <ul style="list-style-type: none"> <li>• <b>M</b>: Conformity label in compliance with directive 2014/31/EU for non-automatic scales (verified models)</li> <li>• <b>22</b>: (Example: 2022) Year in which declaration of conformity was implemented and the CE label was applied (verified models)</li> <li>• <b>0102</b>: Notified Body for Metrology (verified models)</li> <li>• <b>0123</b>: Notified Body for Medical Devices</li> </ul>
	Medical device in accordance with Regulation (EU) 2017/745
	Device meets the requirements of the USA and Canada. Certified and tested by a licensing laboratory (NRTL) of TÜV SÜD Product Services GmbH.
	Device complies with United Kingdom directives <ul style="list-style-type: none"> <li>• <b>M</b>: Conformity label in compliance with UK directive SI 2016 no. 1152 for non-automatic scales (NAWIR)(verified models)</li> <li>• <b>22</b>: (Example: 2022) Year in which declaration of conformity was implemented and the UKCA label was applied (verified models)</li> <li>• <b>xxxx</b>: Notified Body for Medical Devices of the United Kingdom</li> <li>• <b>yyyy</b>: Notified Body for Metrology of the United Kingdom (verified models)</li> </ul>
	Importer/representative in the United Kingdom:  seca Ltd 40 Barn Street B5 5QB Birmingham United Kingdom

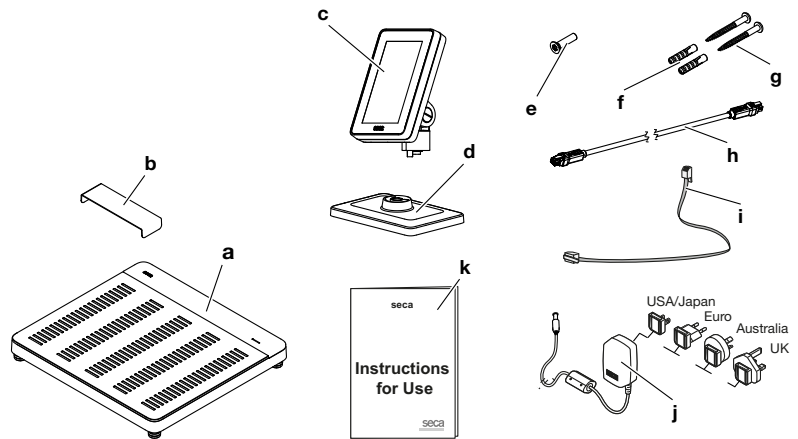
Markings on the device and on the type plate	
Symbol	Meaning
	Importer/representative in Switzerland:  seca ag (schweiz) Medizinische Waagen und Messsysteme Schönmatt Str. 4 CH-4153 REINACH
	Symbol of the US Federal Communications Commission (FCC)
<b>FCC ID</b>	Device license number from the US Federal Communications Commission (FCC)
<b>IC ID</b>	Device license number from Industry Canada
	Type plate on the power supply connection socket <ul style="list-style-type: none"> <li>• Power supply voltage required in V</li> <li>• Maximum current consumption in mA</li> <li>• : Note polarity of device connector</li> <li>• : Operate device with direct current</li> <li>• : Use compatible seca power supply units only</li> </ul>
	LAN interface
	USB interface
	Power LED
	Network LED
	WPS button
	Reset button
	Interface for multifunctional display
	Do not dispose of device in household waste

Markings on the packaging	
Symbol	Meaning
	Protect from moisture
	Arrows indicate top of product  Transport and store in an upright position

Markings on the packaging	
Symbol	Meaning
	Fragile, do not throw or drop
	Permitted min. and max. temperature for transport and storage
	Permitted min. and max. humidity for transport and storage
	Permitted min. and max. air pressure for transport and storage
	Open packaging here
	Packaging material can be disposed of through recycling programs

## 5 STARTING UP DEVICE


### 5.1 Scope of delivery

Item	Component	Pcs.
a	Scale	1
b	Drip guard, transparent	1
c	Multifunctional display	1
d	Positioner	1
e	Hex head socket screw, M4x25	1
f	Wall plug, Ø 5 mm	2

Item	Component	Pcs.
g	Slot-head screw, 4x40 mm	2
h	Display cable	1
i	Network cable	1
j	Plug-in power supply unit with adapters	1
k	Instructions for use	1
–	Hex socket wrench, 2.5 mm	1

## 5.2 Interfaces

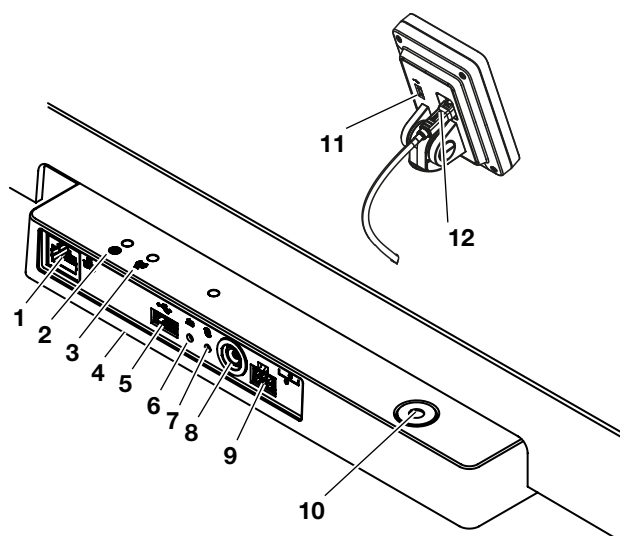
Item	Device component	Function
1	LAN interface	Used to connect the device to the EMR System in your institution (alternative to WiFi connection, <b>seca connect 103</b> software required).
2	Power LED	<ul style="list-style-type: none"> <li>• Illuminated in green: Device is ready for use</li> <li>• Illuminated in red: Device is defective</li> <li>• Flashing green: Device is active as access point</li> </ul>
3	Network LED	<ul style="list-style-type: none"> <li>• Flashing green: Establishing network connection</li> <li>• Illuminated in green: Network connection established</li> <li>• Illuminated in red: Network connection interrupted</li> </ul>
4	WiFi module (internal)	Used to connect the device to the EMR System in your institution (alternative to LAN connection, <b>seca connect 103</b> software required)
5	USB interface, weighing platform	Used to connect a barcode scanner → <a href="#">Optional accessories and spare parts, page 148</a>
6	Reset button	<ul style="list-style-type: none"> <li>• Press and hold (approx. 8 seconds): Reset settings</li> <li>• Press briefly (approx. 1 second): Activate/deactivate access point function</li> </ul>

Item	Device component	Function
7	WPS button	Establishing WiFi connection via WPS
8	Power supply connection	Used to connect the plug-in power supply unit
9	Display socket	Used to connect the multifunctional display
10	Spirit level	Indicates whether the device is horizontal
11	USB interface, multifunctional display	No function in this device variant, use USB interface on weighing platform
12	Display interface	<ul style="list-style-type: none"> <li>Used to supply the multifunctional display with power</li> <li>Used for data transmission between the multifunctional display and the device electronics</li> </ul>

### 5.3 Setting up device

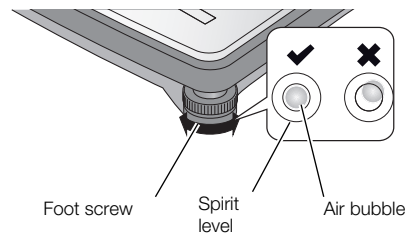To achieve accurate measured results, the floor at the setup location must be level and stable. Soft floors (wooden boards, for example) give under the patient's weight and falsify the measured result.

1. Place the device on a firm, level surface.
2. Align the device by turning the foot screws.


⇒ The device is positioned horizontally when the air bubble of the spirit level is in the precise center of the circle.

**CAUTION!**  
**Injury from a lack of stability**

If the foot screws are screwed out too far, they may come loose from the device. The device will then be unstable.

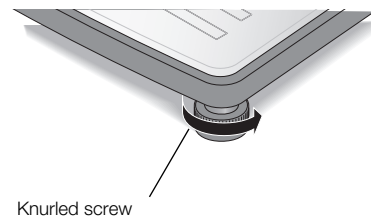
- ▶ Screw the foot screws out a maximum of 10 mm.
- ▶ If the device cannot be aligned horizontally with the screws screwed out as far as possible, the setup location is unsuitable. Set the device up in a suitable location.

3. Tighten the knurled wheels in the direction of the arrow.  
 ⇒ The foot screws are secured against being adjusted.

## 5.4 Fitting the multifunctional display to the positioner

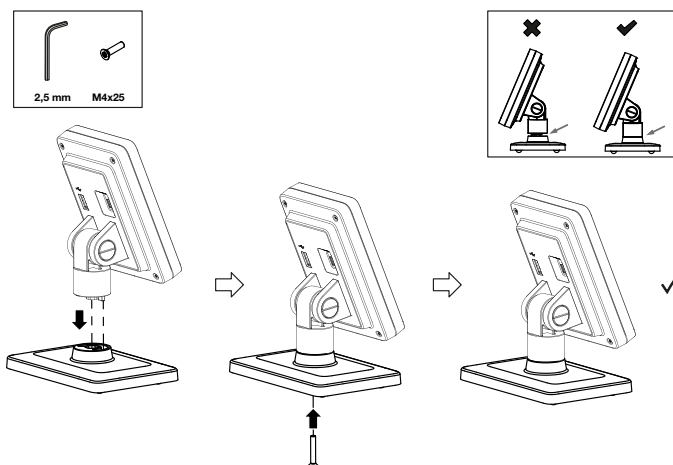1. Screw the multifunctional display onto the positioner as shown in the illustration.

2. You have the following options for continuing:
  - ▶ Place the multifunctional display on a horizontal, non-slip surface
  - ▶ Install the multifunctional display on the wall
3. Connect the multifunctional display to the weighing scale → [Connecting the multifunctional display to the scale, page 98.](#)


## 5.5 Installing the multifunctional display on the wall
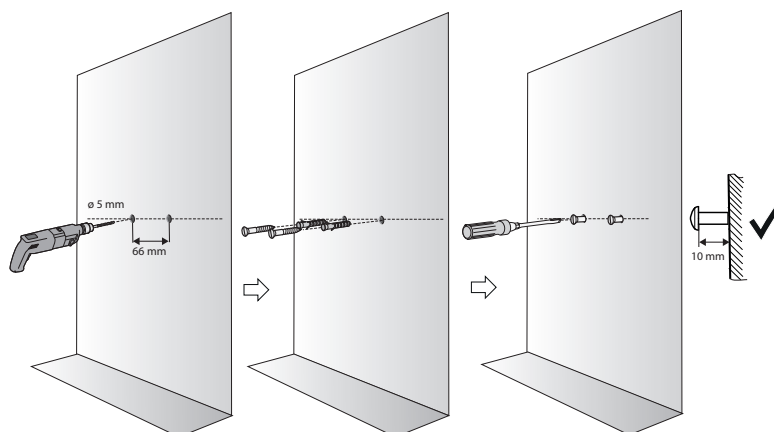
### NOTE


The supplied wall plugs are intended for mounting on solid walls. Special wall plugs may be required for other wall types.

- ▶ Note the length of the display cable before determining the height of the installation point.

1. Screw the multifunctional display to the positioner.
2. Mark the drill holes at the required height.
3. Drill the holes ( $\varnothing$  5 mm, approx. 35 mm deep).
4. Insert the wall plugs into the drill holes.
5. Insert the supplied screws into the wall plugs.
6. Tighten the screws until the screw shank protrudes approx. 10 mm out of the wall (see illustration).

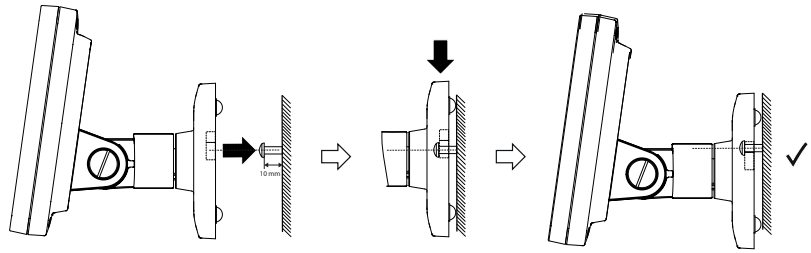7. Place the multifunctional display with the positioner onto the screws (see illustration).

8. Connect the multifunctional display to the weighing platform.


## 5.6 Connecting the multifunctional display to the scale
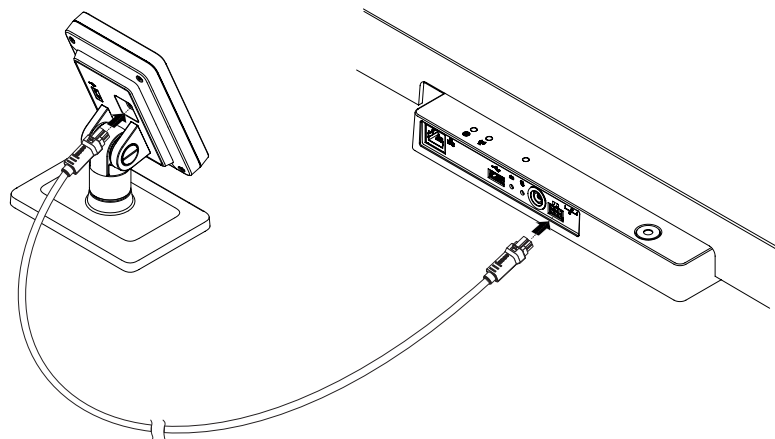
### NOTICE!

#### Faulty measurement as a result of force shunt

If the multifunctional display cable touches the weighing platform, the weight of the patient cannot be measured correctly.

- ▶ Route the cable so that it cannot touch the weighing platform.
- ▶ Connect the display cable to the scale and to the multifunctional display as shown in the illustration.

## 5.7 Connecting a barcode scanner (optional)

A barcode scanner can be connected to the USB interface of the weighing platform.

The barcode scanner is required for the following functions:

- **Configuration:** Define network data in the **seca connect 103** software and transmit them to the device using the QR code: → [Setting up network functions, page 123](#)
- **Operation:** Record patient and user IDs for transmitting measured results to an EMR System: → [Completing the measurement, page 110](#)

### ⚠ WARNING! Injury

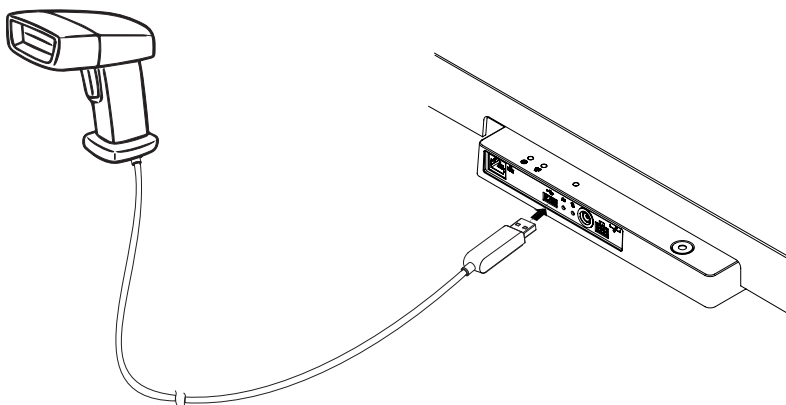
- ▶ Route the connector cable so that patients cannot become caught or strangle themselves in it.
- ▶ Route the connecting cable so as to prevent patients and users tripping.

**NOTE**

- ▶ Observe the maximum permitted current consumption of the barcode scanner.
- ▶ Use only barcode scanners recommended by seca.
- ▶ The device is compatible with NFC/RFID scanners. For details, contact seca Service.

To connect a barcode scanner, proceed as follows:

1. Ensure that the device is disconnected from the power supply.
2. Plug the USB connector of the barcode scanner into the USB socket of the weighing platform.

3. Hang the barcode scanner in a suitable holder.

**NOTICE!****Faulty measurement**

The barcode scanner and scanner holder are in the weighing-sensitive area of the device. If the barcode scanner is not replaced in the scanner holder after scanning, the measured result will be falsified.

- ▶ Place the scanner back in the scanner holder after each scanning procedure.
4. Establish the power supply → [Establishing power supply, page 99](#).

## 5.8 Establishing power supply

---

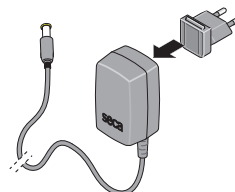
The device is supplied with power by a plug-in power supply unit.

**WARNING!**


**Personal injury or damage to the device as a result of incorrect power supply units**

Conventional power supply units may deliver a higher voltage than is indicated on them. The measuring device may overheat, catch fire, melt or short-circuit.

- ▶ Use only original seca power supply units with a controlled 12 V output voltage.
1. Plug the adapter required for your power supply into the power supply unit.

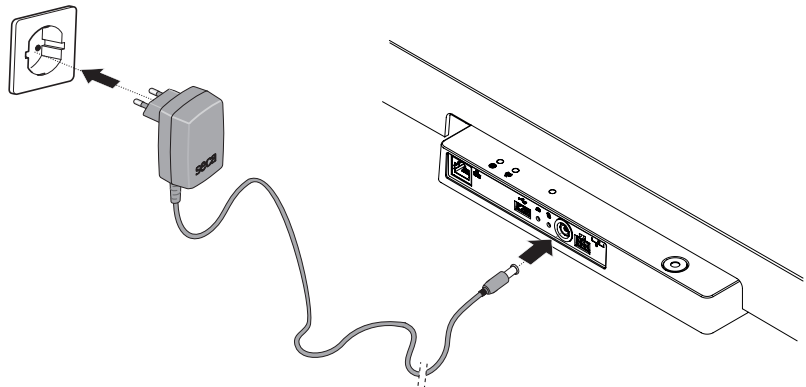2. Insert the device connector of the power supply unit into the power supply connection socket of the device.
3. Plug the power supply unit into a power supply socket.

4. Perform a function check → [Function check, page 132](#).

## 5.9 Installing the drip guard

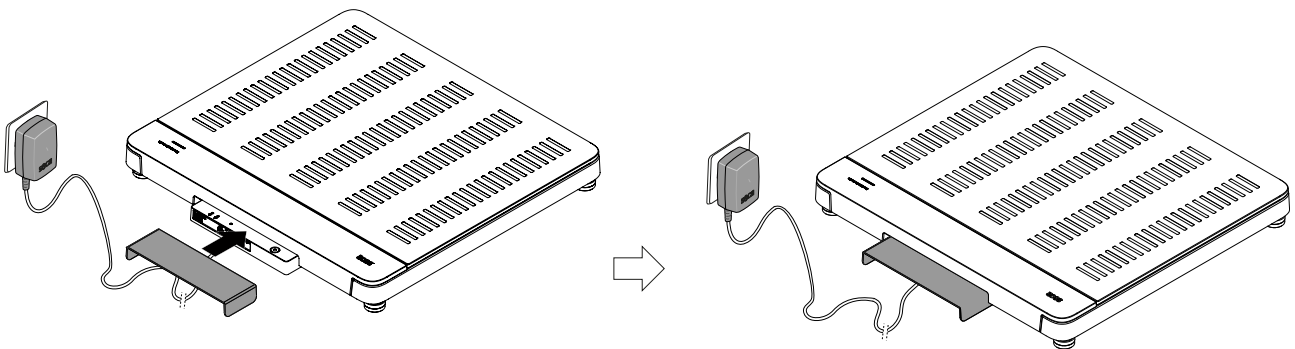
### NOTICE!

#### Damage to device due to the ingress of fluids

Damage may occur to the device if dripping water or other dripping fluids enter the device via the connection panel.

- ▶ Only operate the device with the drip guard installed.

1. Connect all cables as described in these instructions for use and in the installation instructions for the compatible products.
2. Position the drip guard on the connection panel as shown in the illustration.
3. Push the drip guard beneath the weighing platform up to the stop.

## 5.10 Adapting device settings

You have the following options for setting the device up for different usage situations:

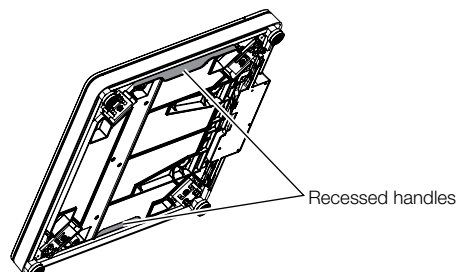
- → [Changing device mode, page 112](#)
- → [Calling up/exiting a menu, page 113](#)
- → [Setting up network functions, page 123](#)

## 5.11 Transporting device

The device is equipped with recessed handles which enable transport over short distances.

1. Remove the drip guard.
2. Disconnect all the device cable connections (e.g. power supply, network).
3. Raise the device by reaching into the recessed handles.

4. Transport the device to its new setup location.
5. Set up the device → [Setting up device, page 96](#).
6. Re-establish all cable connections.
7. Install the drip guard.

## 6 OPERATION

### **WARNING!** **Injury from falling**

- ▶ Ensure that the device is steady and level.
- ▶ Route connector cables (if present) so that neither users nor the patient can trip over them.
- ▶ Assist people with limited motor skills when they are getting up, e.g. from a wheelchair.
- ▶ Ensure that the patient does not step directly onto or off the edges of the weighing platform.
- ▶ Ensure that the patient steps onto and off the weighing platform slowly and safely.

### **WARNING!** **Danger of slipping**

- ▶ Ensure that the weighing platform is dry before the patient steps onto it.
- ▶ Ensure that the patient has dry feet before stepping onto the weighing platform.
- ▶ Ensure that the patient steps onto and off the weighing platform slowly and safely.

### **NOTE**

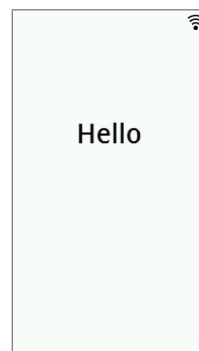
The availability of some functions is dependent on device mode. If you require functions that are not available in the current device mode, ask your administrator or hospital technician whether the device mode can be changed.

## 6.1 Starting the measurement procedure

Device mode	Function available
Basic	•
Advanced	•
Expert	•
Service	•

### Activating the multifunctional display (stand-alone operation)

The multifunctional display will switch to standby mode (→ [Setting standby time, page 120](#)) after a set time period. The following screensaver is displayed in stand-alone operation:


Screensaver for stand-alone operation

1. Press the multifunctional display to activate it.  
⇒ The main screen is displayed.
2. Ask the patient to step onto the weighing platform.
3. Complete the measurement as described in the relevant sections of these instructions for use.

### Activating the multifunctional display (network connection)

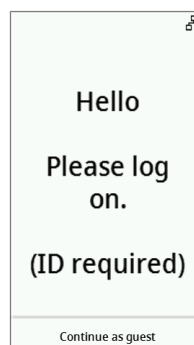The multifunctional display will switch to standby mode (→ [Setting standby time, page 120](#)) after a set time period. The following screensaver is displayed with a network connection:

Screensaver for network connection

1. Press the multifunctional display to activate it.  
⇒ The main screen is displayed.
2. Scan your ID and the ID of the patient.

**NOTE**

If you press the **Continue as guest** key, you can start the measurement immediately and scan the IDs later.

3. Ask the patient to step onto the weighing platform.
4. Complete the measurement as described in the relevant sections of these instructions for use.

**Using a reserved device (network connection)**

You can reserve the device (from firmware version 1.3) for your patient in the **seca analytics 125** software (depending on version). The name of the patient is displayed on the reserved device:

1. Ask the patient to step onto the weighing platform.  
⇒ The main screen is displayed.
2. Complete the measurement as described in the relevant sections of these instructions for use.

## 6.2 Measuring weight

Device mode	Function available
Basic	•
Advanced	•
Expert	•
Service	•

**CAUTION!**  
Injury from falling

Persons with limited mobility may fall when stepping onto the weighing platform.

► Support people with limited mobility when they step onto the scale.

1. Make sure that there is no load on the weighing platform.
2. Ask the patient to step onto the weighing platform.
3. Ask the patient to keep still.
4. Read off the measured result.

## 6.3 Entering height manually

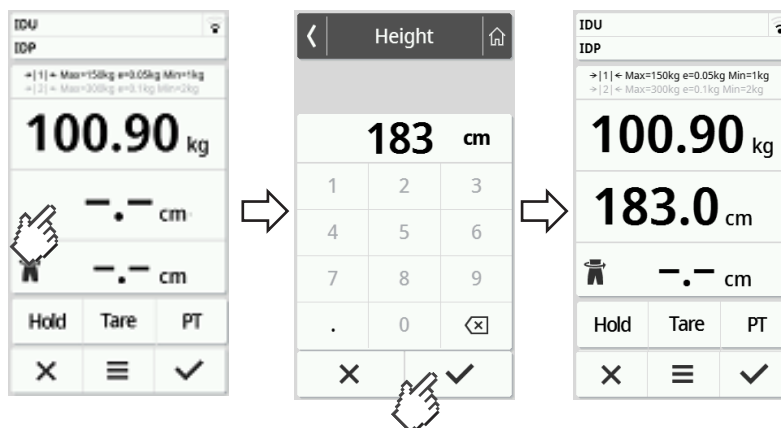
Device mode	Function available
Basic	•
Advanced	•
Expert	•
Service	•

**CAUTION!**  
Patient hazard

To prevent misinterpretations, measured results for medical purposes must only be displayed and used in SI units (weight: kilograms/grams, height: meters/centimeters). Some devices have the option of displaying measured results in different units. This is purely an additional function.

- Only use measured results in SI units.
- The user takes sole responsibility for the use of measured results in non-SI units.

1. Press the **Height** field.
2. Enter the height.
3. Press the ✓ key to confirm your entry.

4. Press the ✕ key to clear your entry.

## 6.4 Using extended weighing functions

### Taring additional weight (Tare)

Device mode	Function available
Basic	–
Advanced	•
Expert	•
Service	•

Use the **Tare** function to prevent an additional weight (e.g. a towel) from affecting the patient's weight.

#### NOTICE!

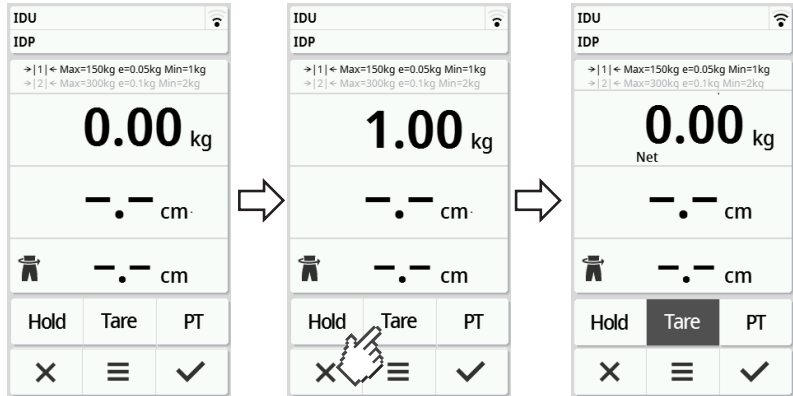
##### Faulty measurement as a result of force shunt

If an additional weight, (e.g. a large towel) contacts the surface on which the scale is placed, weight will not be measured correctly.


- Make sure that additional weights are only placed on the scale's weighing platform.

✓ There is no load on the scale.

1. To activate the **Tare** function, proceed as follows:
  - a) Place an additional weight (here: 1 kg) on the weighing platform.
  - b) Press the **Tare** key.
  - c) Wait until the value **0.00** and the message **NET** are displayed.

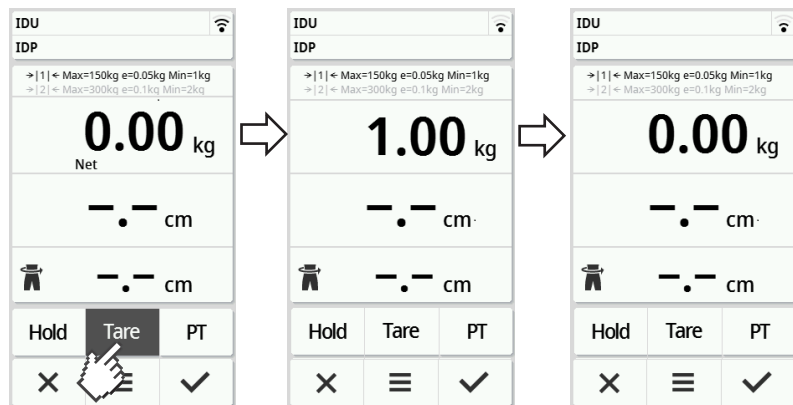2. Weigh the patient.
3. Read off the measured result.  
⇒ The additional weight is deducted automatically.
4. To deactivate the function, proceed as follows:
  - d) Remove the weight from the weighing platform.
  - e) Press the **Tare** key.
  - f) Wait until the message **NET** goes off and the additional weight is displayed.
  - g) Remove the additional weight from the weighing platform.

**NOTE**

The maximum weight which can be displayed is reduced by the weight of the objects which have been tared.

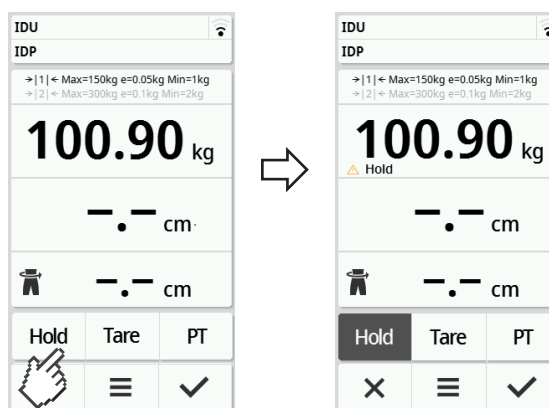
## Permanently displaying the weight (Hold)

Device mode	Function available
Basic	–
Advanced	•
Expert	•
Service	•

When the **Hold** function is activated, weight continues to be displayed after the weight has been removed from the scale. This enables you to attend to the patient before recording the weight.

✓ There is no load on the scale.

1. Ask the patient to step onto the scale.
2. Press the **Hold** key.
3. Wait until the weight has stopped flashing.
  - ⇒ The message **Hold** appears.

4. To deactivate the function, press the **Hold** key.
  - ⇒ The **Hold** message is no longer displayed.

### NOTE

- If the **Autohold** function is activated, weight and height are automatically displayed permanently as soon as stable measured values have been achieved (→ [Activating Autohold function, page 114](#)).
- If you wish to update measured values (weight and height), press the **Weight** display field or press the **Hold** key again. The measurement is repeated and the updated measured values are displayed permanently.

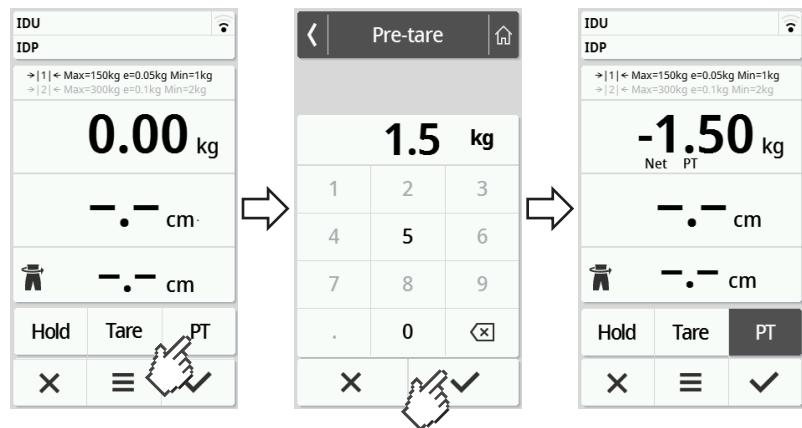
## Permanently store additional weight (Pre-Tare)

Device mode	Function available
Basic	–
Advanced	•
Expert	•
Service	•

The **Pre-tare** function can be used to save an additional weight permanently and subtract it from a measured result automatically, e.g. a flat-rate figure for shoes and clothing.

### Activate Pre-tare function

1. Remove the weight from the weighing platform.
2. Press the **PT** key.
3. Enter the value.
4. Confirm the value with the ✓ key.
  - ⇒ The set additional weight (here: 1.5 kg) is displayed with a minus sign in front.
  - ⇒ The messages **NET** and **PT** are displayed.

5. Ask the patient to step onto the scale.
  - ⇒ The patient's weight is displayed.
  - ⇒ The saved additional weight has been deducted automatically.

### Deactivate Pre-tare function

1. Remove the weight from the weighing platform.
2. Press the **PT** key.
3. Clear the value with the ✕ key.
  - ⇒ The set additional weight is no longer displayed.
  - ⇒ The function is deactivated.

## Switching weighing range

After the scale is switched on, weighing range 1 is active. If a particular weight is exceeded, the scale automatically switches to weighing range 2.

► To switch back to weighing range 1, completely remove the weight from the scale.

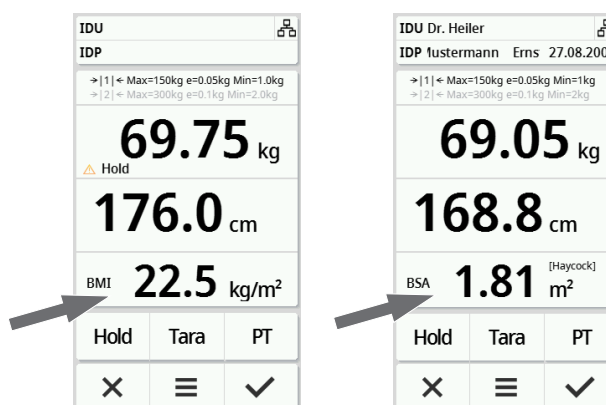
⇒ Weighing range 1 is active again.

## Automatic calculation of BMI or BSA

Device mode	Function available
Basic	•
Advanced	•
Expert	•
Service	•

The device automatically calculates Body Mass Index (**BMI**) or Body Surface Area (**BSA**) depending on the default setting (→ [Selecting BMI/BSA/Waist circumf.](#), page 116).


1. Determine the patient's weight (→ [Measuring weight](#), page 104).
  2. Determine the patient's height (→ [Entering height manually](#), page 104).
- ⇒ Either **BMI** or **BSA** is displayed depending on the default setting:

## Entering waist circumference

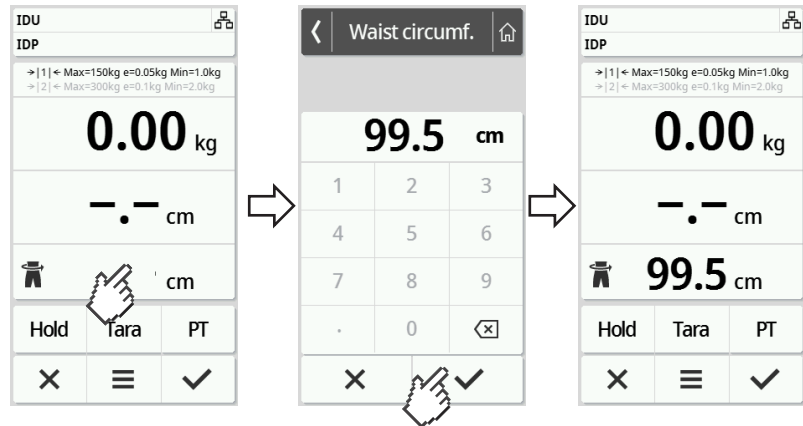If the  symbol is visible on the main screen, you can enter the patient's waist circumference. The waist circumference entry can be configured → [Selecting BMI/BSA/Waist circumf.](#), page 116.

1. Press the  display field.
2. Enter the waist circumference.
3. Confirm the entry with the  key.  
⇒ Waist circumference is displayed on the main screen.

## 6.5 Completing the measurement

### Stand-alone operation

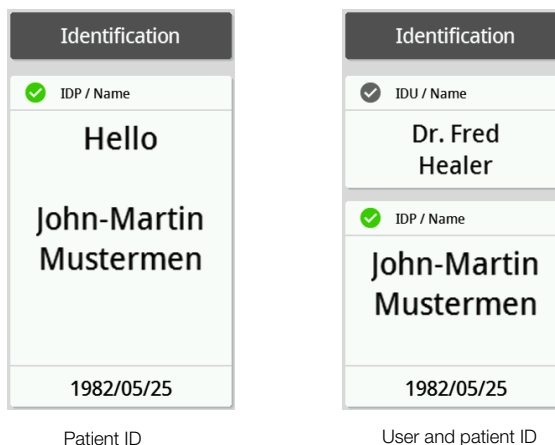
To complete a measurement on devices that are not connected to an EMR System or the **seca analytics 125** software, proceed as follows:

1. Make sure that the displayed measured values are plausible.
2. Transfer the displayed measured values to the patient file manually.
3. Ask the patient to step off the weighing platform.
4. Press the  key.  
⇒ Measured values and manual entries are discarded.  
⇒ The device is ready for the next measurement.

## Devices with connection to an EMR System

To complete a measurement on devices connected to an EMR System, proceed as follows:


1. Make sure that the displayed measured values are plausible.
2. Press the ✓ key.
3. Scan the patient and/or user ID.
  - ⇒ The device indicates whether the scanned IDs are correct:

### NOTE

Whether IDs have to be scanned and at what point during the measurement is defined when connecting the device to your EMR System. If you have any queries in this regard, contact your administrator or hospital technician.

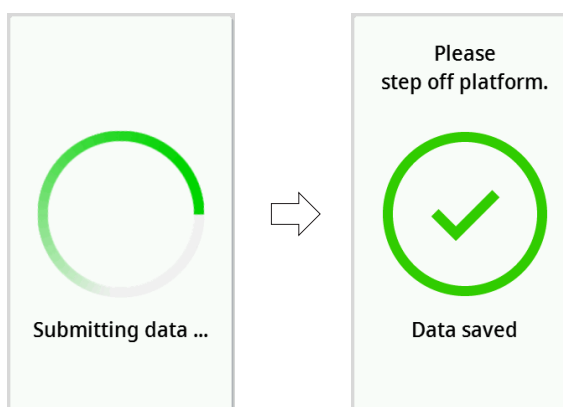4. Press the ✓ key.
  - ⇒ The measured results are submitted to the EMR System and are assigned to the electronic patient file.


5. Ask the patient to step off the weighing platform.
  - ⇒ The device is ready for the next measurement.

## Devices with a connection to the seca analytics 125 software

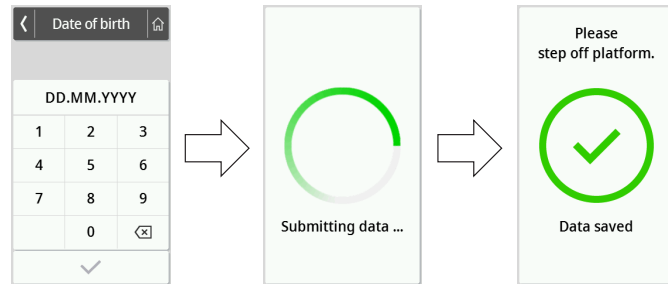To complete a measurement on devices connected to the **seca analytics 125** software, proceed as follows:

1. Press the ✓ key.  
⇒ The **Date of birth** dialog window appears.
2. Enter the patient's date of birth.
3. Press the ✓ key.  
⇒ The measured results are submitted to the **seca analytics 125** software.

4. Ask the patient to step off the weighing platform.  
⇒ The device is ready for the next measurement.

## 7 CONFIGURATION

### 7.1 Basic functions

#### Changing device mode

The following device modes are available for setting the device up for different usage situations:

Mode	Functions	Use	Recommended user group
<b>Basic</b>	<ul style="list-style-type: none"> <li>• Measurement functions:                             <ul style="list-style-type: none"> <li>– Perform a measurement</li> <li>– Read off results</li> </ul> </li> <li>• Menu:                             <ul style="list-style-type: none"> <li>– Restart the device</li> </ul> </li> </ul>	Guided measurements	Hospital personnel
<b>Advanced</b>	<ul style="list-style-type: none"> <li>• Measurement functions:                             <ul style="list-style-type: none"> <li>– Perform a measurement</li> <li>– Read off results</li> <li>– Use additional functions</li> </ul> </li> <li>• Menu:                             <ul style="list-style-type: none"> <li>– Limited device configuration</li> </ul> </li> </ul>	Guided measurements	Hospital personnel

Mode	Functions	Use	Recommended user group
<b>Expert</b>	<ul style="list-style-type: none"> <li>• Measurement functions: <ul style="list-style-type: none"> <li>– Perform a measurement</li> <li>– Read off results</li> <li>– Use additional functions</li> </ul> </li> <li>• Menu: <ul style="list-style-type: none"> <li>– Configure the device</li> <li>– Configure network connection</li> </ul> </li> </ul>	<ul style="list-style-type: none"> <li>• Guided measurements</li> <li>• Configure the device</li> <li>• Network connection</li> </ul>	<ul style="list-style-type: none"> <li>• Hospital personnel</li> <li>• Hospital technicians</li> <li>• IT administrators</li> </ul>
<b>Service</b>	<ul style="list-style-type: none"> <li>• Measurement functions: <ul style="list-style-type: none"> <li>– Perform a measurement</li> <li>– Read off results</li> <li>– Use additional functions</li> </ul> </li> <li>• Menu: <ul style="list-style-type: none"> <li>– Configure the device</li> <li>– Configure network connection</li> <li>– Additional service functions</li> </ul> </li> </ul>	Service	Authorized service technicians

To select a device mode, proceed as follows:

1. Press and hold the  key (approx. 5 sec.) until the **Device mode** menu appears.
  - ⇒ The current device mode is displayed.
2. Press the desired device mode.
  - ⇒ The function is active.
3. Press the  key in the header.
  - ⇒ The main screen is displayed.

### Calling up/exiting a menu

1. To call up the menu, press the  key.
  - ⇒ The **Settings** menu is displayed.
2. To exit the menu, press the  key.
  - ⇒ The main screen is displayed.

#### NOTE

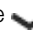
The setting options available in the menu depend on the product variant/combination of products being used. The menu of your device may have a smaller scope than that shown in these instructions for use.


## Using the PDF version of the instructions for use (QR code)

Device mode	Function available
Basic	–
Advanced	•
Expert	•
Service	•

You can scan a QR code which can be used to access the PDF version of these instructions for use and load them e.g. onto your smartphone or tablet PC.

To scan the QR code, proceed as follows:

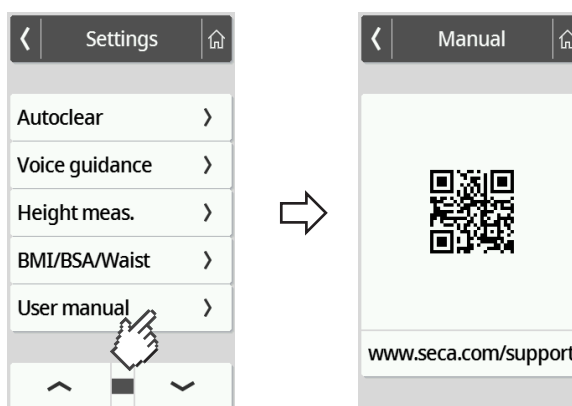1. Press the  key.  
⇒ The **Settings** menu is displayed.
2. Press the  or  key until the **User manual** menu item is displayed.
3. Press the **User manual** item.  
⇒ A QR code is displayed.

4. Scan the QR code (e.g. with your smartphone or tablet).  
⇒ This takes you to the [www.seca.com/support](http://www.seca.com/support) website where you can download the instructions for use.

## 7.2 Measuring

### Activating Autohold function

Device mode	Function available
Basic	–
Advanced	•
Expert	•
Service	•

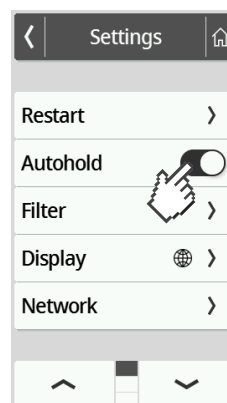
If you activate the **Autohold** function, it is no longer necessary to activate the **Hold** function manually for each individual measurement.

On devices with a measuring rod, the setting also applies to the display of height.

#### NOTE

This function is activated at the factory on some models. The function can be deactivated.

1. Press the  key.  
⇒ The **Settings** menu is displayed.

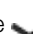
2. Press the  or  key until the **Autohold** menu item is displayed.
3. Select the desired setting:
  -  Function activated
  -  Function deactivated
4. To exit the menu, press the  key.


## Setting filter

Device mode	Available
Basic	–
Advanced	•
Expert	•
Service	•

The **Filter** function can be used to avoid interference during weight determination. The selected setting has the following effects on measurements with the **Hold/Autohold** function activated:

- Sensitivity with which the weight display reacts to patient movements
- Time period until a weight is displayed permanently

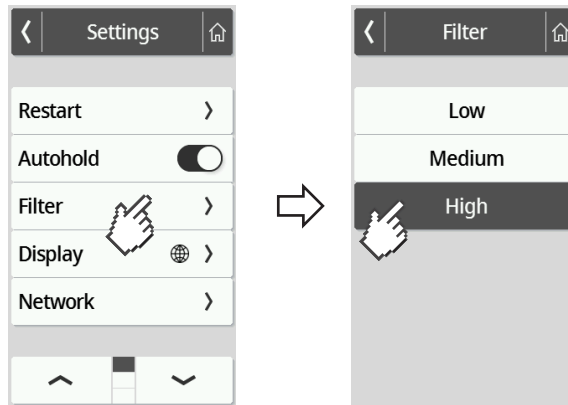1. Press the  key.  
⇒ The **Settings** menu is displayed.
2. Press the  or  key until the **Filter** menu item is displayed.
3. Press the **Filter** item.  
⇒ The current setting is displayed.

4. Press the desired filter stage.  
⇒ The setting is active.

Settings	Weight determination
Low	Fast
Medium	Medium
High	Slow

#### NOTE

With the **Low** setting and patients who are not very steady on their feet, it is possible that no weight will be displayed permanently despite the **Hold** function being activated.

### Selecting BMI/BSA/Waist circumf.

Device mode	Function available
Basic	–
Advanced	–
Expert	•
Service	•

You can specify whether the device calculates Body Mass Index (**BMI**) or Body Surface Area (**BSA**) as soon as the patient's weight and height are available.

You can also set the device so that it does not display the **BMI** or **BSA** value calculated, but waist circumference **Waist circumf.** can be entered manually.

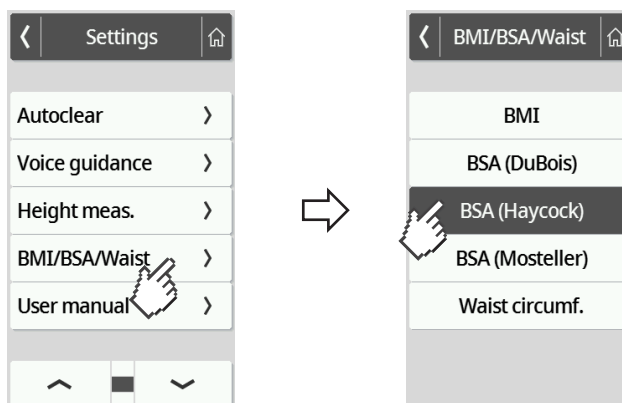
#### NOTE

If the **Waist circumf.** setting is selected, automatic BMI/BSA calculation is not possible.

1. Press the key.  
⇒ The **Settings** menu is displayed.
2. Press the or key until the **BMI/BSA/Waist** menu item is displayed.
3. Press the **BMI/BSA/Waist** item.

- Press the desired setting:  
⇒ The setting is active.



## 7.3 Adjusting display settings

### Setting display language

Device mode	Function available
Basic	–
Advanced	•
Expert	•
Service	•

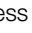The display language can be set.

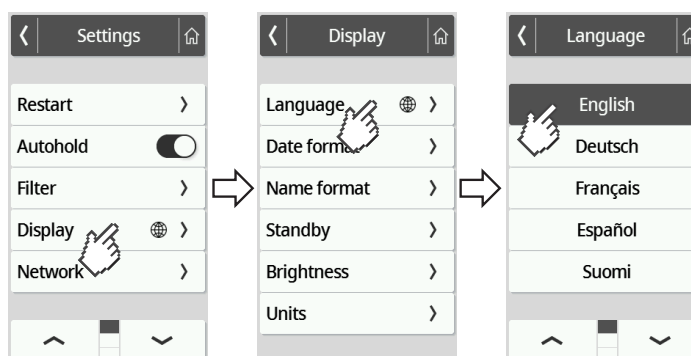- Press the  key.  
⇒ The **Settings** menu is displayed.
- Press the  or  key until the **Display** menu item is displayed.
- In the **Display** menu, select the **Language** item.
- Press the desired language.  
⇒ The setting is active.

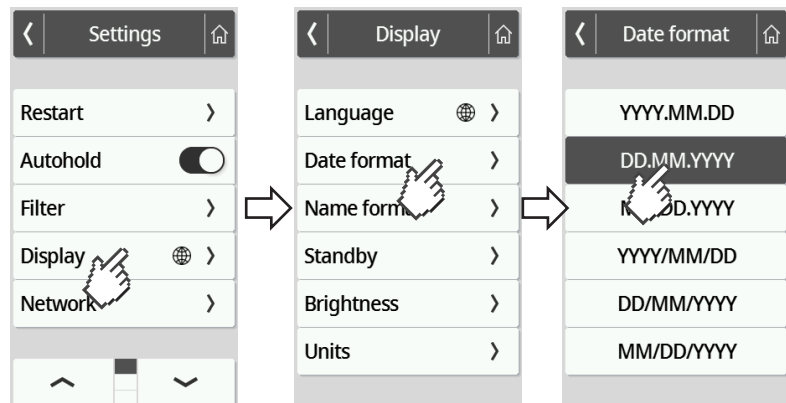
- To exit the menu, press the  key.

## Setting date format

Device mode	Function available
Basic	–
Advanced	•
Expert	•
Service	•

The format in which the patient's date of birth is displayed can be set.

1. Press the  key.  
⇒ The **Settings** menu is displayed.
2. Press the  or  key until the **Display** menu item is displayed.
3. In the **Display** menu, select the **Date format** item.
4. Press the desired date format.  
⇒ The setting is active.

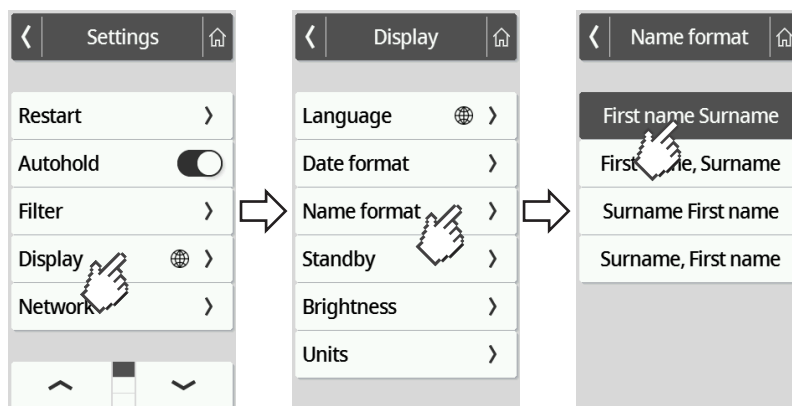
5. To exit the menu, press the  key.

## Setting name format

Device mode	Function available
Basic	–
Advanced	•
Expert	•
Service	•

The format in which the names of patients and users are displayed can be set.

1. Press the  key.  
⇒ The **Settings** menu is displayed.
2. Press the  or  key until the **Display** menu item is displayed.
3. In the **Display** menu, select the **Name format** item.
4. Press the desired name format.  
⇒ The setting is active.

5. To exit the menu, press the  key.

## Setting standby time

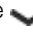
Device mode	Function available
Basic	–
Advanced	•
Expert	•
Service	•


The time period after which the multifunctional display goes to standby mode can be set.

### **WARNING!** Electric shock

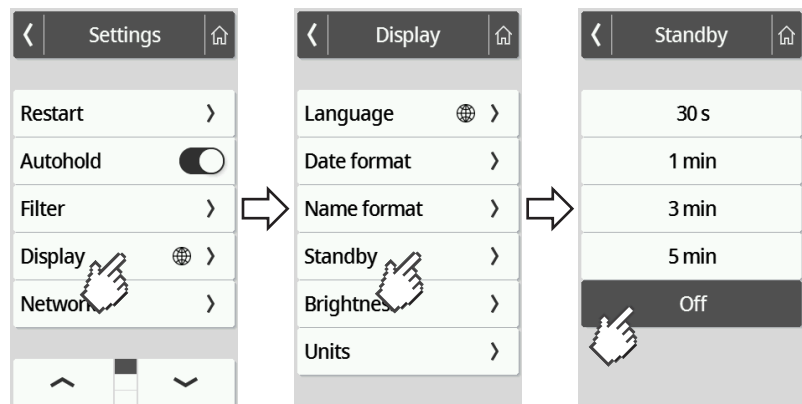The device is not de-energized when the display goes off.

- ▶ The device is not equipped with an on/off switch. Remove the power supply connector if the device needs to be de-energized, e.g. for hygiene treatment or maintenance work.

1. Press the  key.  
⇒ The **Settings** menu is displayed.
2. Press the  or  key until the **Display** menu item is displayed.
3. In the **Display** menu, select the **Standby** item.
4. Press the desired setting.  
⇒ The setting is active.

### **NOTE**

If the **Off** setting is selected in the **Standby** menu, the multifunctional display remains permanently active.

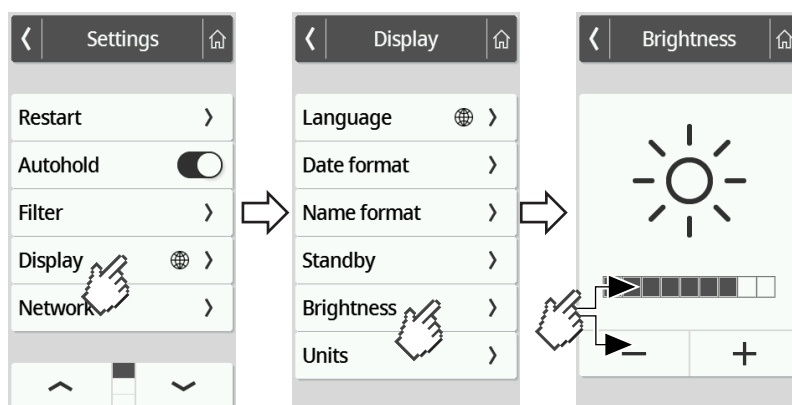
5. To exit the menu, press the  key.

## Setting display brightness

Device mode	Function available
Basic	–
Advanced	•
Expert	•
Service	•

Display brightness can be adjusted in stages (0 = off, 9 = max).

1. Press the  key.
  - ⇒ The **Settings** menu is displayed.
2. Press the  or  key until the **Display** menu item is displayed.
3. In the **Display** menu, select the **Brightness** item.
4. Adjust the brightness:
  - ▶ Press the plus/minus keys
  - ▶ Press the stages in the selection bar
  - ⇒ The setting is active.

5. To exit the menu, press the  key.

## Switching units

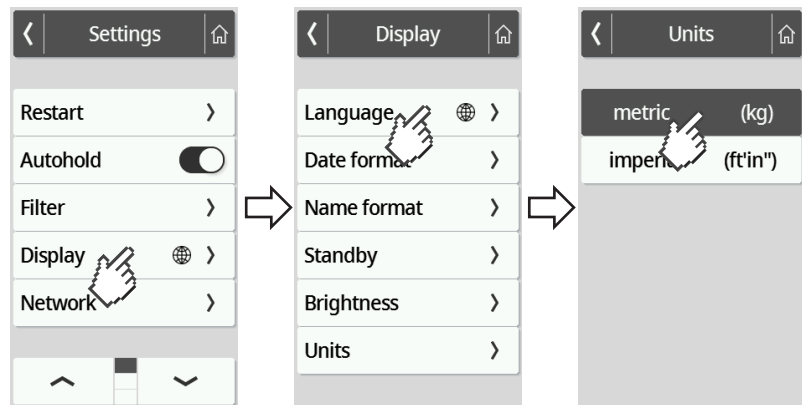
Device mode	Function available
Basic	–
Advanced	–
Expert	•
Service	•

### CAUTION! Patient hazard

To prevent misinterpretations, measured results for medical purposes must only be displayed and used in SI units (weight: kilograms/grams, height: meters/centimeters). Some devices have the option of displaying measured results in different units. This is purely an additional function.

- ▶ Only use measured results in SI units.
- ▶ The user takes sole responsibility for the use of measured results in non-SI units.

1. Press the  key.  
⇒ The **Settings** menu is displayed.
2. Press the  or  key until the **Display** menu item is displayed.
3. In the **Display** menu, select the **Units** item.
4. Press the desired system of units.

- ⇒ The setting is active.
  - ⇒ Measured results are displayed in the selected system of units.
5. To exit the menu, press the  key.

## 7.4 Setting up network functions

### **CAUTION!** Malfunction, implausible measured results

If network settings are not carried out correctly, measured results may be assigned incorrectly or lost.

- ▶ Have the steps described in this section carried out by your administrator or hospital technician. If you have any questions, contact seca Service.

### **NOTE**

As soon as the device is connected to a network, the **Autohold** function is activated automatically. The **Autohold** function cannot be deactivated if the device is connected to a network.

The following conditions must be met in order to be able to transmit measured values to the **seca analytics 125** software or to a third-party EMR System:

**seca analytics 125** software (direct connection):

- Device is connected to the server for the **seca analytics 125** software
- Device is connected to your network via a LAN or WiFi connection

### **NOTE**

In individual cases it may make sense not to connect the device directly to the **seca analytics 125** software, but rather via the **seca connect 103** software. This will be agreed during project preparation.

Third-party EMR System (via **seca connect 103**):

- Device is connected to the server for the **seca connect 103** software
- An interface to the EMR System has been set up in the **seca connect 103** software – in agreement with the third-party supplier
- Device is connected to your network via a LAN or WiFi connection
- A barcode scanner is connected to the device

Once the connection has been made, measurement consists of the following steps:

- Record ID(s) using the barcode scanner; alternatively, if connected directly to **seca analytics 125**: Submit IDs to the device
- Record measured values on the device
- Use the **seca connect 103** software to transmit measured results to the EMR System

### **NOTE**

Individual settings for the measurement can be made in the **seca connect 103** software or the **seca analytics 125** software. These settings are agreed during project preparation and set up by seca Service.

## Entering server address

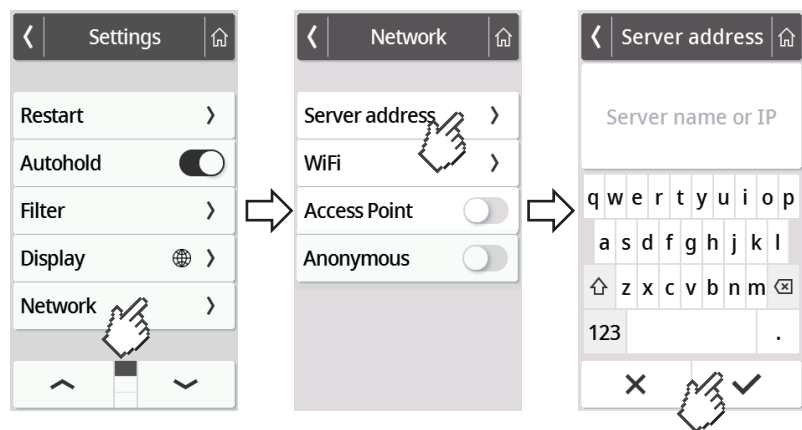
Device mode	Function available
Basic	–
Advanced	–
Expert	•
Service	•

In order to be able to use network functions, the device must be connected to one of the following servers - depending on your individual application:

- **seca connect 103**, local installation: Local server on which the **seca connect 103** software is installed.
- **seca connect 103**, cloud installation: Cloud server (you will have received access data during project implementation)
- **seca analytics 125**, cloud installation: Cloud server (you will have received access data during project implementation)

Which of the options mentioned applies to your application will have been agreed during project preparation.

1. Press the  key.  
⇒ The **Settings** menu is displayed.
2. Press the  or  key until the **Network** menu item is displayed.
3. Press the **Network** item.
4. Press the **Server address** item.
5. Enter the IP address of the server or the server name (DNS):
  - a) Enter the value
  - b) Confirm your entry by pressing the  key

6. You have the following options for continuing:
  - ▶ LAN: Connect the device to the network using a LAN cable
  - ▶ Establish a WiFi connection → [Connecting the device to a WiFi network, page 126](#)


## Activating/deactivating the WiFi function

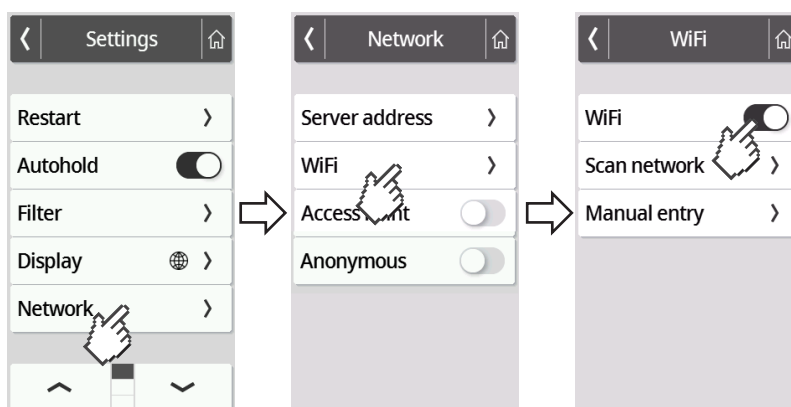Device mode	Function available
Basic	–
Advanced	–
Expert	•
Service	•

To activate/deactivate the WiFi function for the device, proceed as follows:

1. Press the  key.  
⇒ The **Settings** menu is displayed.
2. Press the  or  key until the **Network** menu item is displayed.
3. Press the **Network** item.  
⇒ The current setting is displayed:


4. Press the desired setting for the **WiFi** item:
  -  Function activated
  -  Function deactivated
5. To exit the menu, press the  key.


## Connecting the device to a WiFi network

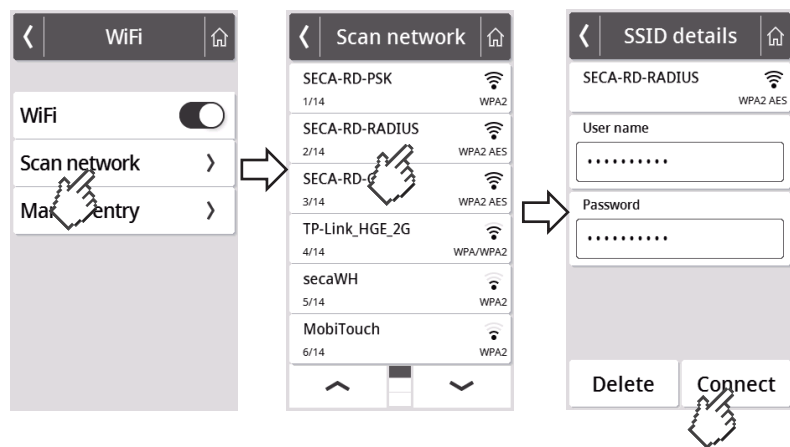Device mode	Function available
Basic	–
Advanced	–
Expert	•
Service	•

1. Ensure that there is no LAN connection - disconnect the LAN cable from the device if there is one.
2. Ensure that the WiFi function of the device is activated → [Activating/deactivating the WiFi function, page 125](#).
3. Press the  key.
  - ⇒ The **Settings** menu is displayed.
4. Press the  or  key until the **WiFi** menu item is displayed.
5. Press the **WiFi** menu item.
  - ⇒ You have the following options for continuing:
    - ▶ Search for network automatically (recommended and described below)
    - ▶ Integrate device in a WiFi network manually
6. Press the **Scan network** menu item.
  - ⇒ The device searches for available WiFi networks. This may take a moment.

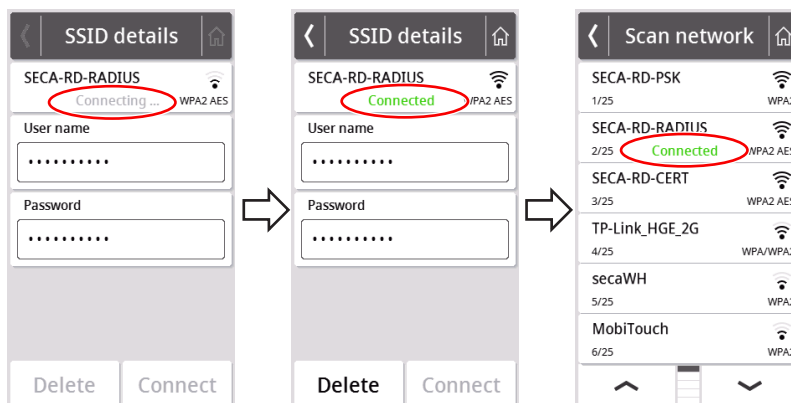7. Press the network you wish to use (here: "SECA-RD-RADIUS" with encryption standard "WPA2 AES").
8. Enter the user name and password for the WiFi network by pressing the relevant text field.
9. Press the **Connect** key.


- ⇒ The device connects (**Connecting**) to the router of the WiFi network.
- ⇒ As soon as the device is connected to the WiFi network, the message **Connected** is permanently on.

### Connecting the device to a WiFi network (WPS)

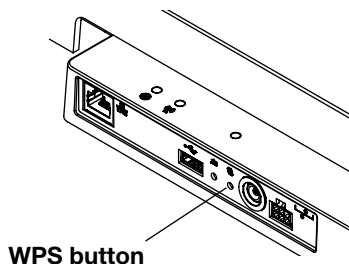Connect your device to the WiFi network via WPS if no barcode scanner is connected to the device and you have access to the router.

1. Ensure that the WiFi function of the device is activated → [Activating/deactivating the WiFi function, page 125](#).
2. Press the WPS button on the router and on the connection panel of the weighing platform.
  - ⇒ The device connects to the router of the WiFi network.
  - ⇒ As soon as the device is connected to the WiFi network, the  symbol is permanently on.

#### NOTICE!

##### Malfunction, incomplete data transmission

Further settings must be made to enable measurement data to be transmitted to an EMR System via the **seca connect 103** software.

- Observe the **seca 103/452** system instructions for use.

## Permitting anonymous measurements

Device mode	Function available
Basic	–
Advanced	–
Expert	–
Service	•

If the device is connected to a suitable EMR System via the **seca connect 103** software, you can permit anonymous measurements. If you activate this function, the device does not request user ID or patient data (date of birth, patient ID).


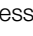
### WARNING!

#### Incorrect assignment of measured results, data loss

If measured results are incorrectly assigned or lost, this will lead to misinterpretations and consequently to misdiagnoses.

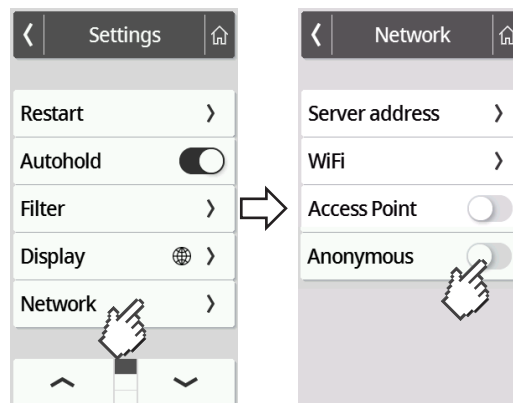- ▶ Ensure that your work environment supports anonymous measurements so that clear assignment of measured results is always assured.
- ▶ Use this function only in consultation with seca Service.

1. Press the  key.  
⇒ The **Settings** menu is displayed.
2. Press the  or  key until the **Network** menu item is displayed.
3. Press the **Network** item.  
⇒ The current setting is displayed.

4. Press the desired setting for the **Anonymous** item:
  -  Function activated
  -  Function deactivated
5. To exit the menu, press the  key.


## Automatically clearing measured values (Autoclear)

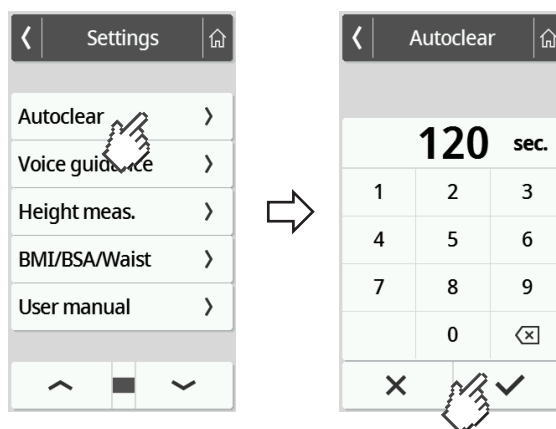Device mode	Function available
Basic	–
Advanced	–
Expert	•
Service	•

Out-of-date measured results and patient data lead to incorrect calculation of BMI or BSA or to implausible bioimpedance analyses. The period of time after which the following parameters are cleared automatically can be set:

- Weight
- Height
- **BMI**
- **BSA**
- Patient ID

1. Press the  key.  
⇒ The **Settings** menu is displayed.
2. Press the  or  key until the **Autoclear** menu item is displayed.
3. Press the **Autoclear** item.
4. Specify the time after which the device is to discard measured results and patient data:
  - a) Enter value (minimum: 1 sec./maximum: 3600 sec./1 h)
  - b) Confirm your entry by pressing the  key

5. To exit the menu, press the  key.

## 7.5 Factory settings

### Overview of factory settings

Function	Factory setting
Hold	Off
Tare	0 kg
Pre-tare	0 kg
Height	0 cm
Autohold	Off
Device mode	Expert
Autoclear <sup>a</sup>	300 sec.
Filter	Low
Display: Language	English
Display: Date format	YYYY/MM/DD
Display: Name format	First name Sur-name
Display: Standby	Off
Display: Brightness	Stage 7 of 9
BMI/BSA/Waist	BMI
Units	Metric (kg, cm)
Server address <sup>a</sup>	None
WiFi <sup>a</sup>	On
Access Point <sup>a</sup>	Off
Anonymous	Off

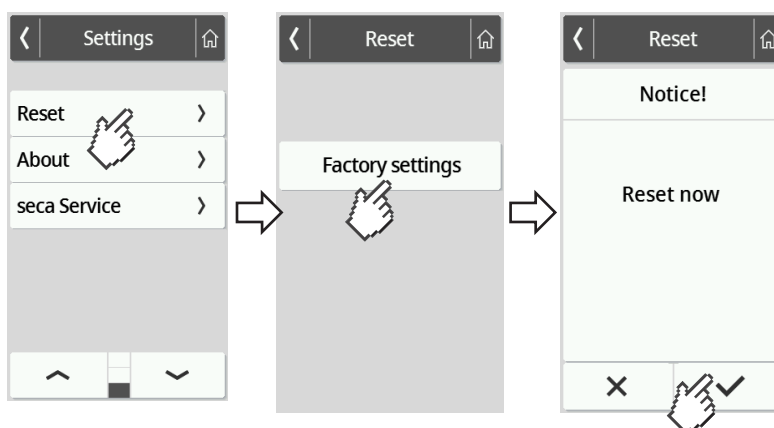
<sup>a</sup> Individual setting is **not** reset to factory settings.

### Restoring factory settings

Device mode	Function available
Basic	–
Advanced	–
Expert	•
Service	•

1. Press the  key.  
⇒ The **Settings** menu is displayed.

2. Press the  or  key until the **Reset** menu item is displayed.
3. Press the **Reset** item.

4. Press the **Factory settings** key.
5. Press the  key.
  - ⇒ The device will be reset to factory settings.
  - ⇒ The main screen is displayed again.

#### NOTE

The following network settings are **not** reset:

- **Autoclear**
- **Server address**
- **Server port**
- **WiFi**
- **Access Point**

## 8 HYGIENE TREATMENT

### **WARNING!** **Electric shock**

Use of fluids on the device may cause an electric shock.

- ▶ Disconnect the power supply connector before each hygiene treatment.
- ▶ Ensure that no fluids penetrate the device.

### **WARNING!** **Risk of infection**

- ▶ Subject the device to a hygiene treatment at regular intervals as described in this section.

### **NOTICE!** **Damage to device**

Unsuitable cleaning agents and disinfectants may damage the sensitive surfaces of the device and lead to transparent components becoming clouded.

- ▶ Do not use aggressive or abrasive cleaning agents.
- ▶ Do not use organic solvents (e.g. white spirit or petroleum spirit).

## 8.1 Cleaning

- ▶ If required, moisten a soft cloth with a mild soap solution and wipe the device over with it.

## 8.2 Disinfecting

1. Disinfect the device at regular intervals with a disinfectant suitable for sensitive surfaces and acrylic glass (e.g. 70 % ethanol).
2. Follow the instructions for use for the disinfectant.
3. Disinfect the device:
  - ▶ Moisten a soft cloth with disinfectant and wipe down the device with it.
  - ▶ Comply with the intervals, see table:

Interval	Component
Before each measurement	Weighing platform
After each measurement	Weighing platform
As required	<ul style="list-style-type: none"><li>• Multifunctional display</li><li>• Display cable</li></ul>

## 8.3 Sterilizing

The device must not be sterilized.

# 9 FUNCTION CHECK

- ▶ Perform a function check before each use.

A complete function check includes:

- A visual inspection for mechanical damage
- A test of the alignment of the device
- A visual and function check of the display elements
- A function check of all the controls shown in the section entitled "Overview"
- A function check of the optional accessories

If you find faults or deviations during the function check, first try to remedy the fault with the aid of the "Troubleshooting" section in this document.

 **CAUTION!**  
**Personal injury**

If you find faults or deviations during the function check which you are unable to remedy with the aid of the "Troubleshooting" section in this document, you must not use the device.

- ▶ Have the device repaired by seca Service or an authorized service partner.
- ▶ Follow the section entitled "Servicing" in this document.

## 10 TROUBLESHOOTING

If faults occur when operating the device, first attempt to remedy them yourself using the following tables. If the fault persists, contact seca Service.

With some faults, an error code appears on the multifunctional display. Please let seca Service have the error code when you contact them.

Information about display messages and the structure of error codes can be found here:

- → [Traffic light system for display messages, page 138](#)
- → [Error codes, page 139](#)

### 10.1 General faults

Fault	Cause	Remedy
No menu access possible	<b>Basic</b> device mode active	Clarify with the administrator/hospital technician whether the device can be operated in a different device mode <ul style="list-style-type: none"> <li>• → <a href="#">Changing device mode, page 112</a></li> <li>• → <a href="#">Functions/device mode, page 144</a></li> </ul>
Required function not available	Device mode in which the function is not provided is active	Clarify with the administrator/hospital technician whether the device can be operated in a different device mode <ul style="list-style-type: none"> <li>• → <a href="#">Changing device mode, page 112</a></li> <li>• → <a href="#">Functions/device mode, page 144</a></li> </ul>
Multifunctional display does not react when keys are pressed	Device is in an undefined state following implausible input	<ul style="list-style-type: none"> <li>• → <a href="#">Restarting the device, page 137</a></li> <li>• If the error recurs, inform seca Service</li> </ul>
	With combinations of devices involving a handrail and/or measuring rod: Additional display connected	<ul style="list-style-type: none"> <li>• → <a href="#">Restarting the device, page 137</a></li> <li>• If the error recurs, inform seca Service</li> </ul>
Multifunctional display remains dark after pressing	No connection to weighing platform	<ul style="list-style-type: none"> <li>• Check whether the display cable is connected correctly</li> <li>• If the error recurs, inform seca Service</li> </ul>
	No plug-in power supply unit connected	<ul style="list-style-type: none"> <li>• Check whether the plug-in power supply unit is connected correctly</li> <li>• If the error recurs, inform seca Service</li> </ul>
	Device is in an undefined state following implausible input	<ul style="list-style-type: none"> <li>• → <a href="#">Restarting the device, page 137</a></li> <li>• If the error recurs, inform seca Service</li> </ul>
	Multifunctional display defective	Inform seca Service

## 10.2 Measuring weight

Fault/error code	Cause	Remedy
Displayed weight is implausible	Weighing electronics using outdated zero point	<ul style="list-style-type: none"> <li>Remove the weight from the weighing platform</li> <li>Press <b>Weight</b> display field</li> <li>→ <a href="#">Restarting the device, page 137</a></li> <li>Wait until main screen is displayed again</li> </ul>
	Weighing electronics defective	Inform seca Service
<b>Autohold</b> function cannot be deactivated	Device is connected to a network (intended behavior): <b>Autohold</b> function is activated automatically	If necessary, disconnect device from network
<b>001-272XX-XXX</b> to <b>008-272XX-XXX</b>	Load cell or weight calculation module defective	Inform seca Service
<b>010-272XX-XXX</b>	Scale has been switched on with too high a load	<ul style="list-style-type: none"> <li>→ <a href="#">Restarting the device, page 137</a></li> <li>If the error recurs, inform seca Service</li> </ul>
<b>013-272XX-XXX</b>	Scale was caused to oscillate and was unable to determine the zero point	<ul style="list-style-type: none"> <li>→ <a href="#">Restarting the device, page 137</a></li> <li>If the error recurs, inform seca Service</li> </ul>
<b>016-272XX-XXX</b>	Maximum capacity exceeded	Ask patient to step off the scale
<b>019-272XX-XXX</b>	Ambient temperature too high or too low	<ul style="list-style-type: none"> <li>Observe ambient conditions for operation, transport, and storage → <a href="#">General technical data, page 145</a></li> </ul>
<b>020-272XX-XXX</b> to <b>023-272XX-XXX</b>	One corner of the scale has been loaded excessively	<ul style="list-style-type: none"> <li>Distribute weight evenly</li> </ul>
		<ul style="list-style-type: none"> <li>→ <a href="#">Restarting the device, page 137</a></li> <li>If the error recurs, inform seca Service</li> </ul>
		<ul style="list-style-type: none"> <li>Inform seca Service</li> </ul>
<b>024-272XX-XXX</b>	No GAL value	Inform seca Service

## 10.3 Data transmission

Fault/error code	Cause	Remedy
The "Date of birth" dialog window appears after the ✓ key is pressed	Patient ID not scanned	<ul style="list-style-type: none"> <li>Press the <b>X</b> key</li> <li>Scan patient ID</li> </ul>
	ID scanning not set up	Check Workflow settings for the <b>seca connect 103</b> or <b>seca analytics 125</b> software

Fault/error code	Cause	Remedy
	Patient does not yet have a file in the EMR System	<ul style="list-style-type: none"> <li>Enter date of birth and press ✓ key again</li> <li>Create patient file in the EMR System and assign measurement</li> </ul>
	Device is connected to the <b>seca analytics 125</b> software	<ul style="list-style-type: none"> <li>Enter date of birth</li> <li>Press the ✓ key</li> </ul>
After the ✓ key is pressed, there is no request for patient or user data	ID(s) scanned at the start of the measurement	<ul style="list-style-type: none"> <li>Not a malfunction, measured results are assigned to the patient and saved in the EMR System</li> <li>Check assignment in EMR System</li> </ul>
	ID scanning not set up	Check Workflow settings for the <b>seca connect 103</b> or <b>seca analytics 125</b> software
	<b>Anonymous</b> function activated	<ul style="list-style-type: none"> <li>Not a malfunction, measured results are sent to the EMR System</li> <li>In the EMR System, ensure that measured results are correctly assigned</li> </ul>
 icon appears	An obligatory measurement has not been performed	<ul style="list-style-type: none"> <li>Press the ✕ key</li> <li>Perform the missing measurement</li> </ul>
	Scanned ID is invalid	Scan valid ID
 icon appears	Optional measured value (e.g. height) has not been determined	<ul style="list-style-type: none"> <li>Press ✕ key and determine optional measured value</li> <li>Press ✓ key and end measurement</li> <li>If desired: Check Workflow settings for the <b>seca connect 103</b> or <b>seca analytics 125</b> software</li> </ul>
 icon appears	Optional ID (e.g. user ID) has been scanned	If desired: Check Workflow settings for the <b>seca connect 103</b> or <b>seca analytics 125</b> software
✓ key appears grayed out	There are no data available to be confirmed	<ul style="list-style-type: none"> <li>Perform a measurement</li> <li>Scan IDs (user/patient)</li> </ul>
✕ key appears grayed out	There are no data available to be cleared	<ul style="list-style-type: none"> <li>Perform a measurement</li> <li>Scan IDs (user/patient)</li> </ul>
Autoclear function: An entry of "0 seconds" is not accepted; the factory setting (300 seconds) is suggested instead	Implausible input; switching off the function is not intended in the factory	<ul style="list-style-type: none"> <li>Accept factory setting</li> <li>Manually enter value between 1 and 3600 seconds</li> </ul>

Fault/error code	Cause	Remedy
Workflow LED does not light up	Measurement not yet started	Start measurement (→ <a href="#">Starting the measurement procedure, page 102</a> )
	No network connection	Set up network connection → <a href="#">Setting up network functions, page 123</a>
	WiFi function deactivated	Activate the WiFi function → <a href="#">Activating/deactivating the WiFi function, page 125</a>
	Workflow LED defective	Inform seca Service
Workflow LED illuminated red	ID not found in EMR System or in the seca software	Create the ID in the EMR System or in the seca software
	Device has not saved measured results to the clipboard	Repeat measurement
	Measured results have not been sent to an EMR System or to seca software	<ul style="list-style-type: none"> <li>Repeat measurement</li> <li>Check WiFi connection</li> </ul>
<b>001-288XX-XXX</b>	No connection to the server	<ul style="list-style-type: none"> <li>Check LAN cable</li> <li>Check network settings → <a href="#">Setting up network functions, page 123</a></li> </ul>
<b>002-288XX-XXX</b>	No data transmission possible	<ul style="list-style-type: none"> <li>Check network settings → <a href="#">Setting up network functions, page 123</a></li> <li>Check Workflow settings for the <b>seca connect 103</b> or <b>seca analytics 125</b> software</li> </ul>
<b>004-288XX-XXX</b>	Barcode scanned at the wrong time	Repeat measurement and maintain correct operating sequence
		<ul style="list-style-type: none"> <li>→ <a href="#">Restarting the device, page 137</a></li> <li>If the error recurs, inform seca Service</li> </ul>
<b>005-288XX-XXX</b>	Barcode invalid or damaged	Use a valid, undamaged barcode
	Patient ID/user ID not found	<p>In the database (<b>seca analytics 125</b> or EMR System):</p> <ul style="list-style-type: none"> <li>Create patient file</li> <li>Create user account</li> </ul>
<b>006-288XX-XXX</b>	Unable to transmit measurement to EMR System	Check Workflow settings for the <b>seca connect 103</b> or <b>seca analytics 125</b> software
<b>010-288XX-XXX</b>	Update failed	<ul style="list-style-type: none"> <li>Restart update</li> <li>If the error recurs, inform seca Service</li> </ul>
<b>019-288XX-XXX</b>	Overcurrent at USB interface of weighing platform	<ul style="list-style-type: none"> <li>Check connected USB device</li> <li>If the error recurs, do not use the USB device</li> </ul>

Fault/error code	Cause	Remedy
020-288XX-XXX 022-288XX-XXX	No connection to <b>seca connect 103</b> software	<ul style="list-style-type: none"> <li>Repeat measurement</li> <li>Check Workflow settings for the <b>seca connect 103</b> or <b>seca analytics 125</b> software</li> </ul>
		<ul style="list-style-type: none"> <li>→ <a href="#">Restarting the device, page 137</a></li> <li>If the error recurs, inform seca Service</li> </ul>
050-280XX-XXX	Overcurrent at USB interface of multi-functional display	<ul style="list-style-type: none"> <li>Check connected USB device</li> <li>If the error recurs, do not use the USB device</li> </ul>
052-280XX-XXX	Internal device communication error	<ul style="list-style-type: none"> <li>→ <a href="#">Restarting the device, page 137</a></li> <li>If the error recurs, inform seca Service</li> </ul>
053-280XX-XXX	Communication error between weighing platform and multifunctional display	<ul style="list-style-type: none"> <li>→ <a href="#">Restarting the device, page 137</a></li> <li>If the error recurs, inform seca Service</li> </ul>

## 10.4 Restarting the device

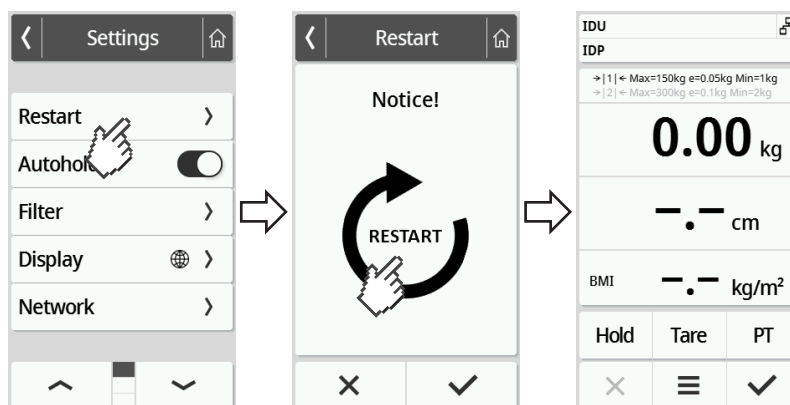
In individual cases (e.g. following implausible input) it may be necessary to restart the device. To do so, use the **Restart** function in the menu or interrupt the power supply and then restore it.

### NOTE

All the individual settings in the device are retained in the event of a restart. If you want to reset the device to factory settings, proceed as described in the relevant section: → [Factory settings, page 130](#)

### Using the “Restart” menu function

- Make sure that there is no load on the weighing platform.
- Press the  key.  
⇒ The **Settings** menu is displayed.
- Press the  or  key until the **Restart** menu item is displayed.


- Press the **Restart** item.

5. Press the  symbol.  
⇒ The device restarts.
6. Wait until the main screen is displayed again.  
⇒ The device is ready for operation.

### Interrupting and restoring the power supply (power supply operation)

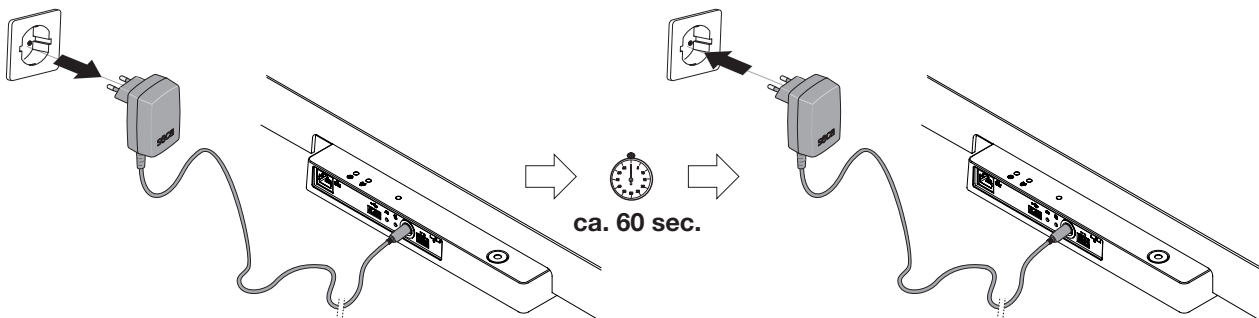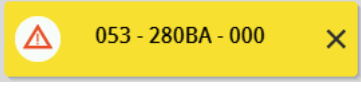
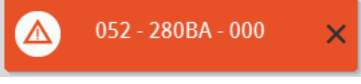
If restarting using the display is unsuccessful, you can briefly interrupt the power supply to the device:

1. Make sure that there is no load on the weighing platform.
2. Disconnect the power supply unit from the power supply socket.
3. Wait about a minute.


4. Plug the power supply unit back into the socket.  
⇒ The device and the multifunctional display switch on automatically.  
⇒ The device is ready for operation.

## 10.5 Traffic light system for display messages

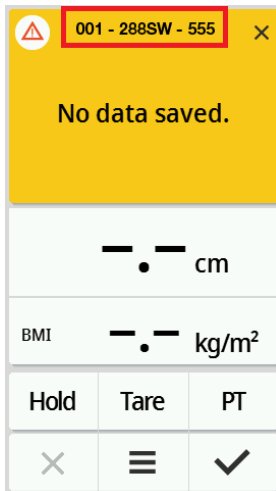Symbol	Description
	Green: Action successful, e.g. data sent to an EMR System or to the <b>seca analytics 125</b> software
	Yellow: Incorrect operation or malfunction, can be remedied by the user with the aid of the troubleshooting tables in these instructions for use (→ <a href="#">Troubleshooting, page 133</a> ).
	Red: Device error that cannot be remedied by the user, inform seca Service.

## 10.6 Error codes

### 001 - 288SW - 555

Model number; here: seca mBCA 555 scale

Firmware version of the assembly concerned; here: Index "W"

Hardware version of the assembly concerned; here: Index "S"

Assembly concerned; here: 288 = Interface module

Error number; here: 001 = no connection to the server

# 11 SERVICING

## 11.1 Verified scales

### CAUTION!

Faulty measurements as a result of verification being performed poorly or not at all

- ▶ Have verification performed only by authorized persons.
- ▶ Always have verification performed if one or more quality seals are damaged.

seca recommends having your device serviced prior to verification.

### CAUTION!

Faulty measurements as a result of poor servicing

- ▶ Have servicing and repairs carried out exclusively by seca Service or an authorized service partner.
- ▶ You can find a service partner in your vicinity at [www.seca.com](http://www.seca.com) or by emailing [service@seca.com](mailto:service@seca.com).

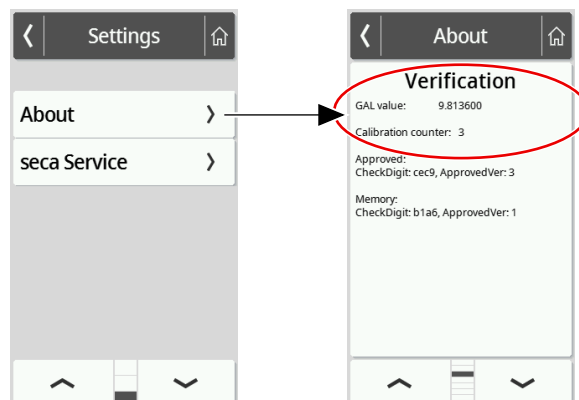
Have authorized persons perform verification in line with national legal regulations.

Verification is necessary whenever one or more quality seals are damaged or the contents of the verification counter no longer match the number on the applicable verification counter seal. If quality seals are damaged, contact seca Service directly.

Verifications may only be performed by authorized agencies. To guarantee this, the scale is equipped with a verification counter which records each change in verification-related data. The GAL value used by the device can also be read off.


If you want to check whether the scale has been properly verified, proceed as follows:

1. Press the key.  
⇒ The **Settings** menu is displayed.
2. Press **About**.
3. Press the or key until the **Verification** menu item is displayed.

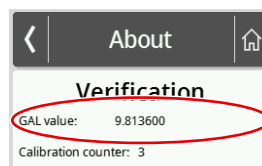
4. Read off the GAL value (figure shows example values).

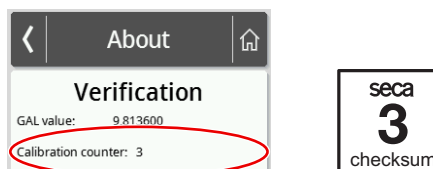5. Read off the verification counter.

⇒ The value must match the number indicated on the verification counter seal (figure shows example values).

Both numbers have to match for the verification to be valid. If the verification sticker and the verification counter do not match, the scale must be verified. Please contact your service partner or seca Service. Once the scale has been verified, a new, updated verification counter sticker is used to identify the verification counter reading. The person authorized to perform verification secures this verification with an additional seal. The verification counter sticker can be ordered from seca Service.

## 11.2 Non-verified scales

The product needs to be set up carefully and serviced regularly. Depending on how frequently the product is used, seca recommends servicing at intervals of 3 to 5 years.

### CAUTION!

#### Faulty measurements as a result of poor servicing

- ▶ Have servicing and repairs carried out exclusively by seca Service or an authorized service partner.
- ▶ You can find a service partner in your vicinity at [www.seca.com](http://www.seca.com) or by emailing [service@seca.com](mailto:service@seca.com).

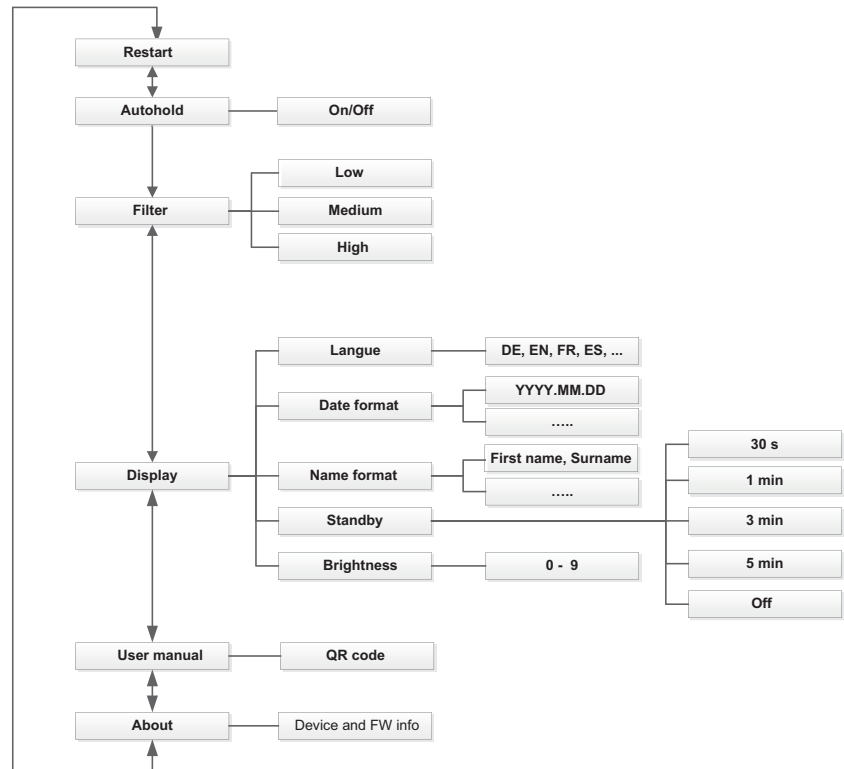
## 12 TECHNICAL DATA

### 12.1 Menu structures


#### "Basic" device mode

In **Basic** device mode, only the **Restart** menu item is available.

#### "Advanced" device mode

"Expert"/"Service" device modes

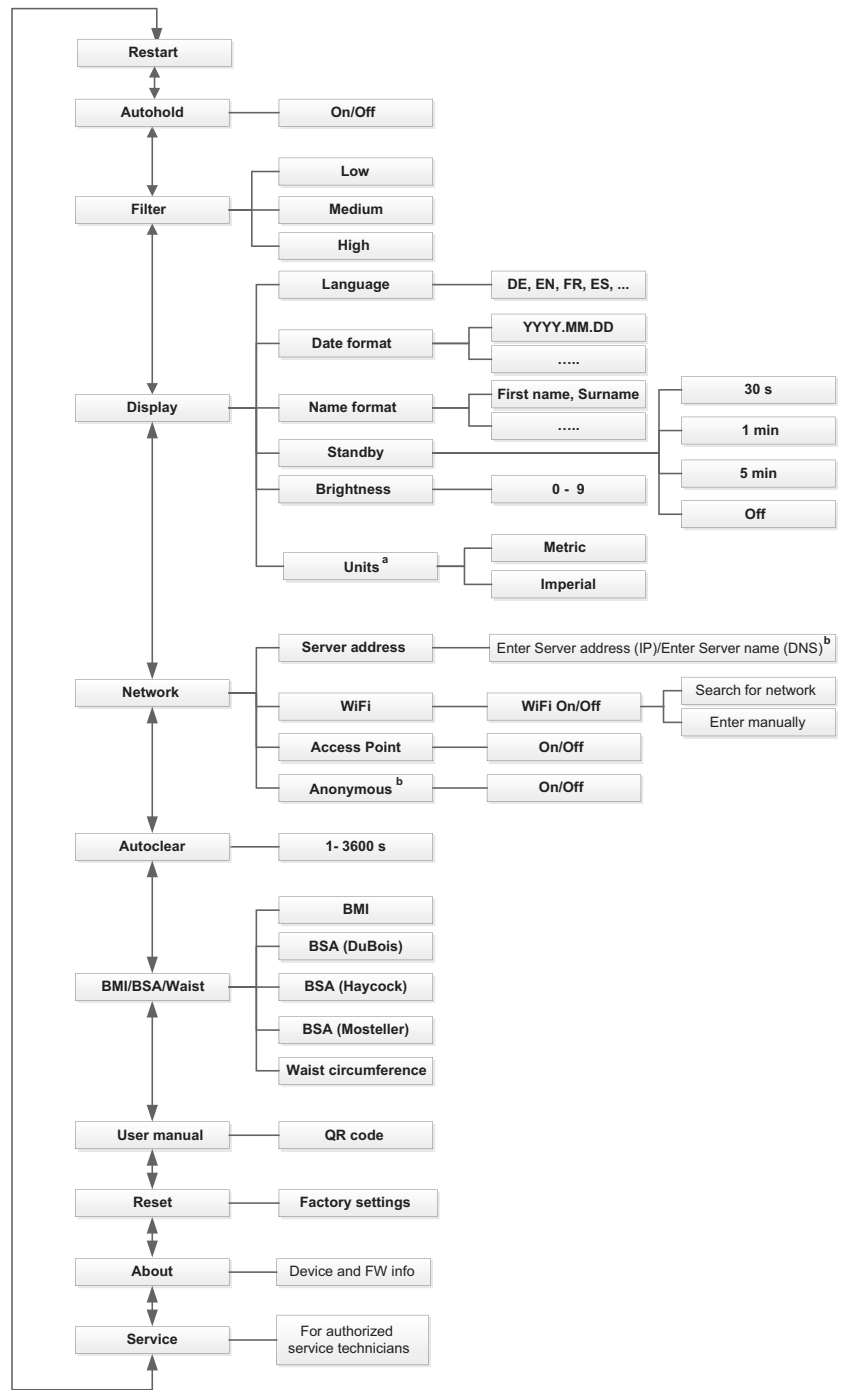a Only for non-verified scales  
 b Only use in consultation with seca Service

## 12.2 Functions/device mode

Function	Device mode			
	Basic	Advanced	Expert	Service
<b>Measure</b>				
Calculate <b>BMI/BSA</b> automatically	•	•	•	•
Measure weight	•	•	•	•
Record IDs (user/patient) <sup>a</sup>	•	•	•	•
Enter height manually	•	•	•	•
Permanently display measured results ( <b>Hold</b> )	–	•	•	•
Submit measured results <sup>a</sup>	•	•	•	•
Enter waist circumference	–	•	•	•
Tare additional weight ( <b>Tare</b> )	–	•	•	•
Permanently save the additional weight ( <b>Pre-tare</b> )	–	•	•	•
<b>Configure</b>				
Permit anonymous measurement <sup>a,b</sup>	–	–	–	•
Use the PDF version of the instructions for use (QR code)	–	•	•	•
<b>Autoclear</b> function: Define time period	–	–	•	•
Activate <b>Autohold</b> function	–	•	•	•
Use <b>Restart</b> function	•	•	•	•
Set filter (sensitivity of the scale to patient movements)	–	•	•	•
Set date format	–	•	•	•
Set name format	–	•	•	•
Set display brightness	–	•	•	•
Set display language	–	•	•	•
Read off verification counter reading (verified scales)	–	•	•	•
Switch over units (non-verified scales)	–	–	•	•
Read off GAL value	–	•	•	•
Connect device to WiFi network (WPS)	–	–	•	•
Connect device to WiFi network (directly)	–	–	•	•
Connect devices to WiFi network ( <b>seca connect 103</b> )	–	–	•	•

Function	Device mode			
	Basic	Advanced	Expert	Service
With network connection:				
Enter server name (DNS)	–	–	•	•
Enter IP address of server	–	–	•	•
Call up menu	–	•	•	•
Service functions <sup>c</sup>	–	–	–	•
Set standby time	–	•	•	•
Switch between BMI/BSA calculation and waist circumference input	–	–	•	•
Restore factory settings	–	–	•	•
Activate/deactivate WiFi module	–	–	•	•

<sup>a</sup> Devices with connection to an EMR System or the **seca analytics 125** software (directly or via **seca connect 103** software)

<sup>b</sup> Only use following discussion with seca Service

<sup>c</sup> For authorized service technicians only

## 12.3 General technical data

General technical data	
Ambient conditions, operation: <ul style="list-style-type: none"> <li>• Temperature</li> <li>• Air pressure</li> <li>• Humidity</li> </ul>	+10 °C to +40 °C (50 °F to 104 °F) 700 hPa – 1060 hPa 20 % – 80 %, no condensation
Ambient conditions, storage: <ul style="list-style-type: none"> <li>• Temperature</li> <li>• Air pressure</li> <li>• Humidity</li> <li>• Warm-up time from lowest storage temperature to operational temperature               <ul style="list-style-type: none"> <li>– At ambient temperature 20 °C</li> <li>– At ambient temperature 20 °C with condensation</li> </ul> </li> <li>• Cooling time from highest storage temperature to operational temperature (at ambient temperature 20 °C)</li> </ul>	-10 °C to +65 °C (14 °F to 149 °F) 700 hPa – 1060 hPa 0 % – 95 %, no condensation  8 h 24 h 8 h

General technical data	
Ambient conditions, transport	
<ul style="list-style-type: none"> <li>• Temperature</li> <li>• Air pressure</li> <li>• Humidity</li> </ul>	<p>-10 °C to +65 °C (14 °F to 149 °F)</p> <p>700 hPa – 1060 hPa</p> <p>0 % – 95 %, no condensation</p>
Power supply: Plug-in power supply unit	
<ul style="list-style-type: none"> <li>• Supply voltage</li> <li>• Maximum current consumption</li> </ul>	<p>12 V</p> <p>max. 1.5 A</p>
Power supply voltage	100 V – 240 V
Power supply frequency	50 Hz – 60 Hz
Power consumption	max. 18 W
IEC 60601-1: Medical electrical device, Type BF	
Type of protection to IEC 60529	IP 21
Duty cycle	Continuous duty
Medical device in accordance with Regulation (EU) 2017/745:	Class I with measuring function
Application parts to IEC 60601-1:	Multifunctional display, glass plate
Interfaces:	
<ul style="list-style-type: none"> <li>• USB</li> <li>• WiFi</li> <li>• LAN</li> <li>• Internal bus system/multifunctional display</li> </ul>	<p>USB 2.0, max. 500 mA</p> <p>IEEE 802.11b/g/n/e/i</p> <p>IEEE 802.3u</p> <p>seca device bus (SDB)</p>

## 12.4 Dimensions and weights

Dimensions and weights	
Scale	
Dimensions:	
<ul style="list-style-type: none"> <li>• Depth</li> <li>• Width</li> <li>• Height</li> </ul>	<p>574 mm</p> <p>600 mm</p> <p>70 mm</p>
Net weight	Approx. 14 kg

## 12.5 Weight measurement

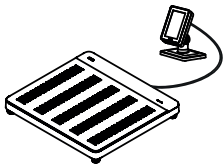
Verified model	
Verification in line with Directive 2014/31/EU	Class III
Maximum capacity	
• Weighing range 1	150 kg
• Weighing range 2	300 kg
Minimum capacity	
• Weighing range 1	1 kg
• Weighing range 2	2 kg
Graduation	
• Weighing range 1	50 g
• Weighing range 2	100 g
Tare range	300 kg (subtractive)
Accuracy on initial verification	
• Weighing range 1: 0 to 25 kg	± 25 g
• Weighing range 1: 25 to 100 kg	± 50 g
• Weighing range 1: 100 to 150 kg	± 75 g
• Weighing range 2: 0 to 50 kg	± 50 g
• Weighing range 2: 50 to 200 kg	± 100 g
• Weighing range 2: 200 to 300 kg	± 150 g

Non-verified model	
Maximum capacity	360 kg
Minimum capacity	1 kg
Graduation	50 g
Tare range	360 kg (subtractive)
Accuracy	
• 0 kg to 50 kg	± 50 g
• 50 kg to 360 kg	± 50 g / ± 0.1 %

## 13 OPTIONAL ACCESSORIES AND SPARE PARTS

Accessory/spare part	Article number
Switch-mode power supply: 100-240 V~ / 50-60 Hz, 12 V= / 1.5 A / 18 W	68 32 10 272
Barcode scanner	See recommendation at <a href="http://www.seca.com">www.seca.com</a>
Configuration software	→ <a href="#">Compatible seca products, page 148</a>
Analysis software	

## 14 COMPATIBLE SECA PRODUCTS

Scale	Handrail	Measuring rod	Configuration software	Analysis software
<b>seca Medical, housing color: White</b>				
 <p><b>seca 651/650</b> 651 7021 099 650 1321 009</p>	-	-	 <p><b>seca connect 103</b> from Version 2.0</p>	 <p><b>seca analytics 125</b></p>

## 15 DISPOSING OF THE DEVICE

Do not dispose of the device in your household waste. The device must be properly disposed of as electronic scrap. Follow your respective national regulations. For more information, please contact seca Service at [service@seca.com](mailto:service@seca.com).

## 16 WARRANTY

There is a two-year warranty period from delivery for defects attributable to poor materials or workmanship. All movable parts, e.g. batteries, cables, power supply units, rechargeable batteries etc. are exempt. Defects which come under the warranty will be repaired for the customer free of charge against proof of purchase. Additional claims cannot be considered. Costs of transport to and from seca are the responsibility of the customer if the device is located somewhere other than the customer's headquarters. In the event of transport damage, claims under warranty can only be made if the complete original packaging was used for transport and the device was secured and fastened in it according to its originally packaged condition. You should therefore keep all packaging parts.

The warranty will be voided if the device is opened by persons not expressly authorized by seca to do so.

In the event of a warranty issue, please contact your local seca office or the dealer from whom you ordered the product.

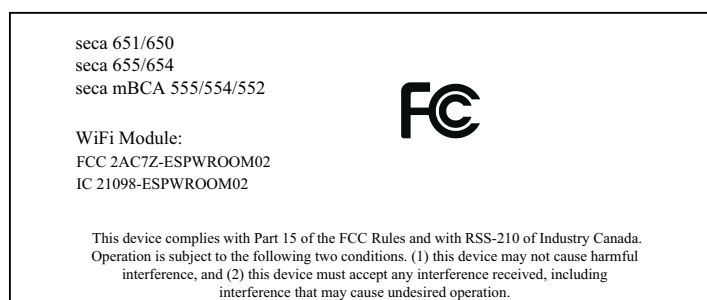
## 17 DECLARATIONS OF CONFORMITY

### 17.1 Europe

seca gmbh & co. kg hereby declares that the product complies with the terms of the applicable European directives and regulations. The unabridged declaration of conformity can be found at [www.seca.com](http://www.seca.com).

### 17.2 USA and Canada

#### NOTE

This device complies with Part 15 of the FCC Rules and with RSS-210 of Industry Canada. Operation is subject to the following two conditions:

- This device may not cause harmful interference.
- This device must accept any interference received, including interference that may cause undesired operation.

#### NOTE

Changes or modifications made to this equipment not expressly approved by seca may void the FCC authorization to operate this equipment.

#### NOTE

Radiofrequency radiation exposure information: This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with minimum distance of 1 m between the radiator and your body. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

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