
# seca 285/284

## Instructions for use

17-10-07-670-100b1\_2025-02S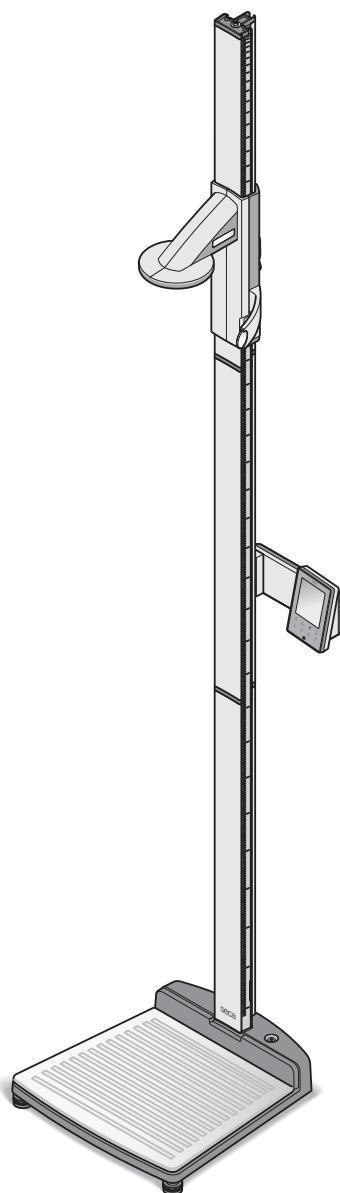

<b>Deutsch – Gebrauchsanweisung .....</b>	<b>3</b>
<b>English – Instructions for use .....</b>	<b>47</b>
<b>Français – Mode d’emploi.....</b>	<b>89</b>
<b>Italiano – Istruzioni per l’uso .....</b>	<b>132</b>
<b>Español – Instrucciones de uso .....</b>	<b>176</b>
<b>Português – Manual de instruções de utilização .....</b>	<b>220</b>
<b>Polski – Instrukcja obsługi.....</b>	<b>264</b>

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## TABLE OF CONTENTS

<b>1 About this document .....</b>	<b>49</b>	5.6 Calibrating height measurement .....	69
1.1 Representation in text .....	49	Calibrating automatically .....	69
1.2 Representation in diagrams .....	49	Calibrating manually .....	69
<b>2 Description of device .....</b>	<b>49</b>	5.7 Disinfecting device .....	70
2.1 Intended use.....	49		
2.2 Contraindications .....	49		
2.3 Clinical benefit .....	50		
2.4 Patient target group.....	50		
2.5 User qualification .....	50		
Administration/network operation .....	50		
Operation.....	50		
Assembly .....	50		
2.6 Functional description .....	50		
Measuring weight and height .....	50		
External interface module (optional accessory) .....	50		
seca connect 103 software (optional accessory) .....	50		
<b>3 Safety information .....</b>	<b>50</b>		
3.1 Safety information in these instructions for use.....	50		
3.2 Basic safety information .....	51		
Handling device .....	51		
Preventing electric shock .....	52		
Preventing injuries and infections.....	52		
Preventing damage to device.....	53		
Handling measuring results .....	53		
Handling packaging material.....	54		
Handling batteries and rechargeable batteries.....	54		
<b>4 Overview .....</b>	<b>55</b>		
4.1 Controls .....	55		
4.2 Symbols in the display .....	57		
4.3 Labels .....	57		
4.4 Menu structure for multifunctional display ..	60		
4.5 Navigating in the menu of the display .....	60		
4.6 Menu structure for head slide .....	61		
4.7 Navigating in the menu of the head slide ....	61		
<b>5 Starting up device.....</b>	<b>62</b>		
5.1 Scope of delivery.....	62		
5.2 Assembling device .....	63		
Introduction .....	63		
Fitting Frankfurt measure .....	63		
Fitting second column element .....	63		
Fitting multifunctional display.....	64		
Fitting third column element .....	65		
Fitting push-in scale .....	66		
5.3 Setting up device.....	66		
5.4 Transporting device.....	67		
5.5 Establishing power supply .....	68		
Inserting batteries .....	68		
Connecting power supply unit.....	68		
<b>6 Operating the scale .....</b>	<b>70</b>		
6.1 Weighing.....	70		
Switch on device .....	70		
Performing weighing .....	70		
Activating/deactivating key lock.....	70		
Weighing babies/toddlers (2 in 1).....	71		
Displaying measuring result permanently (hold).....	71		
Entering patient data (input) .....	71		
Entering patient's gender .....	72		
Determining Body Mass Index (BMI).....	72		
Determining Body Fat Ratio (BFR) .....	73		
Clearing stored values (clear) .....	73		
Switching weighing range automatically ..	73		
Switching off the scale .....	73		
6.2 Other functions (menu).....	74		
Deleting values automatically (ACLR) .....	74		
Switching between BMI and BFR (bodY) .....	74		
Setting display backlighting (LCd).....	74		
Entering height manually (HGht) .....	75		
Saving additional weight permanently (Pt) .....	75		
Activating Autohold (AHOLD) function .....	76		
Activating/deactivating beeps (bEEP) .....	76		
Setting filtering (FIL) .....	76		
Non-verified models: Switching weight unit (Unitl) .....	77		
Restoring factory settings (rESEt) .....	77		
<b>7 Operating the head slide.....</b>	<b>78</b>		
7.1 Measuring height .....	78		
Switching on head slide .....	78		
Starting height measurement .....	78		
Displaying measuring result permanently (Hold) .....	78		
Performing relative measurements (ZERO) .....	79		
Switching off head slide .....	79		
7.2 Other functions (menu) .....	79		
Setting display backlighting (LCd) .....	79		
Activating beeps (bEEP) .....	80		
Switching unit of height (Unit) .....	80		
Restoring factory settings (rESEt) .....	80		
<b>8 Hygiene treatment .....</b>	<b>81</b>		
8.1 Cleaning .....	81		
8.2 Disinfecting .....	81		
8.3 Sterilizing .....	81		
<b>9 Function check .....</b>	<b>82</b>		
<b>10 Troubleshooting.....</b>	<b>82</b>		
10.1 General faults .....	82		
10.2 Changing the batteries .....	83		

<b>11 Servicing .....</b>	<b>84</b>
11.1 Weight measurement, verified models .....	84
11.2 Weight measurement, non-verified models .	85
11.3 Height measuring devices .....	85
<b>12 Technical data.....</b>	<b>85</b>
12.1 General technical data.....	85
12.2 Weight measurement .....	86
12.3 Height measurement .....	87
<b>13 Optional accessories and spare parts.....</b>	<b>87</b>
<b>14 Disposal .....</b>	<b>87</b>
14.1 Disposing of the device .....	87
14.2 Disposing of batteries and rechargeable batteries.....	87
<b>15 Warranty .....</b>	<b>88</b>
<b>16 Declaration of conformity .....</b>	<b>88</b>

# 1 ABOUT THIS DOCUMENT

## 1.1 Representation in text

Symbol	Description
✓	Requirement for actions
►	Action
1. 2.	Actions with specified sequence
a) b)	Steps of an action with specified sequence
⇒	Result of an action
• •	First level of a list
— —	Second level of a list

## 1.2 Representation in diagrams

Symbol	Description
→	Indicates directions of movement of the device or of device components
└─┐	Navigation path in menu displays
✓	<ul style="list-style-type: none"> <li>Correct action</li> <li>Correct result of action</li> </ul>
✗	<ul style="list-style-type: none"> <li>Incorrect action</li> <li>Incorrect result of action</li> </ul>
✓	End of a procedure, e.g. the installation of a part

# 2 DESCRIPTION OF DEVICE

## 2.1 Intended use

The electronic measuring station supports physicians in decision-making regarding diagnosis or therapy, based on the patient's weight, height or changes of the measured parameters.

## 2.2 Contraindications

No contraindications are known.

## 2.3 Clinical benefit

The electronic measuring station supports physicians in decision-making regarding diagnosis or therapy, based on the patient's weight or height.

By that, the electronic measuring station only provides an indirect clinical benefit.

The physician must perform additional examinations for an accurate diagnosis.

## 2.4 Patient target group

The device is intended for persons of any age who can stand independently and who do not exceed the maximum weight capacity of the device. Babies and toddlers who cannot stand independently can be weighed using the **2 in 1** function whilst being held by an adult who can stand independently.

## 2.5 User qualification

<b>Administration/network operation</b>	The device may only be set up and incorporated in a network by experienced administrators or hospital technicians.
<b>Operation</b>	The device may only be operated by persons with formal training in the healthcare sector or in medicine.
<b>Assembly</b>	Devices shipped partly assembled may only be assembled by sufficiently qualified persons (such as specialist dealers, hospital technicians or seca Service technicians).

## 2.6 Functional description

<b>Measuring weight and height</b>	Weight is recorded by four load cells. Height is recorded by scanning a digital scale. Body Mass Index ( <b>BMI</b> ) or Body Fat Ratio ( <b>BFR</b> ) can be calculated automatically from the measuring results.
<b>External interface module (optional accessory)</b>	Data can be sent to the <b>seca connect 103</b> software via a WiFi or LAN connection with the aid of the <b>seca 452</b> external interface module (optional).
<b>seca connect 103 software (optional accessory)</b>	The <b>seca connect 103</b> software obtains measurement data from an internal or external interface module and sends them to compatible seca measuring devices or to third-party EMR systems.

# 3 SAFETY INFORMATION

## 3.1 Safety information in these instructions for use

### DANGER!

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries will occur.

### WARNING!

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries may result.

**CAUTION!**

Used to identify a hazardous situation. If you fail to take note of this information, minor to moderate injuries may result.

**NOTICE!**

Used to identify possible incorrect usage of the device. If you fail to take note of this information, you may damage the device, or the measuring results may be incorrect.

**NOTE**

Includes additional information about use of the device.

## 3.2 Basic safety information

---

**Handling device**

- ▶ Please take note of the information in these instructions for use.
- ▶ Keep the instructions for use in a safe place. The instructions for use are a component of the device and must be available at all times.
- ▶ In the interest of patient safety, you and your patients are obliged to report serious events that occur in connection with this product to the manufacturer and to the authority responsible in your country.

**DANGER!****Risk of explosion**

- ▶ Do not use the device in an environment in which one of the following gases has accumulated:
  - oxygen
  - flammable anesthetics
  - other flammable substances/air mixtures

**CAUTION!****Patient hazard, damage to device**

- ▶ Additional devices connected to medical electrical devices must provide evidence of compliance with the relevant IEC or ISO standards (e.g. IEC 60950 for data-processing devices). Furthermore, all configurations must comply with the requirements of standards for medical systems (see IEC 60601-1-1 or Section 16 of edition 3.1 of IEC 60601-1 respectively). Anyone connecting additional devices to medical electrical devices is considered a system configurator and therefore responsible for ensuring that the system complies with the requirements of standards for systems. This also applies to additional devices recommended by seca. Your attention is drawn to the fact that local laws take precedence over the above-mentioned requirements of standards. In the event of any queries, please contact your local specialist dealer or Technical Service.

**CAUTION!****Patient hazard, damage to device**

- ▶ Have servicing carried out regularly as described in the relevant section of this document.
- ▶ Technical modifications may not be made to the device. The device does not contain any user-serviceable parts. Only have servicing and repairs performed by an authorized seca service partner. You can find your local service partner at [www.seca.com](http://www.seca.com).
- ▶ Use only seca original accessories and spare parts, otherwise seca will not grant any warranty.

**CAUTION!**  
**Patient hazard, malfunction**

- ▶ Keep other medical electrical devices, e.g. high-frequency surgical devices, a minimum distance of approx. 1 meter away to prevent faulty measurements or wireless transmission interference.
- ▶ Keep HF devices such as cellphones at a minimum distance of approx. 1 meter to prevent faulty measurements or wireless transmission interference.
- ▶ The actual transmission output of HF devices may require minimum distances of more than 1 meter. For details, go to [www.seca.com](http://www.seca.com).

**Preventing electric shock**

**WARNING!**  
**Electric shock**

- ▶ Set up devices which can be operated with a power supply unit so that the power supply socket is within easy reach and the power supply can be disconnected quickly.
- ▶ Ensure that your local electricity supply matches the details on the power supply unit.
- ▶ Never touch the power supply unit with wet hands.
- ▶ Do not use extension cables and multiple outlets.
- ▶ Ensure that cables are not crushed or damaged by sharp edges.
- ▶ Ensure that cables do not come into contact with hot objects.
- ▶ Do not operate the device at an altitude of more than 3000 m above sea level.

**Preventing injuries and infections**

**WARNING!**  
**Injury from falling**

- ▶ Ensure that the device is steady and level.
- ▶ Route connector cables (if present) so that neither users nor the patient can trip over them.
- ▶ Assist people with limited mobility, e.g. when they are getting up from a wheelchair.
- ▶ Ensure that the patient does not step straight onto or off the edges of the patient standing area.
- ▶ Ensure that the patient steps onto and off the patient standing area slowly and safely.

**WARNING!**  
**Risk of slipping**

- ▶ Ensure that the patient standing area is dry before the patient steps onto it.
- ▶ Ensure that the patient has dry feet before stepping onto the patient standing area.
- ▶ Ensure that the patient steps onto and off the patient standing area slowly and safely.

**CAUTION!**  
**Injury, damage to device**

The patient standing area consists of a glass plate. Damage (e.g. scratches, cracks or chips) represent a risk of injury. Damage can also lead to the glass plate breaking.

- ▶ Do not put any sharp-edged objects on the glass plate.
- ▶ Before using each time, check the glass plate for scratches, cracks and chips. If you find damage of this kind, have the glass plate replaced with a new one.
- ▶ Do not use the device if the glass plate is damaged.

**WARNING!**  
**Risk of infection**

- ▶ Before and after every measurement, wash your hands to reduce the risk of cross-contamination and nosocomial infections.
- ▶ Subject the device to a hygiene treatment at regular intervals as described in the relevant section of these instructions for use.
- ▶ Ensure that the patient does not have any infectious diseases.
- ▶ Ensure that the patient does not have any open wounds or infectious skin alterations which may come into contact with the device.

**Preventing damage to device**

**NOTICE!**

**Damage to device**

- ▶ Ensure that fluids never get inside the device. These can destroy the electronics.
- ▶ For devices with power supply operation: Switch off the device before you disconnect the power supply unit from the power supply socket.
- ▶ For devices with power supply operation: If the device is not to be used for an extended period, disconnect the power supply unit from the power supply socket. Only then is the device de-energized.
- ▶ For devices with battery or rechargeable battery operation: If you are not using the device for an extended period of time, remove batteries or rechargeable batteries. Only then is the device de-energized.
- ▶ Do not drop the device.
- ▶ Do not subject the device to shocks or vibrations.
- ▶ Perform a function check before each use as described in the corresponding section in this document. Do not operate the device if it is not working properly or is damaged.
- ▶ Do not place the device in direct sunlight and ensure that it is not placed in the direct proximity of a heat source. The excessive temperatures could damage the electronics.
- ▶ Avoid rapid temperature fluctuations. If the device is transported so that a temperature difference of over 20 °C occurs, the device must be left to stand for at least 2 hours before it is switched on, otherwise condensation may form; this may damage the electronics.
- ▶ Use the device only in the intended ambient conditions outlined in the section entitled "Technical data".
- ▶ Store the device only in the intended storage conditions outlined in the section entitled "Technical data".
- ▶ Use only cleaning agents and disinfectants which match the details in the section entitled "Hygiene treatment".
- ▶ For scales: Ensure that maximum capacity is not exceeded.

**Handling measuring results**

**WARNING!**  
**Patient hazard**

To prevent misinterpretations, measuring results for medical purposes must only be displayed and used in SI units (kilograms/grams, meters/centimeters). Some devices have the option of displaying measuring results in different units. This is purely an additional function.

- ▶ Only use measuring results in SI units.
- ▶ The user takes sole responsibility for the use of measuring results in non-SI units.

**NOTICE!****Inconsistent measuring results**

- ▶ Before you save and continue using measured values determined using this device (e.g. in seca software or in an EMR system), ensure that the measured values are plausible.
- ▶ If measured values have been sent to seca software or to an EMR system, ensure before continuing to use them that the measured values are plausible and assigned to the correct patient.

**Handling packaging material****WARNING!****Risk of suffocation**

Packaging materials made of plastic film (bags) present a risk of suffocation.

- ▶ Store packaging material out of the reach of children.
- ▶ If the original packaging material is no longer available, only use plastic bags with safety holes to reduce the risk of suffocation. Use recyclable materials if possible.

**NOTE**

Store the original packaging material for future use (e.g. returning for servicing).

**Handling batteries and rechargeable batteries****WARNING!****Injury**

Batteries and rechargeable batteries contain harmful substances which may explode and so escape if not handled properly.


- ▶ Do not try to recharge batteries.
- ▶ Do not expose batteries/rechargeable batteries to heat.
- ▶ Do not burn batteries/rechargeable batteries.
- ▶ If harmful substances are leaking out, avoid contact with the skin, eyes, and mucous membranes. Rinse affected areas of the body with plenty of clean water and seek medical help at once.

**NOTICE!****Damage to device and malfunction due to incorrect handling**

- ▶ Use only the type of battery/rechargeable battery quoted in this document.
- ▶ Always replace all batteries/rechargeable batteries at the same time.
- ▶ Do not short-circuit batteries/rechargeable batteries.
- ▶ If you do not use the device for an extended period of time, remove the batteries/rechargeable batteries. This prevents acid from leaking into the device.
- ▶ If harmful substances have penetrated the device, do not continue using it. Have the device checked by an authorized seca service partner and repaired if necessary.

## 4 OVERVIEW


### 4.1 Controls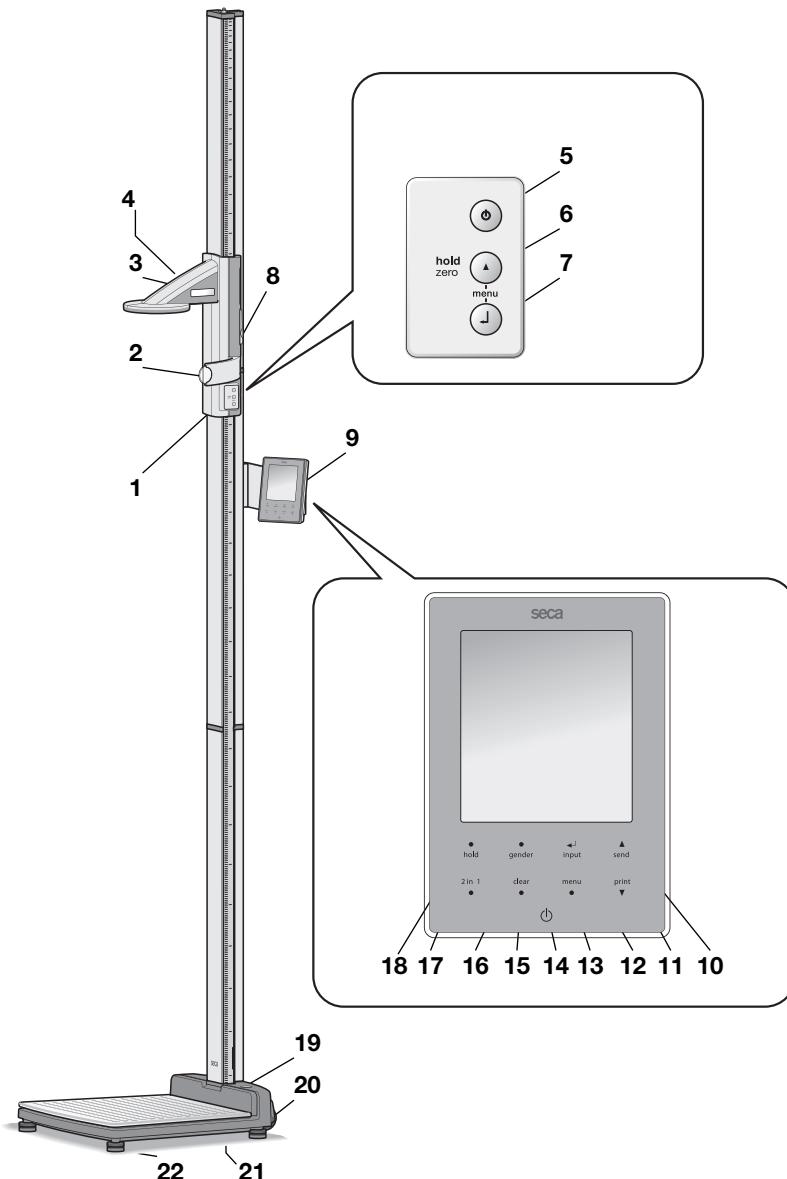

Item	Control	Function
1	Head slide	For determining height
2	Frankfurt measure	Pull-out measure for aligning the head in accordance with the Frankfurt horizontal plane.
3	Display	Display element for measuring results and for configuration
4	Battery compartment	Takes battery holder with 4 AA-type batteries, 1.5 V
5	⊕	Switch the head slide on or off

Item	Control	Function
6		<ul style="list-style-type: none"> <li>During measuring: <ul style="list-style-type: none"> <li>Press briefly: Activate <b>Hold</b> function</li> <li>Press and hold: Set zero point</li> </ul> </li> <li>In the menu: <ul style="list-style-type: none"> <li>Select submenu, select menu item</li> <li>Set value (press briefly: Value is changed by 1 increment, press and hold: Value is changed until key is released)</li> </ul> </li> </ul>
7		<ul style="list-style-type: none"> <li>During measuring: Send measuring result to multifunctional display</li> <li>In the menu: <ul style="list-style-type: none"> <li>Confirm selected menu item</li> <li>Save set value</li> </ul> </li> </ul>
8	Brake button	<ul style="list-style-type: none"> <li>Holds the head slide in position</li> <li>Press: Move head slide</li> </ul>
9	Multifunctional display	Central control and display element
10	send	<ul style="list-style-type: none"> <li>Send measuring results to an EMR system via <b>seca connect 103</b> software (condition: <b>seca 452</b> interface module, available as an option)</li> <li>In the menu: <ul style="list-style-type: none"> <li>Select submenu, select menu item</li> <li>Increase value (press briefly: Value is increased by 1 increment, press and hold: Value is increased until key is released)</li> </ul> </li> </ul>
11	print	<p>In the menu:</p> <ul style="list-style-type: none"> <li>Select submenu, select menu item</li> <li>Reduce value (press briefly: Value is reduced by 1 increment, press and hold: Value is reduced until key is released)</li> </ul>
12	input	<ul style="list-style-type: none"> <li>During weighing: <ul style="list-style-type: none"> <li>Enter patient data (age, gender, PAL)</li> </ul> </li> <li>In the menu: <ul style="list-style-type: none"> <li>Confirm selected menu item</li> <li>Save set value</li> </ul> </li> </ul>
13	menu	<ul style="list-style-type: none"> <li>During weighing: <ul style="list-style-type: none"> <li>Call up multifunctional display menu.</li> </ul> </li> <li>In the menu: <ul style="list-style-type: none"> <li>Press briefly: Go back one menu level</li> <li>Press and hold: Exit menu</li> </ul> </li> </ul>
14		Switch multifunctional display and scale on and off
15	clear	For deleting data (patient data, height, <b>BMI</b> , <b>BFR</b> ) entered manually or received wirelessly
16	gender	Enter the patient's gender
17	2 in 1	Start the <b>2 in 1</b> function for weighing babies and toddlers
18	hold	Activate the <b>Hold</b> function
19	Spirit level	Indicates whether the device is horizontal
20	Casters	For transporting short distances
21	Power supply connection	For connecting the power supply unit
22	Foot screws	For precise alignment

## 4.2 Symbols in the display

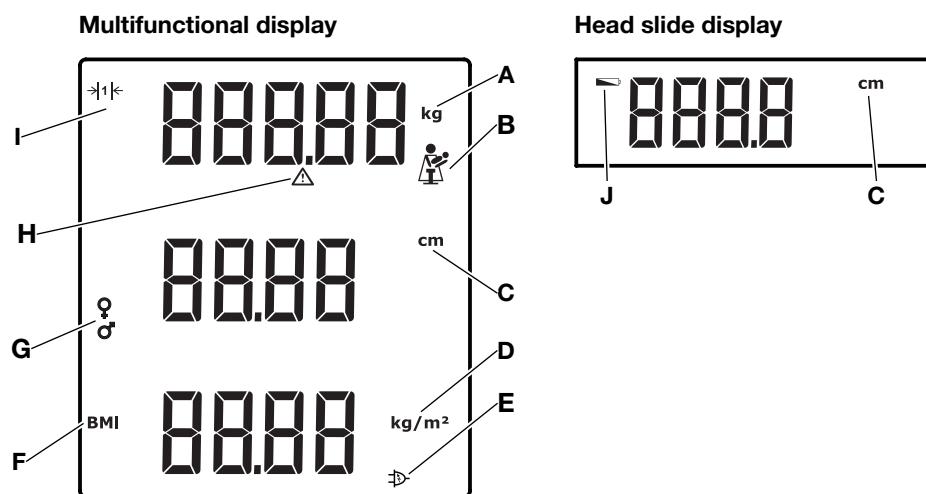Item	Symbol	Meaning
A	kg	Unit in which weight is displayed
B		2 in 1 function active for weighing babies and toddlers
C	cm	Unit in which height is displayed
D	kg/m <sup>2</sup>	Unit in which BMI is stated
E		Operation with power supply unit
F	BMI	Indicates that Body Mass Index (BMI) will be calculated automatically
G		Patient's gender
H		Non-verified function active (for technical reasons, symbol is displayed even on non-verified scales)
I	→ 1 ←	Active weighing range
J		Batteries are low

## 4.3 Labels


Markings on the device and on the type plate	
Symbol	Meaning
	Name and address of manufacturer, date of manufacture
UDI	Unique Device Identifier (product identification number)
REF	Article number

Markings on the device and on the type plate	
Symbol	Meaning
<b>SN</b>	Serial number
<b>GAL</b>	Value in m/s <sup>2</sup> (model-dependent) <ul style="list-style-type: none"> <li>• Gravitational acceleration on earth</li> <li>• Depends on the intended location</li> </ul>
<b>ProdID</b>	Product identification number
<b>Approval Type</b>	Type designation of design approval
	Follow instructions for use
	Medical electrical device, Type B
<b>IP20</b>	Type of protection in accordance with IEC 60529: <ul style="list-style-type: none"> <li>• Protection against ingress of solid foreign bodies with a diameter of over 12.5 mm</li> <li>• Protection against access with fingers</li> <li>• No protection from ingress of water</li> </ul>
<b>e</b>	Value in units of mass (verified models) Used to classify and verify a scale
<b>d</b>	Value in units of mass (non-verified models) States the difference between two consecutive display values
<b>→ x ←</b>	Active weighing range
	Class III scale to 2014/31/EU
	Device complies with EU directives <ul style="list-style-type: none"> <li>• <b>M:</b> Conformity label in compliance with directive 2014/31/EU for non-automatic scales (verified models)</li> <li>• <b>24:</b> (Example: 2024) Year in which declaration of conformity was implemented and the CE label was applied (verified models)</li> <li>• <b>0102:</b> Notified Body for Metrology (verified models)</li> <li>• <b>0123:</b> Notified Body for Medical Devices</li> </ul>
<b>MD</b>	Medical device in accordance with Regulation (EU) 2017/745
	Device meets the requirements of the USA and Canada. Certified and tested by a licensing laboratory (NRTL) of TÜV SÜD Product Services GmbH.
	Device complies with United Kingdom directives <ul style="list-style-type: none"> <li>• <b>M:</b> Conformity label in compliance with UK directive SI 2016 no. 1152 for non-automatic scales (NAWIR)(verified models)</li> <li>• <b>24:</b> (Example: 2024) Year in which declaration of conformity was implemented and the UKCA label was applied (verified models)</li> <li>• <b>xxxx:</b> Notified Body for Medical Devices of the United Kingdom</li> <li>• <b>yyyy:</b> Notified Body for Metrology of the United Kingdom (verified models)</li> </ul>
	Importer/representative in the United Kingdom: seca Ltd 40 Barn Street B5 5QB Birmingham United Kingdom

Markings on the device and on the type plate	
Symbol	Meaning
	Importer/representative in Switzerland: seca ag (schweiz) Medizinische Waagen und Messsysteme Schönmatt Str. 2 CH-4153 REINACH
	Type plate on the power supply connection socket <ul style="list-style-type: none"> <li>Power supply voltage required in V</li> <li>Maximum current consumption in mA</li> <li>---: Note polarity of device connector</li> <li>--: Operate device with direct current</li> <li>: Use compatible seca power supply units only</li> </ul>
	Do not dispose of device in household waste

Markings on the packaging	
Symbol	Meaning
	Protect from moisture
	Arrows indicate top of product Transport and store in an upright position
	Fragile, do not throw or drop
	Permitted min. and max. temperature for transport and storage
	Permitted min. and max. humidity for transport and storage
	Permitted min. and max. air pressure for transport and storage
	Open packaging here
	Packaging material can be disposed of through recycling programs

## 4.4 Menu structure for multifunctional display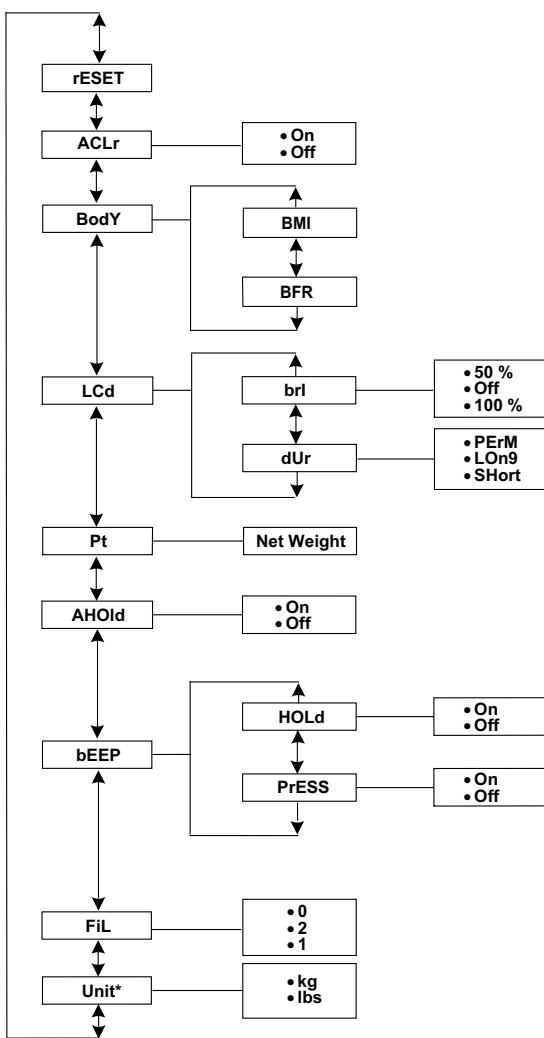

\* Non-verified scales

<b>ACLr</b>	Autoclear
<b>BMI</b>	Body Mass Index
<b>BFR</b>	Body Fat Ratio
<b>LCd</b>	Liquid Crystal Display
<b>brl</b>	Brightness
<b>dUr</b>	Duration
<b>PErM</b>	Permanent
<b>LOn9</b>	Long
<b>Pt</b>	Pre-Tare
<b>AHold</b>	Autohold

## 4.5 Navigating in the menu of the display

1. Press the key of the multifunctional display.
2. Press the **menu** key.  
⇒ The last menu item selected is displayed (in this case: Autohold **AHOld**).
3. Press the **▲ (send)** or **▼ (print)** key until the desired menu item is displayed (in this case: filtering **FiL**).
4. Confirm the selection by pressing the **input** key.  
⇒ The current setting for the menu item or a submenu are displayed (in this case: level **0**).
5. To change the setting, press the **▲ (send)** or **▼ (print)** key until the desired setting is displayed (in this case: level **2**).
6. Confirm the setting by pressing the **input** key.  
⇒ The setting is adopted.  
⇒ The menu closes.


**NOTE**

Briefly press the **menu** key to go back one menu level.

Press and hold the **menu** key to exit the menu at any time.

If no key is pressed for about 30 seconds, you exit the menu automatically.

## 4.6 Menu structure for head slide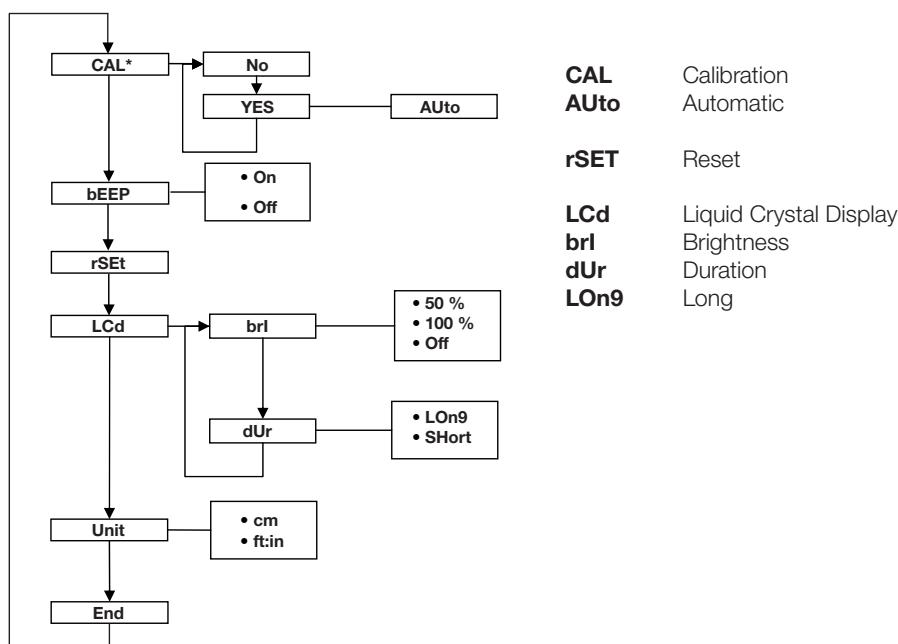

\* If measured value is shown in the display

## 4.7 Navigating in the menu of the head slide

1. Press the key of the head slide.  
⇒ ---- is shown in the display.

**bEEP**

2. Press and hold the and keys simultaneously.  
⇒ The last menu item selected is displayed (in this case: **bEEP**).

**Lcd**

3. Press the key until the desired menu item is displayed (in this case: **Lcd** display settings).

**dUr**

4. Confirm the selection by pressing the key.  
⇒ The current setting for the menu item or a submenu are displayed (in this case: Duration of display backlighting **dUr**).

**br l**

5. To amend the setting or call up a different submenu, press the key until the desired setting is displayed (in this case: Display brightness **brl**).

**50**

6. Confirm the selection by pressing the key.  
⇒ The current setting for the menu item or a submenu are displayed (in this case: Display brightness **50%**).

**100**

7. To amend the setting or call up a different submenu, press the key until the desired setting is displayed (in this case: Display brightness **100 %**).


8. Confirm the setting by pressing the key.  
⇒ The setting is adopted.  
⇒ The menu closes.

**NOTE**

If no key is pressed for about 30 seconds, you exit the menu automatically.

## 5 STARTING UP DEVICE

### 5.1 Scope of delivery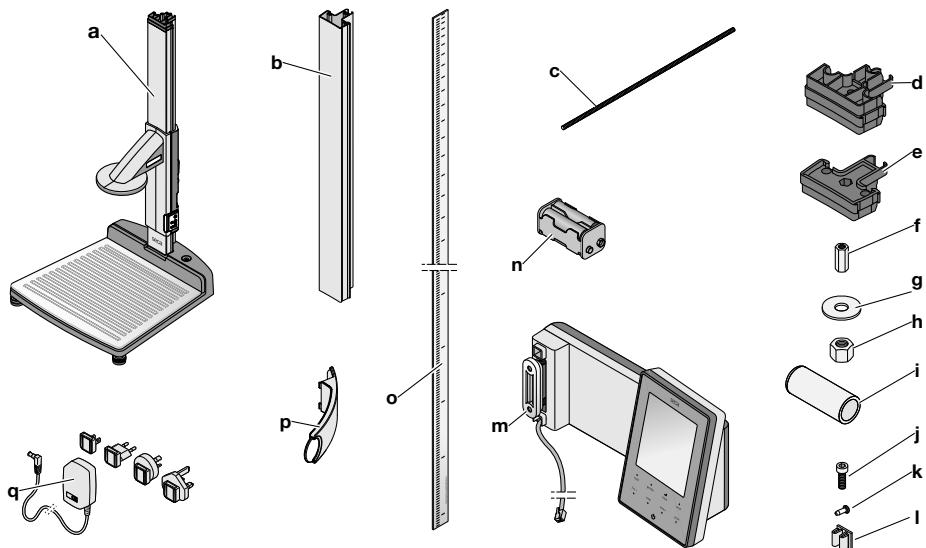


Item	Component	Pcs.
<b>a</b>	Lower part of device, preassembled: • First column element • Head slide	1
<b>b</b>	Column element	2
<b>c</b>	Threaded rod	2
<b>d</b>	Column connector	1
<b>e</b>	End cap	1
<b>f</b>	Coupling nut	1
<b>g</b>	Plain washer	1
<b>h</b>	Nut	1
<b>i</b>	Decorative cap	1
<b>j</b>	Hex head socket screw	2
<b>k</b>	Cross-head screw	2
<b>l</b>	Cable clips	3
<b>m</b>	Multifunctional display with connector cable	1
<b>n</b>	Battery holder with 4 batteries (AA-type)	1
<b>o</b>	Push-in scale	1
<b>p</b>	Frankfurt measure	1
<b>q</b>	Power supply unit	1
-	Calibration rod	1
-	Instructions for use	1

Item	Component	Pcs.
-	Cover for Frankfurt measure	1
-	Tool kit: • Hex socket wrench, 5 mm • Open-ended spanner, 10 mm	1

## 5.2 Assembling device


**Introduction** The first column element and the head slide are assembled on the lower part of device as standard.

- Use an assistant to perform the assembly.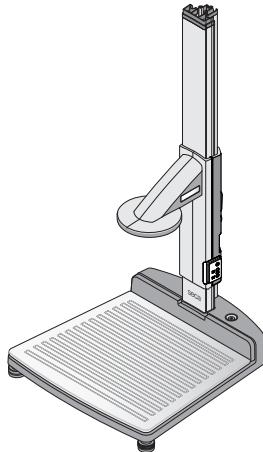


### Fitting Frankfurt measure


1. Hold the Frankfurt measure up against the head slide as shown in the illustration.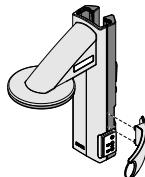

2. Insert the two cross-head screws in the bores.
3. Tighten the cross-head screws (2 Nm).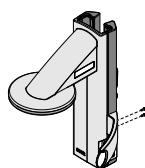

### Fitting second column element

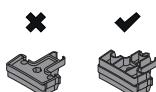
#### NOTICE!


##### Damage to device as a result of assembly errors

If the column connector and the end cap are switched, it will not be possible to assemble the device so that it is fit for purpose.

- Ensure that you use the column connector when assembling the second column element.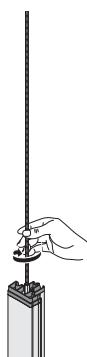


1. Screw a threaded rod (approx. 3 Nm) into the coupling nut of the first column element.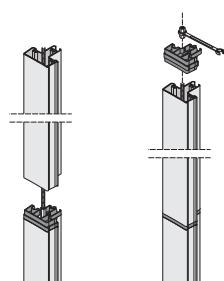


2. Push a column element over the threaded rod onto the first column element.
3. Place the column connector on the second column element so that the threaded rod protrudes out of the bore in the column connector.
4. Screw a coupling nut onto the threaded rod.
5. Tighten the coupling nut.

### Fitting multifunctional display

The multifunctional display is suspended in the groove of the column elements and fixed in position with a screw clamp. Any assembly height can be selected.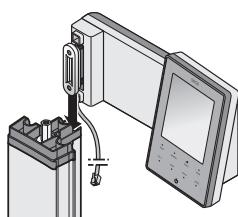

1. Insert the tenon block of the multifunctional display in the groove of the second column element.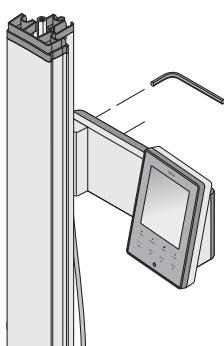

2. Tighten up the two hex socket screws (8 Nm) to fix the multifunctional display in the desired position.
3. Route the connector cable of the multifunctional display in the groove of the column elements as far as the lower part of device.
4. Draw the connector cable through the bore of the lower part of device.


#### NOTICE!

##### Malfunction as a result of assembly errors


If the cables are fitted under severe mechanical load, faulty displays and failure of the display may result.

- Route all cables to prevent sharp bends and kinked connectors.


5. Wind the free end of the connector cable onto the cable store in the lower part of device.
6. Plug the connector of the connector cable into the appropriate socket of the weighing platform.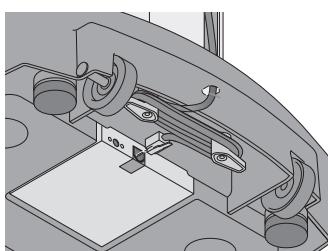


- Fix the connector cable in the groove of the column elements using the cable clips.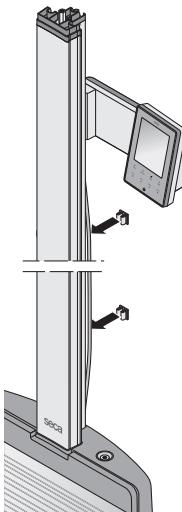

- Swivel the multifunctional display so that it is convenient for you to read.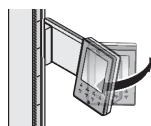

### Fitting third column element

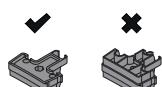
#### **NOTICE!**

#### **Damage to device as a result of assembly errors**


If end cap and column connector are switched, it will not be possible to assemble the device so that it is fit for purpose.

► Ensure that you use the end cap when assembling the third column element.

- Screw a threaded rod (approx. 3 Nm) into the coupling nut of the second column element.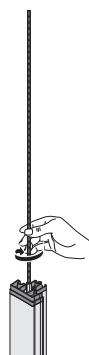

- Push the column element over the threaded rod onto the column connector of the second column element.
- Place the end cap on the third column element so that the threaded rod protrudes out of the bore in the end cap.
- Position the plain washer on the free end of the threaded rod.
- Position the nut on the free end of the threaded rod.
- Tighten the nut (3 Nm).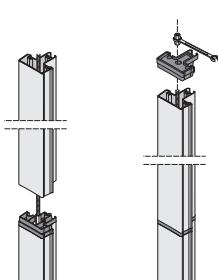

**NOTICE!****Damage to device and malfunction as a result of assembly errors**

If the nut on the end cap is not tightened enough, the column elements will be too loosely connected. The device will be unstable and faulty measurements will occur.


- ▶ Tighten the nut to the specified torque so that the columns are located firmly on one another and there is no gap between the column connector and the third column element.

7. Put the decorative cap on the nut.

**Fitting push-in scale****NOTICE!****Faulty measurements as a result of assembly errors**

If the push-in scale is assembled incorrectly, the head slide cannot determine any measured values which can be used.

- ▶ Align the push-in scale so that the printing remains visible when it is pushed into the groove.
- ▶ Align the push-in scale so that the area with the article number forms the bottom end.
- ▶ Ensure that the push-in scale engages underneath the end cap.
- ▶ Push the push-in scale into the groove on the side of the column elements until the scale engages under the end cap.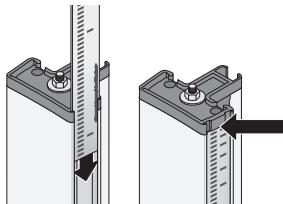

## 5.3 Setting up device

**CAUTION!****Faulty measurement as a result of devices being set up incorrectly**

Soft floors (wooden boards, for example) give under the patient's weight and falsify the measuring result. Incorrect ambient conditions or lack of alignment lead to measurement errors.

- ▶ Select a setup location with a level, stable floor in order to achieve precise measuring results.
- ▶ Use the device only in the intended ambient conditions outlined in the section entitled "Technical data".
- ▶ Align the device before using it and every time it changes location.

**CAUTION!****Injury from a lack of stability**

If the foot screws are screwed out too far, they may come loose from the device. The device will then be unstable.

- ▶ Screw the foot screws out a maximum of 10 mm.
- ▶ If the device cannot be aligned horizontally with the screws screwed out as far as possible, the setup location is unsuitable. Set the device up in a suitable location.

**NOTICE!****Faulty measurement**


Under certain conditions, part of the patient weight will not be included in the measurement. The measuring result is then incorrect.

- Set up the device so that only its foot screws are in contact with the ground.
- Set up the device so that the weighing surface is not in contact with any other objects.
- Set up the device so that the glass plate is in free contact with the lower part of device and is not interfered with by other objects, e.g. a cable.
- During weighing, ensure that the patient does not touch the column of the device.

1. Place the device on a firm, level surface.

2. Undo the knurled wheels.

3. Align the scale by turning its foot screws so that the air bubble of the spirit level is in the exact center of the circle.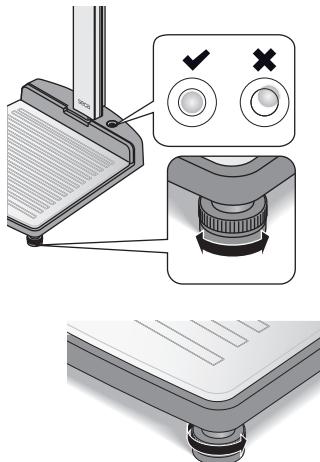

4. Tighten the knurled wheels in the direction of the arrow.  
⇒ The foot screws are secured against being adjusted.


## 5.4 Transporting device

The device is equipped with two casters which enable transport over short distances.

**CAUTION!****Injury, damage to device**

The device must be tilted for transport. If the device is tilted without care, this may lead to injuries and damage to the device.

- Ensure that there are no other persons in the immediate vicinity.
- Ensure that there are no objects in the immediate vicinity.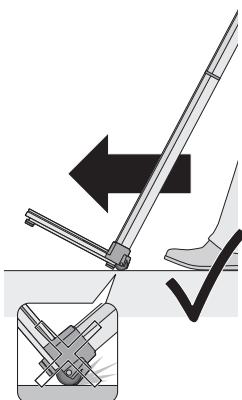



1. Disconnect the power supply unit from the power supply socket.
2. Tilt the device until it can be moved freely on the casters.
3. Transport the device to its new setup location.
4. Return the device to an upright position.
5. Plug the power supply unit into the power supply socket.
6. Align the device → [Setting up device, page 66](#).
7. Perform a function check → [Function check, page 82](#).

## 5.5 Establishing power supply

### Inserting batteries

The battery holder supplied already contains 4 AA-type batteries (1.5 V). To establish the power supply for the head slide, proceed as follows:

1. Take off the cover of the battery compartment.
2. Remove the connector cable from the battery compartment.

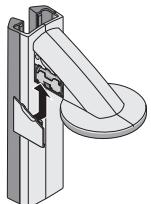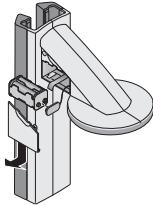3. Connect the battery holder to the connector cable.
4. Insert the battery holder in the battery compartment.

#### NOTICE!

##### Damage to device and malfunction

If the battery compartment is not closed correctly, the battery cable may be damaged.

- When closing the battery compartment, ensure that the battery cable is not trapped between the cover and the battery compartment.

5. Push the cover onto the battery compartment until you hear it engage.

### Connecting power supply unit

The connection for the power supply unit is located on the underside of the lower part of device. To establish the power supply for the scale and multifunctional display, proceed as follows:


#### WARNING!

##### Personal injury or damage to the device as a result of incorrect power supply units


Conventional power supply units may deliver a higher voltage than is indicated on them. The measuring device may overheat, catch fire, melt or short-circuit.

- Use only original seca power supply units with a controlled 12 V output voltage.

1. Tilt the device forward.
2. Insert the device connector of the power supply unit into the connecting socket of the scale.
3. Carefully return the device to an upright position.

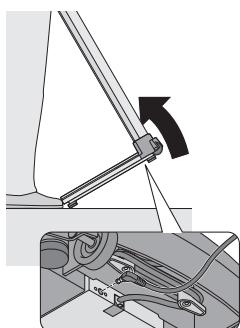
4. Insert the connector required for your power supply into the power supply unit.
5. Plug the power supply unit into a power supply socket.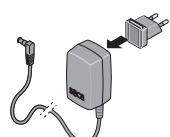


## 5.6 Calibrating height measurement

### Calibrating automatically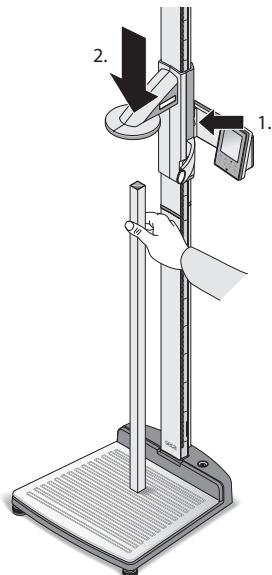

You can perform automated calibration with the aid of the calibration rod supplied.


1. Switch on the head slide.  
⇒ ---- appears in the display.
2. Press and hold the brake button and move the head slide until a height value is shown in the display.
3. Place the calibration rod supplied (0.8 m long) vertically against the lower part of device and keep it in this position.
4. Press and hold the brake button and push the head slide onto the calibration rod.
5. From the head slide menu, select the **CAL** menu item.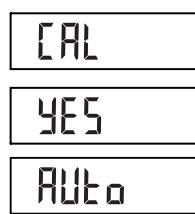


6. Confirm your selection.
7. Select the setting **YES**.
8. Confirm your selection.  
⇒ The **AUto** setting is shown in the display.
9. Confirm the setting.  
⇒ The head slide is calibrated.  
⇒ You can perform height measurements.

### Calibrating manually

You can perform calibration manually using a folding rule (alternative to the calibration rod supplied).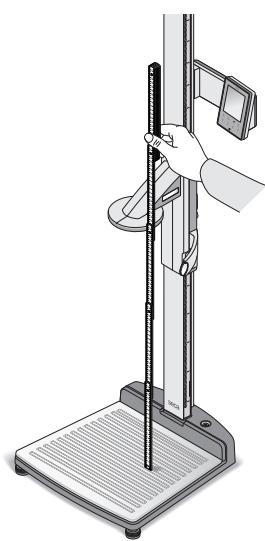

1. Switch on the head slide.  
⇒ ---- appears in the display.
2. Press and hold the brake button and move the head slide until a value is shown in the display.
3. Push the head slide to a position specified using the folding rule (e.g. 83.5 cm).

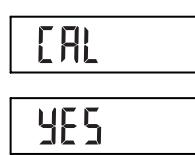4. From the head slide menu, select the **CAL** menu item.
5. Confirm your selection.
6. Select the setting **YES**.

7. Confirm your selection.  
⇒ The **AUTO** setting is shown in the display.
  
8. Enter the position of the head slide (in this case: 83.5 cm).
9. Confirm the entry.  
⇒ The head slide is calibrated.  
⇒ You can perform measurements.

## 5.7 Disinfecting device

- Disinfect the following components **before** initial commissioning (→ **Disinfecting, page 81**):
  - Weighing platform
  - Column
  - Multifunctional display
  - Head slide with Frankfurt measure

# 6 OPERATING THE SCALE


### CAUTION! Personal injury/faulty measurement

- Before using the device each time, perform a function check as described in the corresponding section of these instructions for use.

## 6.1 Weighing

### Switch on device

1. Ensure that there is no load on the scale.
2. Press the key of the multifunctional display.  
⇒ All the elements of the display are shown briefly.  
⇒ **SECA** appears in the display.  
⇒ The scale is operational when **0.00** appears in the display.

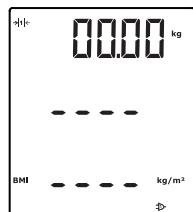### Performing weighing

### Activating/deactivating key lock

1. Ask the patient step onto the scale.  
⇒ The patient's weight is displayed.
2. Read off the measuring result.

Use the key lock to prevent undesired settings being made to the multifunctional display once the device has been switched on.

#### NOTE


No separate symbol for an activated key lock is displayed in the multifunctional display.

Key lock cannot be activated while you are navigating in the menu.

1. Ensure that the device is switched off.
2. Press the key of the multifunctional display.
3. Wait until **SECA** is displayed.

4. Press the **menu** key of the multifunctional display.  
⇒ The key lock is activated.
5. To deactivate the key lock, press the  key of the multifunctional display again.
6. Wait until **SECA** is displayed.
7. Press the **menu** key of the multifunctional display.  
⇒ The key lock is deactivated.

## Weighing babies/toddlers (2 in 1)

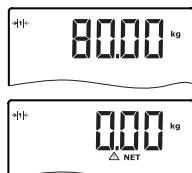The **2 in 1** function can be used to determine the weight of babies and toddlers. The child is held in an adult's arms during weighing to do this. Proceed as follows:

1. Ensure that there is no load on the scale.
2. Ask the adult to stand on the scale.  
⇒ The adult's weight is displayed.
3. Press the **2 in 1** key.  
⇒ The adult's weight is stored as the initial weight.
- ⇒ **0.00**, the  symbol and the **NET** message are displayed.

### NOTICE!

#### Faulty measurement as a result of initial weight changing

If the child is weighed together with a different adult, it will not be possible to determine the child's weight correctly.

- Ensure that the child is always weighed with the adult with whom the initial weight was determined.
  - Ensure that the adult's weight does not change - as a result of their taking off a piece of clothing, for example.
4. Ask the adult to get onto the scale holding the child.  
⇒ The child's weight is displayed.
  - ⇒ The symbols  and  as well as the messages **HOLD** and **NET** are displayed.
  5. To deactivate the function, press the **2 in 1** key or switch off the scale.

If you activate the **Hold** function, the measuring result continues to be displayed once the weight has been removed from the scale. This enables you to attend to the patient before recording the measuring result.

### NOTE

When the **Autohold** function is activated, the measuring result is automatically displayed permanently as soon as a stable weighing value has been achieved.

1. Ensure that there is no load on the scale.
2. Ask the patient to step onto the scale.
3. Press the key.  
⇒ The display flashes until a stable weight is measured. The measuring result is then displayed permanently.  
⇒ The  symbol and the **HOLD** message are displayed.
4. To deactivate the function, press the key.  
⇒ The  symbol and the **HOLD** message are no longer displayed.

## Displaying measuring result permanently (hold)


You can enter the following patient data directly on the multifunctional display: Age, gender, and physical activity level. The patient data are sent with the measuring results when you send them to a compatible seca product or an EMR system.

1. Press the **input** key.  
⇒ The first time this menu is called up after the device starts up, the **PAL** (physical activity level) parameter is displayed.  
⇒ If you call this menu up again whilst the device is switched on, the last parameter selected will be displayed.
2. Use the **▲ send** or **▼ print** key to select a menu item:

## Entering patient data (input)

Parameter	Meaning
<b>PAL</b>	Physical activity level
<b>AGE</b>	Age
<b>GEN</b>	Gender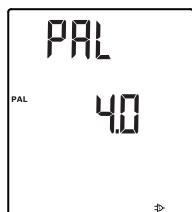

3. Confirm your selection.  
⇒ The value from the previous measurement is displayed.
4. You have the following options for continuing:
  - Adopt value
  - Change value: Press the **▲ send** or the **▼ print** key

Function	Setting
Physical activity level ( <b>PAL</b> )	1.0 to 5.0
Age ( <b>AGE</b> )	<ul style="list-style-type: none"><li>• Up to 3 years in months</li><li>• Up to 18 years in half-years</li><li>• From 18 years in years</li></ul>
Gender ( <b>GEN</b> )	<ul style="list-style-type: none"><li>• Male</li><li>• Female</li></ul>

5. Press the **input** key.  
⇒ The values will be saved.  
⇒ You exit the function automatically.
6. If you also want to make settings for **AGE** and **GEN**, repeat the process.

### Entering patient's gender

#### NOTE

This setting overwrites the setting made under \.

- In order to switch between "male" and "female", press the key.

Body Mass Index puts height and weight in a relationship to one another.

1. Ensure that the **BMI** calculation method is displayed on the multifunctional display  
→ [Switching between BMI and BFR \(bodY\), page 74](#).
2. Ask the patient to step onto the weighing platform.  
⇒ The patient's weight is displayed.
3. If weight is to be displayed permanently, press the **hold** key.  
⇒ The patient's weight is displayed permanently.


#### NOTE

When the **Autohold** function is activated, the measuring result is automatically displayed permanently as soon as a stable weighing value has been achieved.

4. Measure the patient's height → [Starting height measurement, page 78](#).
5. Press the **1** key on the head slide briefly.  
⇒ Height is displayed on the multifunctional display.  
⇒ **BMI** is calculated and displayed automatically.
6. Ask the patient to step off the weighing platform.
7. To prevent out-of-date data leading to faulty results, press the **clear** key.  
⇒ Patient data, height, and **BMI** will be deleted.

#### NOTE

If the **ACLR** function is activated, measured and calculated values will be deleted automatically after 5 minutes.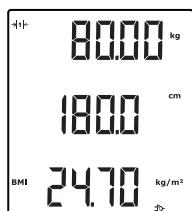

## Determining Body Fat Ratio (BFR)


Body Fat Ratio puts height, weight, and gender in a relationship to one another.

1. Ensure that the **BFR** calculation method is displayed on the multifunctional display → [Switching between BMI and BFR \(bodY\), page 74](#).
2. In order to switch between "male" and "female", press the key.
3. Ask the patient to step onto the weighing platform.  
⇒ The patient's weight is displayed.
4. Press the **hold** key.  
⇒ The patient's weight is displayed permanently.

### NOTE

When the **Autohold** function is activated, the measuring result is automatically displayed permanently as soon as a stable weighing value has been achieved.

5. Measure the patient's height → [Starting height measurement, page 78](#).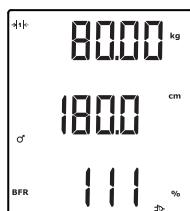

6. Press the **H** key on the head slide briefly.  
⇒ Height is displayed on the multifunctional display.  
⇒ **BFR** is calculated and displayed automatically.
7. Ask the patient to step off the weighing platform.
8. To prevent out-of-date data leading to faulty results, press the **clear** key.  
⇒ Patient data, height, and **BFR** will be deleted.

### NOTE

If the **ACLR** function is activated, measured and calculated values will be deleted automatically after 5 minutes.

## Clearing stored values (clear)

Out-of-date measuring results and patient data lead to faulty calculation of Body Mass Index (**BMI**)/Body Fat Ratio (**BFR**). Use the **clear** key to delete the following measured values and patient data:

- Height
- **BMI**
- **BFR**
- Gender
- **PAL** (physical activity level)
- Age

### NOTE


If the **ACLR** function is activated, measured values and patient data will be deleted automatically after 5 minutes → [Deleting values automatically \(ACLR\), page 74](#).

- ▶ Press the **clear** key.  
⇒ Patient data, height, and Body Mass Index (**BMI**)/Body Fat Ratio (**BFR**) are deleted. ---- is displayed instead.  
⇒ The gender symbol goes out.

## Switching weighing range automatically

The scale has two weighing ranges. In weighing range 1 (→|1|←), graduations in the weight display are finer. In weighing range 2 (→|2|←), you can use the maximum capacity of the scale.

After the scale is switched on, weighing range 1 is active.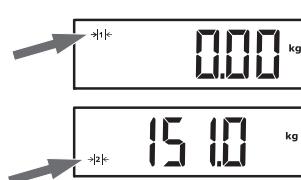

If a particular weight value is exceeded, the scale automatically switches to weighing range 2.

- ▶ To switch back to weighing range 1, completely remove the weight from the scale.  
⇒ Weighing range 1 is active again.
- ▶ Press the **H** key.  
⇒ The multifunctional display goes out.

## Switching off the scale

## 6.2 Other functions (menu)

### Deleting values automatically (ACLr)

Out-of-date measuring results and patient data lead to faulty calculation of **BMI/BFR**.

You can set the device so that the measuring results and patient data below are deleted automatically after 5 minutes:

- Gender
- Physical activity level (**PAL**)
- Age
- Height
- **BMI**
- **BFR**

#### NOTE

If you want to enter **PAL**, age, and gender for the next measurement, the values of the previous measurement will be suggested → [Entering patient data \(input\), page 71](#).

This function is activated at the factory on some models. You can deactivate the function if desired.

ACLr

1. From the menu, select the **ACLr** menu item.
2. Confirm your selection.  
⇒ The current setting is displayed.
3. Select the desired setting:
  - **On**
  - **OFF**
4. Confirm your selection.  
⇒ The menu closes.  
⇒ Patient data, height, and **BMI/BFR** will in each case be deleted 5 minutes after a measurement. ---- is displayed instead.  
⇒ The gender symbol goes out.

### Switching between BMI and BFR (body)

You can select whether the measuring station calculates Body Mass Index (**BMI**) or Body Fat Ratio (**BFR**). The current setting is displayed permanently in the multifunctional display.

The calculation is performed automatically as soon as the patient's weight and height have been measured.

body

1. From the menu, select the **body** menu item.
2. Confirm your selection.  
⇒ The current setting is displayed.
3. Press the **▲ send** or **▼ print** key to switch between **bMi** and **bFr**.
4. Confirm your selection.  
⇒ The setting is adopted.  
⇒ The menu closes.  
⇒ The setting is displayed in the multifunctional display.

### Setting display backlighting (LCD)

You can change the duration and brightness of display backlighting.

Lcd

1. From the menu, select the **Lcd** menu item.
2. Confirm your selection.
3. Select a menu item.
  - Duration (**dUr**)
  - Brightness (**bri**)
4. Confirm your selection.  
⇒ The current setting is displayed.

- Select the desired setting.

Function	Setting
Duration	<ul style="list-style-type: none"> <li><b>SHrt</b> (approx. 15 sec.)</li> <li><b>LOnG</b> (approx. 45 sec.)</li> <li><b>PErM</b> (permanent)</li> </ul>
Brightness	<ul style="list-style-type: none"> <li><b>50 %</b></li> <li><b>100 %</b></li> <li><b>OFF</b></li> </ul>

- Confirm your selection.  
 ⇒ The setting is adopted.  
 ⇒ The menu closes.

### Entering height manually (**HGHt**)


You can enter the patient's height manually - if the head slide is not working, for example.

- From the menu, select the **HGHt** menu item.
- Confirm your selection.  
 ⇒ The current value flashes in the display.
- You have the following options for continuing:  
 ► Adopt value  
 ► Change value: Press the **▲ send** or the **▼ print** key
- Confirm the displayed value.  
 ⇒ The setting is adopted.  
 ⇒ The menu closes.

### Saving additional weight permanently (**Pt**)


Use the **Pre-Tare** function to save an additional weight permanently and have it deducted from a measuring result automatically. For example, you can save a flat-rate figure for shoes and clothing and then always deduct this from the measuring result if a patient is weighed fully-clothed.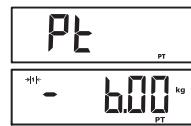

- From the menu, select the **Pt** menu item.  
 ⇒ The last additional weight set flashes in the display (with a minus sign in front of it).  
 ⇒ The **PT** message is displayed.
- You have the following options for continuing:  
 ► Adopt value  
 ► Change value: Press the **▲ send** or the **▼ print** key

#### NOTE

When you enter the value **0**, the function is switched off. The **PT** message is no longer shown in the display.

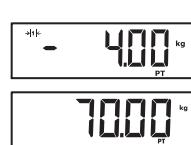- Confirm your selection.  
 ⇒ The set additional weight is displayed with a minus sign in front of it.
- Ask the patient to step onto the weighing platform.  
 ⇒ The **PT** message is displayed.  
 ⇒ The patient's weight is displayed.  
 ⇒ The saved additional weight has been deducted automatically.
- To deactivate the function, select the **Pt** menu item from the menu again.
- Confirm your selection.  
 ⇒ The set additional weight is no longer displayed.  
 ⇒ The function is deactivated.  
 ⇒ The menu closes.

## **Activating Autohold (AHOLD) function**

When you activate the **Autohold** function, the measuring result for every weighing operation is displayed for every weighing procedure after the weight has been removed from the scale. It is then no longer necessary to activate the **Hold** function manually.

### **NOTE**

Regardless of the setting selected here, the weight of the child is always determined by **Autohold** in the **2 in 1** function.

1. From the menu, select the **AHOLD** menu item.
2. Confirm your selection.  
⇒ The current setting is displayed.
3. Select the desired setting:
  - **On**
  - **OFF**
4. Confirm your selection.  
⇒ The setting is adopted.  
⇒ The menu closes.

## **Activating/deactivating beeps (bEEP)**

You can activate beeps for a number of device functions:

1. From the menu, select the **bEEP** menu item.
2. Confirm your selection.
3. Select a menu item:
  - **PrESS**: beep when key is pressed
  - **HOLD**: beep when weighing value is stable
4. Confirm your selection.  
⇒ The current setting is displayed.
5. Select the desired setting:
  - **On**
  - **OFF**
6. Confirm your selection.  
⇒ The setting is adopted.  
⇒ The menu closes.

## **Setting filtering (FIL)**

Setting the filter (**FIL** = filter) allows you to reduce interference when determining weight. The selected setting affects the sensitivity with which the weight display reacts to patient movements and the time until the **Hold** function displays a weight value permanently(**Hold** value).

<b>Setting (FIL)</b>	<b>Sensitivity</b>	<b>Determination of Hold value</b>
<b>0</b>	Sensitive	Slow
<b>1</b>	Medium	Medium
<b>2</b>	Insensitive	Fast

### **NOTE**

With the **0** setting and patients who are not very steady on their feet, it is possible that no weight value is displayed permanently despite the **Hold** function being activated.

The greatest deviation between weight value displayed and actual value occurs at setting **2**.

1. From the menu, select the **FIL** menu item.
2. Confirm your selection.  
⇒ The current setting is displayed.
3. Select a filtering level.

4. Confirm your selection.  
⇒ The setting is adopted.  
⇒ The menu closes.

**Non-verified models:  
Switching weight unit  
(Unit)**

You can select the unit in which weight is to be displayed.

**WARNING!**  
**Patient hazard**

To prevent misinterpretations, measuring results for medical purposes must only be displayed and used in SI units (weight: kilograms/grams, height: meters/cm). Some devices have the option of displaying measuring results in different units. This is purely an additional function.

- Only use measuring results in SI units.
- The user takes sole responsibility for the use of measuring results in non-SI units.

**Unit**

1. From the menu, select the **Unit** menu item.
2. Confirm your selection.  
⇒ The current setting is displayed.
3. Select the unit in which weight is to be displayed:
  - Kilograms/grams (**gr**)
  - Pounds (**lbS**)
4. Confirm your selection.  
⇒ The setting is adopted.  
⇒ The menu closes.

**Restoring factory settings  
(rESEt)**

You can restore the factory settings for the functions below:

Function	Factory setting
Autohold ( <b>AHOLd</b> )	<b>On</b>
Beep ( <b>PrESS</b> )	<b>On</b>
Beep ( <b>HOLD</b> )	<b>On</b>
Filtering ( <b>FIL</b> )	<b>0</b>
Autoclear ( <b>ACLr</b> )	<b>On</b>
Pre-tare ( <b>Pt</b> )	<b>0 kg</b>
Unit, weight ( <b>Unit</b> )	<b>kg (gr)</b>
Display backlighting (brightness <b>bri</b> )	<b>50 %</b>
Display backlighting (duration <b>dUr</b> )	Permanent ( <b>PErM</b> )
Body Mass Index ( <b>BMI</b> )/Body Fat Ratio ( <b>BFR</b> )	Body Mass Index ( <b>bMi</b> )
Physical activity level ( <b>PAL</b> )	<b>1.0</b>
Age in years	<b>18</b>
Age in months	<b>0</b>
Height for calculating Body Mass Index ( <b>BMI</b> )/Body Fat Ratio ( <b>BFR</b> )	<b>170 cm</b>

**rESEt**

1. From the menu, select the **rESEt** menu item.
2. Confirm your selection.  
⇒ The menu closes.
3. Switch off the scale.  
⇒ Factory settings are restored and are available when the scale is switched on.

## 7 OPERATING THE HEAD SLIDE

### CAUTION! Personal injury/faulty measurement

- Before using the device each time, perform a function check as described in the corresponding section of these instructions for use.

### 7.1 Measuring height

#### Switching on head slide



- Press the key of the head slide.
  - ⇒ ---- is shown in the display.
  - ⇒ The head slide is operational.

#### Starting height measurement

##### NOTE

If you want to use the measuring result for automatic calculation of Body Mass Index (**BMI**) or Body Fat Ratio (**BFR**), ensure that the multifunctional display is also switched on.

1. Press and hold the brake button and move the head slide until a measuring result is shown in the display.
2. Press and hold the brake button and move the head slide upwards so that the patient can position him or herself underneath it comfortably.
3. Ask the patient to stand under the head slide:
  - Back to the column of the device
  - Upright posture: Back and head straight
  - Heels against the heel stop
4. Press and hold the brake button and push the head stop down until it is in contact with the patient's head.

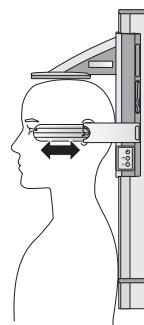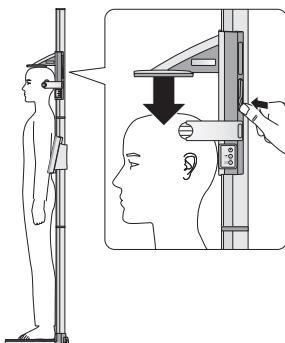5. Take the Frankfurt measure out of the head slide.
6. Align the patient's Frankfurt horizontal planes with the three lines of the Frankfurt measure.
7. Press and hold the brake button and correct the position of the head slide.
8. Read off the measuring result.

#### Displaying measuring result permanently (**Hold**)

If you activate the **Hold** function, the measured value continues to be displayed after the measurement. This allows you to move the head slide before you note the measured value.

1. Position the head slide.
2. Briefly press the key.
  - ⇒ The **HOLD** message is displayed.
  - ⇒ The head slide can be moved without the displayed measured value changing.
3. To deactivate the **Hold** function, press the key.
  - ⇒ The **HOLD** message is no longer displayed.

## Performing relative measurements (ZERO)

The head slide display can be zeroed in any position. This function facilitates relative measurements – for measuring extremities, for example.

1. Press the  key of the head slide.  
⇒ ---- is shown in the display.
2. Press and hold the brake button and move the head slide until a height value is shown in the display.
3. Position the head slide at the point at which you would like to zero the head slide display.
4. Press and hold the  key until the **ZERO** message is displayed.  
⇒ The display is zeroed at the current position of the head slide.
5. Reposition the head slide.  
⇒ Height is measured relative to the set zero point.  
⇒ If the zero point is undershot, the measured values will be shown with a minus sign in front.
6. To deactivate the **zero** function, press and hold the  key until the **ZERO** message is no longer displayed.

### NOTE

If for documentation purposes, you send results of relative measurements to devices which calculate Body Mass Index or Body Fat Ratio automatically, implausible values will result for these parameters.

## Switching off head slide

- Press the  key of the head slide.  
⇒ The head slide display goes out.

### NOTE

The head slide switches off automatically after a short time if it is not moved.

## 7.2 Other functions (menu)

### Setting display backlighting (LCD)

You can change the duration and brightness of display backlighting.

1. From the menu, select the **LCD** menu item.
2. Confirm your selection.
3. Select a menu item:
  - Duration (**dUr**)
  - Brightness (**bri**)
4. Confirm your selection.  
⇒ The current setting is displayed.
5. Select the desired setting.

Function	Setting
Duration ( <b>dUr</b> )	<ul style="list-style-type: none"> <li>• <b>SHrt</b> (approx. 15 sec.)</li> <li>• <b>LOnG</b> (approx. 45 sec.)</li> </ul>
Brightness ( <b>bri</b> )	<ul style="list-style-type: none"> <li>• <b>50 %</b></li> <li>• <b>100 %</b></li> <li>• <b>OFF</b></li> </ul>

6. Confirm your selection.  
⇒ The setting is adopted.  
⇒ The menu closes.

**Activating beeps (bEEP)** You can set whether a beep is heard every time a key is pressed.

bEEP

1. From the menu, select the **bEEP** menu item.
2. Confirm your selection.  
⇒ The current setting is displayed.
3. Select the desired setting:
  - **On**
  - **OFF**
4. Confirm your selection.  
⇒ The setting is adopted.  
⇒ The menu closes.

**Switching unit of height  
(Unit)**

 **WARNING!**  
**Patient hazard**

To prevent misinterpretations, measuring results for medical purposes must only be displayed and used in SI units (weight: kilograms/grams, height: meters/centimeters). Some devices have the option of displaying measuring results in different units. This is purely an additional function.

- Only use measuring results in SI units.
- The user takes sole responsibility for the use of measuring results in non-SI units.

**NOTE**

Height is sent to the multifunctional display and displayed there in the unit set here.

Unit

1. From the menu, select the **Unit** menu item.
2. Confirm your selection.  
⇒ The current setting is displayed.
3. Select the unit in which height is to be displayed:
  - **FEEt**: Feet, inches
  - **Mtr**: Centimeters
4. Confirm your selection.  
⇒ The setting is adopted.  
⇒ The menu closes.

**Restoring factory settings  
(rSET)**

You can restore the factory settings for the functions below:

Function	Factory setting
Beep (bEEP)	<b>On</b>
Display backlighting brightness (bri)	<b>50 %</b>
Display backlighting duration (dUr)	<b>SHrt</b>
Unit of height (Unit)	<b>Mtr (cm)</b>

rSET

1. From the menu, select the **rSET** menu item.
2. Confirm your selection.  
⇒ You exit the menu automatically.
3. Switch off the device.  
⇒ Factory settings are restored and are available when the device is switched back on.

## 8 HYGIENE TREATMENT

### **WARNING!** Electric shock

The device is not de-energized when the on/off button is pressed and the display goes out. Use of fluids on the device may cause an electric shock.

- ▶ Before each hygiene treatment, ensure that the device is switched off.
- ▶ Disconnect the power supply connector before each hygiene treatment.
- ▶ Take the batteries out of the device before every hygiene treatment.
- ▶ Ensure that no fluids penetrate the device.

### **WARNING!** Risk of infection

- ▶ Subject the device to a hygiene treatment at regular intervals as described in this section.

### **NOTICE!**

#### **Damage to device**

Unsuitable cleaning agents and disinfectants may damage the sensitive surfaces of the device and lead to transparent components becoming clouded.

- ▶ Do not use aggressive or abrasive cleaning agents.
- ▶ Do not use organic solvents (e.g. white spirit or petroleum spirit).

### 8.1 Cleaning

- ▶ If required, moisten a soft cloth with a mild soap solution and wipe the device over with it.

### 8.2 Disinfecting

1. Disinfect the device at regular intervals with a disinfectant suitable for sensitive surfaces and acrylic glass (e.g. 70 % ethanol).
2. Follow the instructions for use for the disinfectant.
3. Disinfect the device:
  - ▶ Moisten a soft cloth with disinfectant and wipe down the device with it.
  - ▶ Comply with the intervals, see table:

Interval	Component
<b>Before</b> each measurement	<ul style="list-style-type: none"> <li>• Weighing platform</li> <li>• Head slide</li> </ul>
<b>After</b> each measurement	<ul style="list-style-type: none"> <li>• Weighing platform</li> <li>• Head slide</li> </ul>
As required	<ul style="list-style-type: none"> <li>• Multifunctional display</li> <li>• Column</li> </ul>

### 8.3 Sterilizing

The device must not be sterilized.

## 9 FUNCTION CHECK

- Perform a function check before each use.

A complete function check includes:

- A visual inspection for mechanical damage
- A test of the alignment of the device
- A visual and function check of the display elements
- A function check of all the controls shown in the section entitled "Overview"
- A function check of the optional accessories

If you find faults or deviations during the function check, first try to remedy the fault with the aid of the "Troubleshooting" section in this document.

### CAUTION!

#### Personal injury

If you find faults or deviations during the function check which you are unable to remedy with the aid of the "Troubleshooting" section in this document, you must not use the device.

- Have the device repaired by seca Service or an authorized service partner.
- Follow the section entitled "Servicing" in this document.

## 10 TROUBLESHOOTING

### 10.1 General faults

#### NOTICE!

##### Malfunction

If the device is equipped with the optional **seca 452** external interface module and is connected to the **seca connect 103** software, error messages and error symbols may occur which are not described here.

- Follow the system instructions for use for **seca 103/452**.

If faults occur when operating the device, first attempt to remedy them yourself using the following table. If the fault persists, contact seca Service.

Fault	Cause	Remedy
No weight is displayed with a load on the scale.	The device has no power supply.	<ul style="list-style-type: none"><li>• Switch on device.</li><li>• In battery mode: Insert batteries.</li><li>• In power supply mode: Connect the power supply unit.</li></ul>
<b>0.00</b> is not displayed before weighing takes place.	A load was already on the device before it was switched on.	<ul style="list-style-type: none"><li>• Remove the load from the device.</li><li>• Switch off the device, then switch it back on again.</li></ul>
----- is displayed before weighing takes place.	A load was already on the device before it was switched on.	<ul style="list-style-type: none"><li>• Remove the load from the device.</li><li>• Switch off the device, then switch it back on again.</li></ul>

Fault	Cause	Remedy
The weight display flashes.	Consistent weight value not yet detected.	<ul style="list-style-type: none"> <li>One of the following functions is active: <b>Hold</b>, <b>Autohold</b>, <b>Tare</b>. Wait until the device has detected a consistent weight value. The display stops flashing.</li> <li>Take the load off the device, wait until <b>0.00</b> is displayed and measure again.</li> </ul>
Implausible measuring results.	Patient moved too much during the measurement.	Take the load off the device, wait until <b>0.00</b> is displayed and repeat the measurement procedure.
One display segment is displayed continuously or not at all.	The display is defective.	Inform seca Service.
Calibration of height measurement cannot be performed. The <b>CAL</b> menu item is not displayed in the head slide menu.	The head slide was not moved before the menu was called up.	<ul style="list-style-type: none"> <li>Move the head slide until a measured value is displayed</li> <li>Call up menu</li> <li>Perform calibration (→ <a href="#">Calibrating height measurement, page 69</a>)</li> </ul>
The display backlighting in the head slide display goes out and cannot be switched back on.	Battery voltage is dropping. Display backlighting has been switched off automatically to save energy. You can perform measurements and send data for approximately another 12 h.	→ <a href="#">Changing the batteries, page 83</a>
 is displayed.	Battery voltage is dropping.	→ <a href="#">Changing the batteries, page 83</a>
<b>bAtt</b> is displayed.	The batteries have discharged.	→ <a href="#">Changing the batteries, page 83</a>
<b>StOP</b> is displayed.	Maximum capacity has been exceeded.	Remove the load from the device.
<b>tEMP</b> is displayed.	The ambient temperature of the scale is too high or too low.	<ul style="list-style-type: none"> <li>Operate the device only in the intended ambient conditions (→ <a href="#">General technical data, page 85</a>).</li> <li>After transporting or storing the device, wait about 15 minutes before switching it on.</li> </ul>
Multifunctional display no longer reacts when keys are pressed.	Key lock is active.	Switch off key lock (→ <a href="#">Activating/deactivating key lock, page 70</a> ).
	Device is in an undefined state following implausible input.	<ul style="list-style-type: none"> <li>Take power supply unit out of the power supply socket.</li> <li>Wait approx. 1 minute.</li> <li>Plug the power supply unit into the power supply socket; the scale and the multifunctional display will switch on automatically.</li> </ul>
	The keypad is defective.	Inform seca Service.


## 10.2 Changing the batteries

✓ You need 4 AA-type batteries, 1.5 V.


1. Take off the cover of the battery compartment.
2. Remove the battery holder from the battery compartment.
3. Take the discharged batteries out of the battery holder.
4. Put the new batteries in the battery holder.
5. Insert the battery holder in the battery compartment.

### NOTE

- Dispose of empty batteries as described in the section entitled “Disposal”.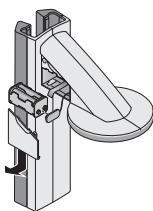

- Ensure the correct polarity of the batteries (markings on the battery holder). If **bAtt** is shown in the display, you have inserted one of the batteries the wrong way round or the batteries have discharged. If batteries are inserted the wrong way round, they must be removed immediately.

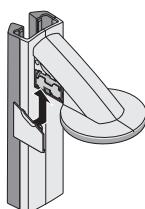**NOTICE!**

**Damage to device and malfunction**

If the battery compartment is not closed correctly, the connector cable may be damaged.

- When closing the battery compartment, ensure that the connector cable is not trapped between the cover and the battery compartment.

6. Push the cover onto the battery compartment until you hear it engage.

## 11 SERVICING

### 11.1 Weight measurement, verified models

seca recommends having your device serviced prior to verification.

**CAUTION!**

**Faulty measurements as a result of poor servicing**

- Have servicing and repairs carried out exclusively by seca Service or an authorized service partner.
- You can find a service partner in your vicinity at [www.seca.com](http://www.seca.com) or by emailing [service@seca.com](mailto:service@seca.com).

**CAUTION!**

**Faulty measurements as a result of verification being performed poorly or not at all**

- Have verification performed only by authorized persons.
- Always have verification performed if one or more quality seals are damaged.


Have authorized persons perform verification in line with national legal regulations.

Verification is necessary whenever one or more quality seals are damaged or the contents of the verification counter no longer match the number on the applicable verification sticker. If quality seals are damaged, contact seca Service directly.

Verifications may only be performed by authorized agencies. To guarantee this, the scale is equipped with a verification counter which records each change in verification-related data.

If you want to check whether the scale has been properly verified, proceed as follows:

1. Ensure that the device is switched off.
2. Press the key of the multifunctional display.  
⇒ All the elements of the display are shown briefly.
3. Press any other key on the multifunctional display (not the key again) whilst all the elements of the display are shown briefly.  
⇒ The symbol is displayed.  
⇒ The current content of the verification counter flashes in the display for a few seconds.
4. Compare the content of the verification counter displayed with the number on the verification counter sticker.

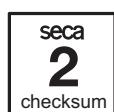Both numbers have to match for the verification to be valid. If the verification sticker and the verification counter do not match, the scale must be verified. Please contact your service partner or seca Service. Once the scale has been verified, a new, updated verification counter sticker is used to identify the verification counter reading. The person authorized to perform verification secures this verification with an additional seal. The verification counter sticker can be ordered from seca Service.

## 11.2 Weight measurement, non-verified models

The product must be set up carefully and serviced regularly. Depending on how frequently the scale is used, seca recommends servicing at intervals of 3 to 5 years.

### CAUTION!

#### Faulty measurements as a result of poor servicing

- ▶ Have servicing and repairs carried out exclusively by seca Service or an authorized service partner.
- ▶ You can find a service partner in your vicinity at [www.seca.com](http://www.seca.com) or by emailing [service@seca.com](mailto:service@seca.com).

## 11.3 Height measuring devices

The product needs to be set up carefully and serviced regularly. Depending on how frequently the product is used, seca recommends servicing at intervals of 3 to 5 years.

### CAUTION!

#### Faulty measurements as a result of poor servicing

- ▶ Have servicing and repairs carried out exclusively by seca Service or an authorized service partner.
- ▶ You can find a service partner in your vicinity at [www.seca.com](http://www.seca.com) or by emailing [service@seca.com](mailto:service@seca.com).

## 12 TECHNICAL DATA

### 12.1 General technical data

seca 285/seca 284	
Dimensions	
• Depth	466 mm
• Width	435 mm
• Height (standard model)	2418 mm
• Height (short model)	2200 mm
Weight of measuring station	
• Standard model	Approx. 16.5 kg
• Short model	Approx. 16.25 kg
Ambient conditions, operation	
• Temperature	+10 °C to +40 °C / +50 °F to 104 °F
• Air pressure	700 – 1060 hPa
• Humidity	30 % – 80 %, no condensation
Ambient conditions, storage	
• Temperature	-10 °C to +65 °C / +14 °F to 149 °F
• Air pressure	700 – 1060 hPa
• Humidity	0 % – 95 %, no condensation

seca 285/seca 284	
Ambient conditions, transport • Temperature • Air pressure • Humidity	-10 °C to +65 °C / +14 °F to 149 °F 700 – 1060 hPa 0 % – 95 %, no condensation
Power supply, head slide Batteries • Supply voltage • Battery type	6 V 4 x type AA, 1.5 V
Power supply, scale/multifunctional display Power supply unit • Supply voltage • Maximum current consumption • Insulated device, protection class II (EN 60601-1)	12 V Typically 500 mA <input checked="" type="checkbox"/>
Power supply voltage	100 V – 240 V
Power supply frequency	50 Hz – 60 Hz
Current consumption, scale/multifunctional display	Approx. 120 mA
Current consumption, head slide	Approx. 80 mA
Maximum running time, head slide	Approx. 2200 minutes
Medical device in line with directive (EU) 2017/745	Class I with measuring function
EN 60601-1: Medical electrical device, Type B	
Type of protection	IP20
Duty cycle	Continuous duty
Application parts in line with IEC 60601-1	Multifunctional display, glass plate, head slide

## 12.2 Weight measurement

Non-verified model	
Maximum capacity	300 kg
Minimum capacity	1 kg
Graduation	50 g
Tare range	300 kg (subtractive)
Accuracy • 0 kg to 33 kg • 33 kg to 300 kg at 20 °C ambient temperature and 45 % humidity	± 50 g ± 0,15 %

Verified model	
Verification in line with Directive 2014/31/EU	Class III
Maximum capacity • Weighing range 1 • Weighing range 2	150 kg 300 kg
Minimum capacity • Weighing range 1 • Weighing range 2	1 kg 2 kg
Graduation • Weighing range 1 • Weighing range 2	50 g 100 g
Tare range	300 kg (subtractive)

Verified model	
Accuracy on initial verification	
• Weighing range 1: 0 to 25 kg	± 25 g
• Weighing range 1: 25 to 100 kg	± 50 g
• Weighing range 1: 100 to 150 kg	± 75 g
• Weighing range 2: 0 to 50 kg	± 50 g
• Weighing range 2: 50 to 200 kg	± 100 g
• Weighing range 2: 200 to 300 kg	± 150 g

## 12.3 Height measurement

Measuring range, graduation, accuracy	
Measuring range, standard model	30 – 220 cm
Measuring range, short model	30 – 200 cm
Graduation	1 mm
Accuracy	± 2 mm

## 13 OPTIONAL ACCESSORIES AND SPARE PARTS

Accessory/spare part	Article number
Switch-mode power supply: 100 V – 240 V~ / 50 Hz – 60 Hz, 12 V= / 0.5 A	68 32 10 270
Accessories for integration via WiFi/LAN • <b>seca 452</b> external interface module • <b>seca connect 103</b> software • Barcode scanner (medical device)	452 0000 009  Will be provided in the course of integration projects Cannot be ordered from seca; for recommendations, see system instructions for use for <b>seca 103/452</b>
Spare parts kit, short column	19 17 01 079 509

## 14 DISPOSAL

### 14.1 Disposing of the device

Do not dispose of the device in your household waste. The device must be properly disposed of as electronic scrap. Follow your respective national regulations. For more information, please contact seca Service at [service@seca.com](mailto:service@seca.com).

### 14.2 Disposing of batteries and rechargeable batteries

Do not dispose of used batteries and rechargeable batteries in household waste, regardless of whether they contain harmful substances or not. As a consumer, you are legally obliged to dispose of batteries and rechargeable batteries via local authority collection points or trade collection points. Only dispose of batteries and rechargeable batteries once they are completely discharged.

## **15 WARRANTY**

There is a two-year warranty period from delivery for defects attributable to poor materials or workmanship. All movable parts, e.g. batteries, cables, power supply units, rechargeable batteries etc. are exempt. Defects which come under the warranty will be repaired for the customer free of charge against proof of purchase. Additional claims cannot be considered. Costs of transport to and from seca are the responsibility of the customer if the device is located somewhere other than the customer's headquarters. In the event of transport damage, claims under warranty can only be made if the complete original packaging was used for transport and the device was secured and fastened in it according to its originally packaged condition. You should therefore keep all packaging parts.

The warranty will be voided if the device is opened by persons not expressly authorized by seca to do so.

In the event of a warranty issue, please contact your local seca office or the dealer from whom you ordered the product.

## **16 DECLARATION OF CONFORMITY**

seca gmbh & co. kg hereby declares that the product complies with the terms of the applicable European directives and regulations. The unabridged declaration of conformity can be found at [www.seca.com](http://www.seca.com).

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