
seca 635/seca 634

Instructions for use

17-10-07-675-100_2025-08S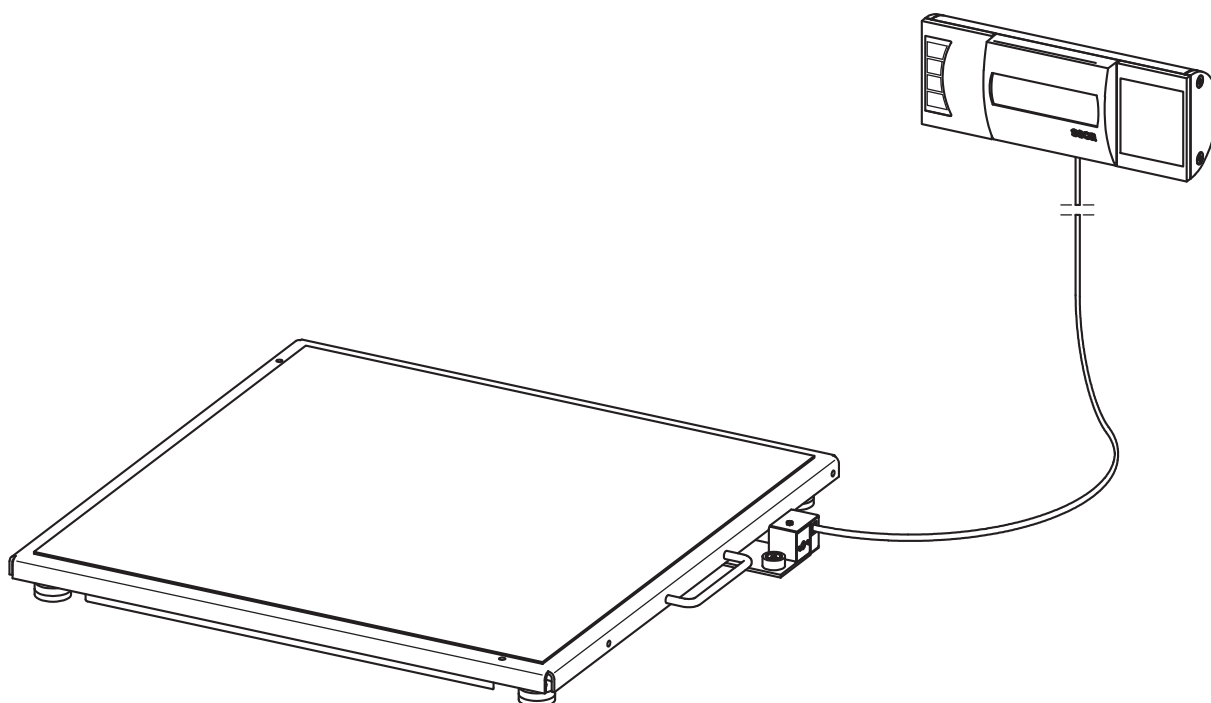

Deutsch – Gebrauchsanweisung	3
English – Instructions for use	32
Français – Mode d’emploi.....	61
Italiano – Istruzioni per l’uso	90
Español – Instrucciones de uso	119
Português – Manual de instruções de utilização	148
Polski – Instrukcja obsługi.....	177

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TABLE OF CONTENTS

1 About this document	33	5.4 Establishing power supply	47
1.1 Representation in text	33	Inserting batteries.....	47
1.2 Representation in diagrams	33	Connecting the power supply unit (optional).....	47
1.3 Devices in this document	33	5.5 Aligning the scale	48
Devices in this document	33	6 Operation.....	48
2 Description of device	34	6.1 Weighing.....	48
2.1 Intended use.....	34	Switching on the scale	48
2.2 Contraindications	34	Weighing a patient.....	49
2.3 Clinical benefit.....	34	Taring off additional weight (TARE)	49
2.4 Patient target group.....	34	Keep the measuring result on the display (HOLD).....	49
2.5 User qualification.....	34	Determining Body Mass Index	50
Assembly.....	34	Switching the weighing range (verified models).....	50
Operation.....	34	Switching off the scale	50
Administration/network operation	34	6.2 Other functions (menu).....	51
2.6 Functional description	34	Deleting saved values automatically (ACLR).....	51
Measuring weight	34	Saving additional weight permanently (Pt)	51
3 Safety information	34	Activating Autohold function (AHOLD).....	52
3.1 Safety information in these instructions for use.....	34	Activating/deactivating beeps (bEEP)	52
3.2 Basic safety information.....	35	Setting filtering (FIL).....	52
Handling device.....	35	Non-verified models: Switching the weight unit (Unit)	53
Preventing electric shock	36	Restoring factory settings (rESet)	53
Preventing injuries and infections.....	36	7 Hygiene treatment.....	54
Preventing damage to device.....	37	7.1 Cleaning	54
Handling measuring results.....	37	7.2 Disinfecting.....	54
Handling packaging material	38	7.3 Sterilizing.....	54
Handling batteries and rechargeable batteries.....	38	8 Function check	55
4 Overview	39	9 Troubleshooting.....	55
4.1 Controls	39	10 Servicing.....	56
4.2 Symbols in the display	40	10.1 Verified scales	56
4.3 Markings on the device and on the type plate.....	40	10.2 Non-verified scales.....	57
4.4 Menu structure	43	11 Technical data.....	57
4.5 Navigating in the menu.....	43	11.1 General technical data	57
5 Starting up device.....	44	11.2 Weight measurement	58
5.1 Scope of delivery.....	44	12 Optional accessories and spare parts.....	59
5.2 Preparing the display housing for operation (desktop display)	44	13 Disposal.....	59
Fitting the angle adapter	44	13.1 Disposing of the device.....	59
Connecting the connecting cable to the platform	45	13.2 Disposing of batteries and rechargeable batteries.....	59
5.3 Preparing the display housing for operation (wall installation)	45	14 Warranty	59
Mounting the wall bracket	45	15 Declaration of conformity	60
Fitting the angle adapter	46		
Putting the display housing on the wall bracket.....	46		
Connecting the connecting cable to the platform	46		

1 ABOUT THIS DOCUMENT

1.1 Representation in text

Symbol	Description
✓	Requirement for actions
►	Action
1. 2.	Actions with specified sequence
a) b)	Steps of an action with specified sequence
⇒	Result of an action
• •	First level of a list
– –	Second level of a list

1.2 Representation in diagrams

Symbol	Description
	Indicates directions of movement of the device or of device components
	Navigation path in menu displays
	• Correct action • Correct result of action
	• Incorrect action • Incorrect result of action
	End of a procedure, e.g. the installation of a part

1.3 Devices in this document

This document applies to the following devices:

- **seca 635:** Verified scale
- **seca 634:** Non-verified scale

2 DESCRIPTION OF DEVICE

2.1 Intended use

The multifunctional scale supports physicians in decision-making regarding diagnosis or therapy, based on weight or weight changes.

2.2 Contraindications

No contraindications are known.

2.3 Clinical benefit

The multifunctional scale supports physicians in decision-making regarding diagnosis or therapy, based on the measured weight.

By that, the multifunctional scale only provides an indirect clinical benefit.

The physician must perform additional examinations for an accurate diagnosis.

2.4 Patient target group

The scale is intended for persons of any age (with the exception of babies) who

- do not exceed the maximum weight capacity of the scale and
- are able to stand on the scale independently without help.

2.5 User qualification

Assembly	Devices shipped partly assembled may only be assembled by sufficiently qualified persons (such as specialist dealers, hospital technicians or seca Service technicians).
-----------------	--

Operation	The device may only be operated by persons with formal training in the healthcare sector or in medicine.
------------------	--

Administration/network operation	The device may only be set up and incorporated in a network by experienced administrators or hospital technicians.
---	--

2.6 Functional description

Weight calculation is performed by four load cells.

3 SAFETY INFORMATION

3.1 Safety information in these instructions for use

DANGER!

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries will occur.

**WARNING!**

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries may result.

**CAUTION!**

Used to identify a hazardous situation. If you fail to take note of this information, minor to moderate injuries may result.

NOTICE!

Used to identify possible incorrect usage of the device. If you fail to take note of this information, you may damage the device, or the measuring results may be incorrect.

NOTE

Includes additional information about use of the device.

3.2 Basic safety information

Handling device

- ▶ Please take note of the information in these instructions for use.
- ▶ Keep the instructions for use in a safe place. The instructions for use are a component of the device and must be available at all times.
- ▶ In the interest of patient safety, you and your patients are obliged to report serious events that occur in connection with this product to the manufacturer and to the authority responsible in your country.

**DANGER!****Risk of explosion**

- ▶ Do not use the device in an environment in which one of the following gases has accumulated:
 - oxygen
 - flammable anesthetics
 - other flammable substances/air mixtures

**CAUTION!****Patient hazard, damage to device**

- ▶ Additional devices connected to medical electrical devices must provide evidence of compliance with the relevant IEC or ISO standards (e.g. IEC 60950 for data-processing devices). Furthermore, all configurations must comply with the requirements of standards for medical systems (see IEC 60601-1-1 or Section 16 of edition 3.1 of IEC 60601-1 respectively). Anyone connecting additional devices to medical electrical devices is considered a system configurator and therefore responsible for ensuring that the system complies with the requirements of standards for systems. This also applies to additional devices recommended by seca. Your attention is drawn to the fact that local laws take precedence over the above-mentioned requirements of standards. In the event of any queries, please contact your local specialist dealer or Technical Service.

**CAUTION!****Patient hazard, damage to device**

- ▶ Have servicing carried out regularly as described in the relevant section of this document.
- ▶ Technical modifications may not be made to the device. The device does not contain any user-serviceable parts. Only have servicing and repairs performed by an authorized seca service partner. You can find a service partner in your vicinity at www.seca.com or by emailing service@seca.com.
- ▶ Use only seca original accessories and spare parts, otherwise seca will not grant any warranty.

**CAUTION!****Patient hazard, malfunction**

- ▶ Keep medical electrical devices such as high-frequency surgical equipment at a minimum distance of approx. 1 meter to prevent faulty measurements.
- ▶ Keep high-frequency equipment such as cellphones at a minimum distance of approx. 1 meter to prevent faulty measurements.
- ▶ The actual transmission output of HF devices may require minimum distances of more than 1 meter. For details, go to www.seca.com.

Preventing electric shock**WARNING!****Electric shock**

- ▶ Set up devices which can be operated with a power supply unit so that the power supply socket is within easy reach and the power supply can be disconnected quickly.
- ▶ Ensure that your local electricity supply matches the details on the power supply unit.
- ▶ Never touch the power supply unit with wet hands.
- ▶ Do not use extension cables and multiple outlets.
- ▶ Ensure that cables are not crushed or damaged by sharp edges.
- ▶ Ensure that cables do not come into contact with hot objects.
- ▶ Do not operate the device at an altitude of more than 3000 m above sea level.

Preventing injuries and infections**WARNING!****Injury from falling**

- ▶ Ensure that the device is steady and level.
- ▶ Route connector cables (if present) so that neither users nor the patient can trip over them.
- ▶ Assist people with limited motor skills when they are getting up, e.g. from a wheelchair.
- ▶ Ensure that the patient does not step directly onto or off the edges of the weighing platform.
- ▶ Ensure that the patient steps onto and off the weighing platform slowly and safely.

**WARNING!****Danger of slipping**

- ▶ Ensure that the weighing platform is dry before the patient steps onto it.
- ▶ Ensure that the patient has dry feet before stepping onto the weighing platform.
- ▶ Ensure that the patient steps onto and off the weighing platform slowly and safely.

**WARNING!****Risk of infection**

- ▶ Before and after every measurement, wash your hands to reduce the risk of cross-contamination and nosocomial infections.
- ▶ Should the patient have contagious diseases, subject the device to a hygiene treatment as described in the relevant section of these instructions for use immediately afterwards.
- ▶ Ensure that the patient does not have any open wounds or infectious skin alterations which may come into contact with the device.
- ▶ Subject the device to a hygiene treatment at regular intervals.

Preventing damage to device

NOTICE!

Damage to device

- ▶ Ensure that fluids never get inside the device. These can destroy the electronics.
- ▶ Switch off the device before you disconnect the power supply unit from the power supply socket.
- ▶ If the device is not to be used for an extended period, disconnect the power supply unit from the power supply socket. Only then is the device de-energized.
- ▶ If you are not using the device for an extended period of time, remove batteries or battery packs. Only then is the device de-energized.
- ▶ Do not drop the device.
- ▶ Do not subject the device to shocks or vibrations.
- ▶ Perform a function check before each use as described in the corresponding section in this document. Do not operate the device if it is not working properly or is damaged.
- ▶ Do not place the device in direct sunlight and ensure that it is not placed in the direct proximity of a heat source. The excessive temperatures could damage the electronics.
- ▶ Avoid rapid temperature fluctuations. If the device is transported so that a temperature difference of over 20 °C occurs, the device must be left to stand for at least 2 hours before it is switched on, otherwise condensation may form; this may damage the electronics.
- ▶ Use the device only in the intended ambient conditions.
- ▶ Do not use the device over 3000 m above mean sea level. Electronic components might become damaged.
- ▶ Store the device only in the intended storage conditions.
- ▶ Use only cleaning agents and disinfectants that match the details in the section entitled "Hygiene treatment".
- ▶ Ensure that maximum capacity is not exceeded.

Handling measuring results

CAUTION!

Patient hazard

To prevent misinterpretations, measuring results for medical purposes must only be displayed and used in SI units (kilograms/grams, meters/centimeters). Some devices have the option of displaying measuring results in different units. This is purely an additional function.

- ▶ Only use measuring results in SI units.
- ▶ The user takes sole responsibility for the use of measuring results in non-SI units.

NOTICE!

Inconsistent measuring results

- ▶ Before you save and continue using measured values determined using this device (e.g. in seca software or in an EMR system), ensure that the measured values are plausible.
- ▶ If measured values have been sent to seca software or to an EMR system, ensure before continuing to use them that the measured values are plausible and assigned to the correct patient.

Handling packaging material

WARNING!

Risk of suffocation

Packaging materials made of plastic film (bags) present a risk of suffocation.

- ▶ Store packaging material out of the reach of children.
- ▶ If the original packaging material is no longer available, only use plastic bags with safety holes to reduce the risk of suffocation. Use recyclable materials if possible.

NOTE

Store the original packaging material for future use (e.g. returning for servicing).

Handling batteries and rechargeable batteries

WARNING!

Injury

Batteries and rechargeable batteries contain harmful substances which may explode and so escape if not handled properly.

- ▶ Do not try to recharge batteries.
- ▶ Do not expose batteries/rechargeable batteries to heat.
- ▶ Do not burn batteries/rechargeable batteries.
- ▶ If harmful substances are leaking out, avoid contact with the skin, eyes, and mucous membranes. Rinse affected areas of the body with plenty of clean water and seek medical help at once.


NOTICE!

Damage to device and malfunction due to incorrect handling

- ▶ Use only the type of battery/rechargeable battery quoted in this document.
- ▶ Always replace all batteries/rechargeable batteries at the same time.
- ▶ Do not short-circuit batteries/rechargeable batteries.
- ▶ If you do not use the device for an extended period of time, remove the batteries/rechargeable batteries. This prevents acid from leaking into the device.
- ▶ If harmful substances have penetrated the device, do not continue using it. Have the device checked by an authorized seca service partner and repaired if necessary.


4 OVERVIEW

4.1 Controls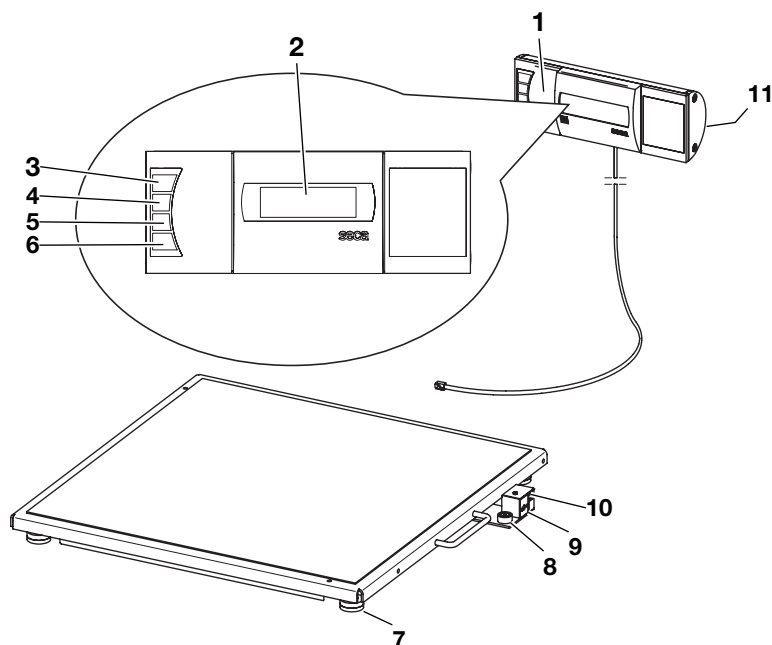

Item	Control	Function
1	Display housing	Central control and display element
2	Display	Display element for measuring results and to configure the device
3		Switches the scale on and off
4		During weighing: - Press briefly: Activate Hold function - Press and hold: Activate Tare function In the menu: - Select submenu, select menu item
5		During weighing: - Press briefly: Activate BMI function - Press and hold: Call up menu In the menu: - Select submenu, select menu item
6		In the menu: - Confirm selected menu item - Save set value
7	Foot screw	4 pcs, for precise alignment
8	Spirit level	Indicates whether the device is horizontal
9	Power supply connection	Serves to connect the optional power supply unit
10	Display connection	Serves to connect the display head
11	Battery compartment	Holder for batteries

4.2 Symbols in the display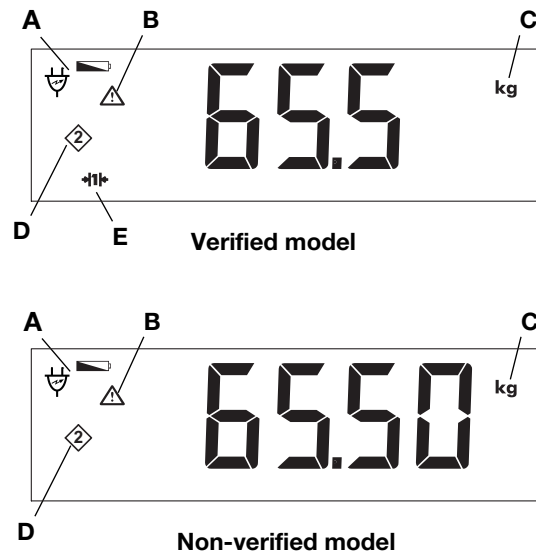


NOTE
In the following chapters of these instructions for use, the display of the verified model is shown, unless a distinction between “verified/non-verified” is required.

Item	Control	Function
A		Batteries are low
		Operation with power supply unit
B		Non-verifiable function active (for technical reasons, symbol is also displayed on non-verified scales)
C	kg lbs	Unit in which weight is displayed (lbs only on non-verified models)
D		Memory currently in use
E	→ 1 ← → 2 ←	Weighing range currently in use (only verified models)

4.3 Markings on the device and on the type plate


Markings on the device and on the type plate	
Symbol	Meaning
	Name and address of manufacturer, date of manufacture
UDI	Unique Device Identifier (product identification number)
REF	Article number

Markings on the device and on the type plate	
Symbol	Meaning
	Serial number
GAL	Value in m/s ² (model-dependent) • Gravitational acceleration on earth • Depends on the intended location
ProdID	Product identification number
Mat.No.	Variant number
Approval Type	Type designation of design approval
	Follow instructions for use
	Medical electrical device, Type B
IP20	Type of protection to IEC 60529: • Protection against ingress of solid foreign bodies with a diameter of over 12.5 mm • Protection against access with fingers • No protection from ingress of water
e	Value in units of mass (verified models) Used to classify and verify a scale
d	Value in units of mass (non-verified models) States the difference between two consecutive display values
	Active weighing range
	Class III scale to directive 2014/31/EU
	Device complies with EU directives • M: Conformity label in compliance with directive 2014/31/EU for non-automatic scales (verified models) • 24: (Example: 2024) Year in which declaration of conformity was implemented and the CE-label was applied (verified models) • 0102: Notified Body for Metrology (verified models) • 0123: Notified Body for Medical Devices
	Medical device in accordance with regulation (EU) 2017/745
	Device meets the requirements of the USA and Canada. Certified and tested by a licensing laboratory (NRTL) of TÜV SÜD Product Services GmbH.
	Device complies with United Kingdom directives • M: Conformity label in compliance with UK directive SI 2016 no. 1152 for non-automatic scales (NAWIR) (verified models) • 24: (Example: 2024) Year in which declaration of conformity was implemented and the UKCA label was applied (verified models) • xxxx: Notified Body for Medical Devices of the United Kingdom • yyyy: Notified Body for Metrology of the United Kingdom (verified models)
 	Importer/representative in the United Kingdom: seca Ltd 40 Barn Street B5 5QB Birmingham United Kingdom

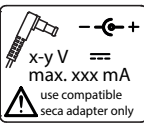Markings on the device and on the type plate	
Symbol	Meaning
	Importer/representative in Switzerland: seca ag (schweiz) Medizinische Waagen und Messsysteme Schön matt Str. 2 CH-4153 REINACH
	Type plate on the power supply connection socket • Power supply voltage required in V • Maximum current consumption in mA • : Note polarity of device connector • : Operate device with direct current • : Use compatible seca power supply units only
	Do not dispose of device in household waste

Markings on the packaging	
Symbol	Meaning
	Protect from moisture
	Arrows indicate top of product Transport and store in an upright position
	Fragile, do not throw or drop
	Permitted min. and max. temperature for transport and storage
	Permitted min. and max. humidity for transport and storage
	Permitted min. and max. air pressure for transport and storage
	Open packaging here
	Packaging material can be disposed of through recycling programs

4.4 Menu structure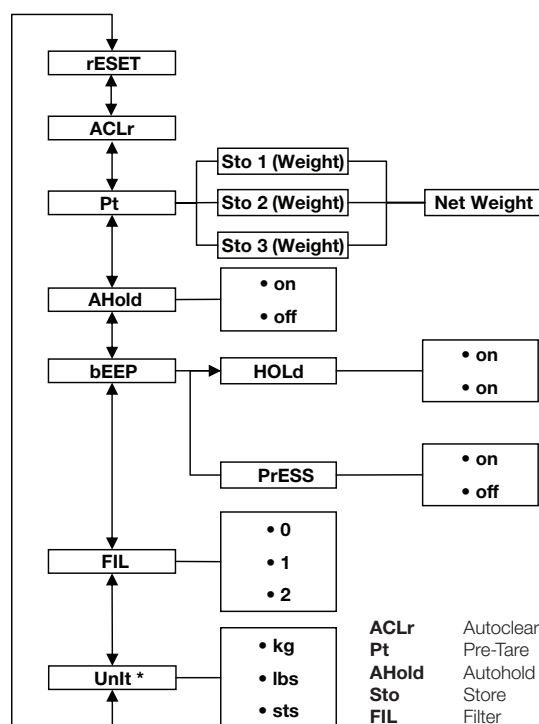

* Unverified scales

4.5 Navigating in the menu

AHOLD

FIL

FIL 0

FIL 2


1. Switch on the scale.
2. Hold down the arrow key ▼ until the menu is called up.
⇒ The last menu item selected is displayed (in this case: Autohold **AHOLD**).
3. Press one of the arrow keys until the desired menu item appears in the display (in this case: filtering **FIL**).
4. Confirm the selection by pressing the ↵ key.
⇒ The current setting for the menu item or a submenu is displayed (in this case: level **0**).
5. To change the setting, press the ▲ or the ▼ key until the desired setting is displayed (in this case: level **2**).
6. Confirm the selection by pressing the ↵ key.
⇒ The menu closes.
7. To make more settings, call up the menu again and proceed as described.

NOTE

If no key is pressed for approx. 24 seconds, the menu will close.

5 STARTING UP DEVICE

5.1 Scope of delivery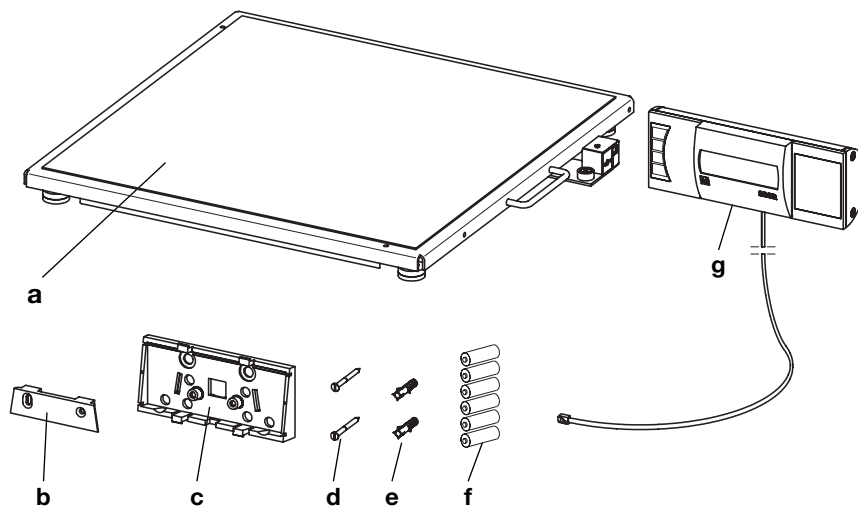

Item	Component	Pcs.
a	Weighing platform	1
b	Wall bracket	1
c	Angle adapter	1
d	Screw, 3 x 35 mm	2
e	Wall plug, Ø 5 mm	2
f	Battery, type AA/LR6, 1.5 V	6
g	Display housing with connecting cable	1
–	Instructions for use, not shown	1

5.2 Preparing the display housing for operation (desktop display)


NOTICE!

Damage to device

The angle adapter cannot be removed after installation.


- Ascertain, **before** installation, that the display housing is to be placed on a table.

Fitting the angle adapter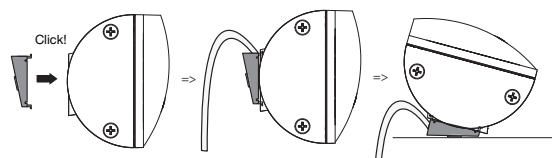

1. Attach the angle adapter to the display housing in the position shown.
2. Make sure that the connecting cable runs in the angle adapter's guide groove.
3. Press the angle adapter onto the display housing until you hear all four lugs click into place.

Connecting the connecting cable to the platform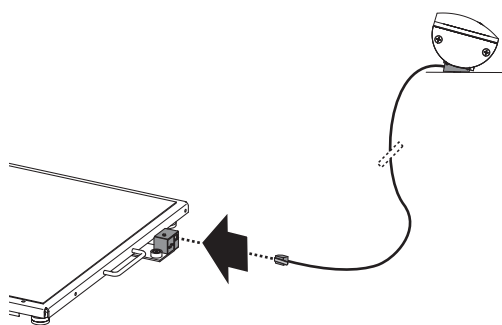

NOTICE!

Faulty measurement as a result of force shunt

If the display housing cable is touching the weighing platform, the weight of the patient cannot be measured correctly.

- Route the cable so that it cannot touch the weighing platform.

NOTE

Note the length of the connecting cable when selecting the setup location.

1. Place the scale on a solid, level surface.
2. Connect the connecting cable to the scale display connection.
3. Place the display housing on a work surface.

5.3 Preparing the display housing for operation (wall installation)

NOTICE!


Damage to device

The angle adapter cannot be removed after installation.

- Ascertain, **before** installation, that the cabled remote display is to be mounted on a wall.

Mounting the wall bracket

Choose an adequately strong, load-bearing wall to mount the device. Standard rawl plugs are included in the scope of delivery for mounting on solid brick walls. We recommend the use of appropriate special wall plugs for other types of walls.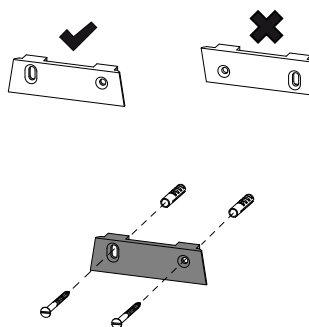


NOTE

Note the length of the connecting cable when selecting the mounting height.


1. Mark the drill holes horizontally to each other at the required height.
2. Drill the holes using a Ø 5 mm drill bit.
3. Insert the wall plugs in the drill holes.
4. Screw the wall bracket to the wall in the orientation shown.

Fitting the angle adapter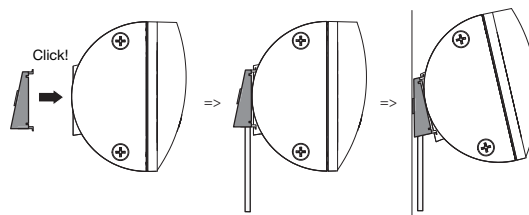

1. Attach the angle adapter to the display housing in the position shown.
2. Make sure that the connecting cable runs in the angle adapter's guide groove.
3. Press the angle adapter onto the display housing until you hear all four lugs click into place.


Putting the display housing on the wall bracket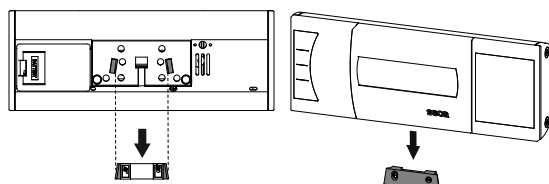

Rear View (Schematic diagram)

- Put the display housing on the wall bracket as shown in the drawing above.

Connecting the connecting cable to the platform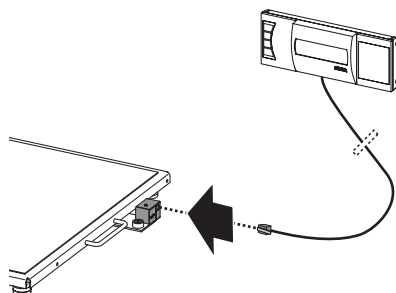

NOTICE!

Faulty measurement as a result of force shunt

If the display housing cable is touching the weighing platform, the weight of the patient cannot be measured correctly.

- Route the cable so that it cannot touch the weighing platform.


NOTE

Note the length of the connecting cable when selecting the setup location.

1. Place the scale on a solid, level surface.
2. Connect the connecting cable to the scale display connection.

5.4 Establishing power supply

Inserting batteries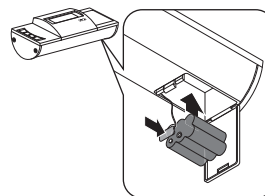

✓ 6 Mignon batteries, type AA, 1.5 V (included in scope of delivery)

1. Press on the latch of the battery compartment.
2. Carefully fold down the lid of the battery compartment.
3. Remove the battery holder from the battery compartment.
4. Put the batteries in the holder.

NOTE

Ensure the correct polarity of the batteries (markings in the battery holder). If **bAtt** appears in the display, you have inserted one of the batteries the wrong way round or the batteries have discharged.

5. Put the battery holder back in the battery compartment.
6. Close the battery compartment.

Connecting the power supply unit (optional)


WARNING!


Personal injury or damage to the device as a result of incorrect power supply units

Conventional power supply units may deliver a higher voltage than is indicated on them. The measuring device may overheat, catch fire, melt or short-circuit.

- Use only original seca power supply units with a controlled 12 V output voltage.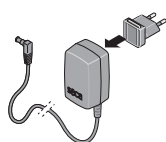

1. If required, insert the power supply connector necessary for your power supply into the power supply unit.

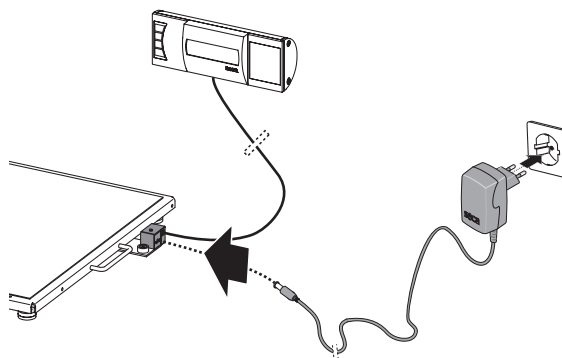2. Insert the device connector of the power supply unit into the connecting socket of the scale.
3. Plug the power supply unit into a power supply socket.

5.5 Aligning the scale

CAUTION!

Faulty measurement as a result of devices being set up incorrectly

Soft floors (wooden boards, for example) give under the patient's weight and falsify the measuring result. Incorrect ambient conditions or lack of alignment lead to measurement errors.

- ▶ Select a setup location with a level, stable floor in order to achieve precise measuring results.
- ▶ Use the device only in the intended ambient conditions outlined in the section entitled "Technical data".
- ▶ Align the device before using it and every time it changes location.


CAUTION!

Faulty measurement as a result of devices being set up incorrectly

If the weighing platform or attachments (e.g. ramps, handrails, rail parts) come into contact with walls or objects, the measuring result will be falsified.

- ▶ Locate the device at a sufficient distance from walls and objects.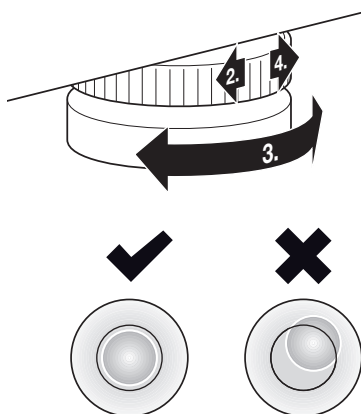

1. Place the scale on a firm, level surface.

2. Undo the knurled wheels.

3. Align the device by turning the foot screws.

The air bubble in the spirit level must be located in the exact center of the circle.

4. Tighten the knurled wheels in the direction of the arrow.

⇒ The foot screws are secured against being adjusted.

6 OPERATION

6.1 Weighing

CAUTION!

Personal injury/faulty measurement

- ▶ Before using the device each time, perform a function check as described in the corresponding section of these instructions for use.

Switching on the scale

- ▶ Press the  key.
 - ⇒ All the elements of the display are shown briefly, and then **SECA** appears in the display.
 - ⇒ The scale is operational when **0.0** (non-verified model: **0.00**) appears in the display.

Weighing a patient

✓ The weighing platform has no load.

1. Ask the patient to step onto the weighing platform.
2. Ask the patient not to move.
3. Read off the measuring result.

**Taring off additional weight (TARE)**

Use the **TARE** function to prevent an additional weight affecting the patient's weight value.

NOTICE!**Faulty measurement as a result of force shunt**

If an additional weight is touching the surface on which the scale is standing, the weight will not be measured correctly.

- Make sure that additional weights are only placed on the scale's weighing platform.

1. Switch on the scale.
2. Place the additional weight on the scale.
3. Hold down the **▲ hold/tare** arrow key until the **NET** message appears in the display.
4. Wait until the display stops flashing and **0.0** appears instead.
⇒ The additional weight has been tared off.
5. Ask the patient to step onto the weighing platform.
6. Ask the patient not to move.
⇒ The patient's weight is displayed.
7. Read off the measuring result.
8. To deactivate the **TARE** function, press the **▲ hold/tare** arrow key until the **NET** message is no longer displayed or turn off the scale.

**NOTE**

The maximum weight which can be displayed is reduced by the weight of the objects which have been tared.

Keep the measuring result on the display (HOLD)

When the **Hold** function is activated, the weight is still displayed after the load has been removed from the scale. This enables you to attend to the patient before recording the weight value.


✓ The scale has no load.

1. Switch on the scale.
2. Weigh the patient as described in the section entitled → [Weighing a patient, page 49](#).
3. Press the **▲ hold/tare** arrow key briefly.
4. Wait until the weight value has stopped flashing.
⇒ The **⚠** symbol (non-verifiable function) and the **HOLD** message are displayed.
⇒ The patient can leave the weighing platform and the weight value will continue to be displayed.
5. To deactivate the function, press the **▲ hold/tare** arrow key briefly.
⇒ The **⚠** symbol and the **HOLD** message are no longer displayed.
⇒ The weight value is no longer displayed.

**NOTE**

If the autohold function is activated, the weight value is automatically displayed permanently until the scale switches itself off or is switched off, see → [Activating Autohold function \(AHold\), page 52](#).

Determining Body Mass Index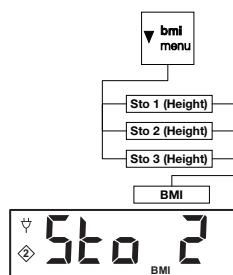

Body Mass Index correlates height and weight. To do this, the patient's height must be entered manually. The device calculates the BMI from the entered value and the measured weight.

You can save heights for a maximum of three patients so that the values are quickly available for repeat measurements.

✓ The scale has no load.

1. Switch on the scale.
2. Press the ▼ **bmi/menu** arrow key briefly.
 - ⇒ The message **BMI** appears.
 - ⇒ The last memory used is displayed (in this case, memory 2).
3. You have the following options for continuing:
 - ▶ Adopt memory
 - ▶ Select different memory: Press the ▲ or the ▼ key
4. Confirm your setting by pressing the ↵ key.
 - ⇒ Arrows flash in the display.
 - ⇒ The last height saved to the selected memory is displayed.
5. You have the following options for continuing:
 - ▶ Adopt value
 - ▶ Change value: Press the ▲ or the ▼ key
6. Confirm your setting by pressing the ↵ key.
 - ⇒ The height entered is saved and can be called up again for calculating a new BMI.

NOTE


Make a note of the memory and assigned height so that you can call up the value again for calculating a new BMI.

7. Weigh the patient as described in the section entitled "Weighing a patient".
 - ⇒ The patient's BMI is automatically calculated and displayed.
8. To deactivate the BMI function, briefly press the ↵ key.

The scale has two weighing ranges. In weighing range 1 (→|1|←), graduations in the weight display are finer. In weighing range 2 (→|2|←), you can use the maximum capacity of the scale.

After the scale is switched on, weighing range 1 is active.

Switching the weighing range (verified models)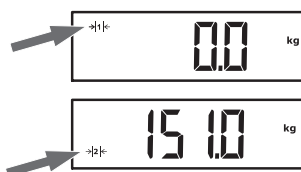

If a particular weight value is exceeded, the scale automatically switches to weighing range 2.

- ▶ To switch back to weighing range 1, completely remove the weight from the scale.
 - ⇒ Weighing range 1 is active again.
- ▶ Press the ⏻ key.

Switching off the scale

NOTE

For battery pack/battery mode only: The device switches off automatically after a short time if there is no load on the scale.

6.2 Other functions (menu)

Deleting saved values automatically (ACLR)

Out-of-date measuring results lead to incorrect calculation of BMI. You can set the device so that the measuring results are deleted automatically after 5 minutes.

NOTE

This function is activated at the factory on some models. The function can be deactivated.


1. From the menu, select the **ACLR** menu item.
2. Confirm your selection.
⇒ The current setting is displayed.
3. Select the desired setting:
 - **On**
 - **OFF**
4. Confirm your selection.
⇒ The menu closes.

Saving additional weight permanently (Pt)

Use the **Pre-Tare** function to save an additional weight permanently and have it deducted from a measuring result automatically. For example, you can save a flat-rate figure for shoes and clothing and then always deduct this from the measuring result if a patient is weighed fully clothed.

The device has three memories for weight values. You can save various weight values and call them up individually depending on the initial situation, so that they are automatically deducted from the measuring result.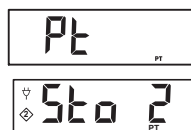

1. From the menu, select the **Pt** menu item.
2. Confirm your selection.
⇒ The last memory used is displayed.
3. You have the following options for continuing:
 - ▶ Adopt memory
 - ▶ Select different memory: Press the ▲ or the ▼ key
4. Confirm your selection.
⇒ Arrows flash in the display.
⇒ The additional weight saved to the selected memory is displayed.
5. You have the following options for continuing:
 - ▶ Adopt value
 - ▶ Change value: Press the ▲ or the ▼ key

NOTE

When you enter the value **0**, the function is switched off. The message **Pt** is no longer shown in the display.

6. Confirm your selection.
7. Ask the patient to step onto the scale.
⇒ The patient's weight is displayed.
⇒ The saved additional weight has been deducted automatically.
8. To deactivate the function, select the **Pt** menu item from the menu again.
9. Confirm your selection.
⇒ The function is deactivated.
⇒ The menu closes.

NOTE

When you switch the scale off, the function is switched off. The message **Pt** is no longer displayed when you switch the scale on again.

Activating Autohold function (AHOLD)

When you activate the **Autohold** function, the measuring result is displayed for every weighing procedure after the weight has been removed from the scale. It is then no longer necessary to activate the **Hold** function manually.

NOTE

This function is activated at the factory on some models. The function can be deactivated.

AHOLD

1. From the menu, select the **AHOLD** menu item.
2. Confirm your selection.
⇒ The current setting is displayed.
3. Select the desired setting:
 - **On**
 - **OFF**
4. Confirm your selection.
⇒ The setting is adopted.
⇒ The menu closes.

Activating/deactivating beeps (bEEP)

bEEP

PrESS

You can activate beeps for a number of device functions:

1. From the menu, select the **bEEP** menu item.
2. Confirm your selection.
3. Select a menu item:
 - **PrESS**: beep when key is pressed
 - **HOLD**: beep when weighing value is stable
4. Confirm your selection.
⇒ The current setting is displayed.
5. Select the desired setting:
 - **On**
 - **OFF**
6. Confirm your selection.
⇒ The setting is adopted.
⇒ The menu closes.

Setting filtering (FIL)

Setting the filter (**FIL** = filter) allows you to reduce interference when determining weight. The selected setting affects the sensitivity with which the weight display reacts to patient movements and the time until the **Hold** function displays a weight value permanently (**Hold** value).

Setting (FIL)	Sensitivity	Determination of Hold value
0	Sensitive	Slow
1	Medium	Medium
2	Insensitive	Fast

NOTE

With the **0** setting and patients who are not very steady on their feet, it is possible that no weight value is displayed permanently despite the **Hold** function being activated.

The greatest deviation between weight value displayed and actual value occurs at setting **2**.

FIL

1. From the menu, select the **FIL** menu item.
2. Confirm your selection.
⇒ The current setting is displayed.
3. Select a filtering level.
4. Confirm your selection.
⇒ The setting is adopted.
⇒ The menu closes.

Non-verified models: Switching the weight unit (Unit)


You can select the unit in which weight is to be displayed.

CAUTION! Patient hazard

To prevent misinterpretations, measuring results for medical purposes must only be displayed and used in SI units (kilograms/grams, meters/centimeters). Some devices have the option of displaying measuring results in different units. This is purely an additional function.

- ▶ Only use the measuring results in SI units.
- ▶ You as the user take sole responsibility for the use of measuring results in non-SI units.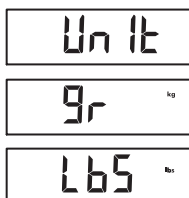

1. From the menu, select the **Unit** menu item.
2. Confirm your selection.
⇒ The current setting is displayed.
3. Select the unit in which weight is to be displayed:
 - Kilograms/grams (**gr**)
 - Pounds (**lbs**)
 - Stones (**sts**)
4. Confirm your selection.
⇒ The setting is adopted.
⇒ The menu closes.

Restoring factory settings (rESet)

You can restore the factory settings for the functions below:

Function	Factory setting
Autohold (AHOLd)	model-depen- dent
Acoustic signal (PrESS)	OFF
Acoustic signal (HOLd)	On
Filtering (FIL)	0
Autoclear (ACLr)	model-depen- dent
Pre-tare (Pt)	0 kg
Height for Body Mass Index (bMi)	170 cm
Unit, weight (Unit)	kg (gr)

1. From the menu, select the **rESet** menu item.
2. Confirm your selection.
⇒ The menu closes.
3. Switch off the scale.
⇒ Factory settings are restored and are available when the scale is switched on.

7 HYGIENE TREATMENT

WARNING! **Electric shock**

The device is not de-energized when the on/off button is pressed and the display goes out. Use of fluids on the device may cause an electric shock.

- ▶ Before each hygiene treatment, ensure that the device is switched off.
- ▶ Disconnect the power supply connector before each hygiene treatment.
- ▶ Take the batteries out of the device before every hygiene treatment.
- ▶ Ensure that no fluids penetrate the device.

WARNING! **Risk of infection**

- ▶ Subject the device to a hygiene treatment at regular intervals as described in this section.

NOTICE!

Damage to device

Unsuitable cleaning agents and disinfectants may damage the sensitive surfaces of the device and impair its operability.

- ▶ Do not use aggressive or abrasive cleaning agents.
- ▶ Do not use organic solvents (e.g. white spirit or petroleum spirit).

7.1 Cleaning

- ▶ If required, moisten a soft cloth with a mild soap solution and wipe the device over with it.

7.2 Disinfecting

1. Disinfect the device at regular intervals with a disinfectant suitable for sensitive surfaces and acrylic glass (e.g. 70 % ethanol).
2. Follow the instructions for use of the disinfectant.
3. Disinfect the device:
 - ▶ Moisten a soft cloth with disinfectant and wipe down the device with it.
 - ▶ Comply with the intervals, see table:

Interval	Component
Before and after each measurement	Weighing platform
As required	Display, keypad

7.3 Sterilizing

The device must not be sterilized.

8 FUNCTION CHECK

- Perform a function check before each use.

A complete function check includes:

- A visual inspection for mechanical damage
- A test of the alignment of the device
- A visual and function check of the display elements
- A function check of all the controls shown in the section entitled “Overview”
- A function check of the optional accessories

If you find faults or deviations during the function check, first try to remedy the fault with the aid of the “Troubleshooting” section in this document.

CAUTION!

Personal injury

If you find faults or deviations during the function check which you are unable to remedy with the aid of the “Troubleshooting” section in this document, you must not use the device.

- Have the device repaired by seca Service or an authorized service partner.
- Follow the section entitled “Servicing” in this document.

9 TROUBLESHOOTING

If faults occur when operating the device, first attempt to remedy them yourself using the following table. If the fault persists, contact seca Service.

Fault	Cause	Remedy
No weight is displayed with a load on the scale.	The device has no power supply.	• Switch on device. • In battery mode: Insert batteries. • In power supply operation: Connect the power supply unit.
 is displayed.	Battery voltage is dropping.	Insert new batteries as soon as possible
bAtt is displayed.	The batteries have discharged.	Insert new batteries
0.0 (verified models)/ 0.00 (non-verified models) is not displayed before weighing takes place.	A load was already on the device before it was switched on.	• Remove the load from the device. • Switch off the device, then switch it back on again.
----- is displayed before weighing takes place.	A load was already on the device before it was switched on.	• Remove the load from the device. • Switch off the device, then switch it back on again.
The weight display flashes.	Consistent weight value not yet detected.	• One of the following functions is active: Hold, Autohold, Tare. Wait until the device has detected a consistent weight value. The display stops flashing. • Take the load off the device, wait until 0.0/0.00 is displayed and measure again.
Implausible measuring results.	Patient moved too much during the measurement.	Take the load off the device, wait until 0.0/0.00 is displayed and repeat the measurement procedure.
One display segment is displayed continuously or not at all.	The display is defective.	Inform seca Service.

Fault	Cause	Remedy
StOP is displayed.	Maximum capacity has been exceeded.	Remove the load from the device.
tEMP is displayed.	The ambient temperature of the scale is too high or too low.	- Operate the device only in the intended ambient conditions (General technical data). - After transporting or storing the device, wait approx. 15 minutes before switching it on.
The device no longer reacts when keys are pressed.	Device is in an undefined state following implausible input.	- Take power supply unit out of the power supply socket. - Wait approx. 1 minute. - Plug the power supply unit into the power supply socket; the scale and the multifunctional display will switch on automatically.
	The keypad is defective.	Inform seca Service.

10 SERVICING

10.1 Verified scales

CAUTION!

Faulty measurements as a result of verification being performed poorly or not at all

- Have verification performed only by authorized persons.
- Always have verification performed if one or more quality seals are damaged.

seca recommends having your device serviced prior to verification.

CAUTION!

Faulty measurements as a result of poor servicing

- Have servicing and repairs carried out exclusively by seca Service or an authorized service partner.
- You can find a service partner in your vicinity at www.seca.com or by emailing service@seca.com.


Have authorized persons perform verification in line with national legal regulations.

Verification is necessary whenever one or more quality seals are damaged or the contents of the verification counter no longer match the number on the applicable verification sticker. If quality seals are damaged, contact seca Service directly.

Verifications may only be performed by authorized agencies. To guarantee this, the scale is equipped with a verification counter which records each change in verification-related data.

If you want to check whether the scale has been properly verified, proceed as follows:

1. Ensure that the device is switched off.
2. Press and hold any key and start the scale.
 - ⇒ The current content of the verification counter flashes in the display for a few seconds.
3. Compare the content of the verification counter output with the number given on the verification counter seal.

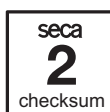Both numbers have to match for the verification to be valid. If the verification sticker and the verification counter do not match, the scale must be verified. Please contact your service partner or seca Service. Once the scale has been verified, a new, updated verification counter sticker is used to identify the verification counter reading. The person authorized to perform verification secures this verification with an additional seal. The verification counter sticker can be ordered from seca Service.

10.2 Non-verified scales

The product needs to be set up carefully and serviced regularly. Depending on how frequently the product is used, seca recommends servicing at intervals of 3 to 5 years.

CAUTION!

Faulty measurements as a result of poor servicing

- Have servicing and repairs carried out exclusively by seca Service or an authorized service partner.
- You can find your local service partner at www.seca.com.

11 TECHNICAL DATA

11.1 General technical data

Parameter	Value
Dimensions, total • Depth • Width • Height	600 mm 640 mm 57 mm
Dimensions, weighing platform • Depth • Width • Height	600 mm 600 mm 57 mm
Weight	approx. 12 kg
Height of digits, display	25 mm
Ambient conditions, operation • Temperature • Air pressure • Humidity	+10 °C to +40 °C / +50 °F to 104 °F 700 – 1060 hPa 30 % – 80 % no condensation
Ambient conditions, storage/transport • Temperature • Air pressure • Humidity	-10 °C to +65 °C / +14 °F to 149 °F 700 – 1060 hPa 0 % – 95 % no condensation
Medical device in accordance with regulation (EU) 2017/745	Class I with measuring function
IEC 60601-1: Medical electrical device, Type B	
Type of protection in accordance with IEC 60529	IP20
Duty cycle	Continuous duty
Service life, from commissioning ^a	at least 10 years
Application parts according to IEC 60601-1	Weighing platform, display housing, film keypad, display

Parameter	Value
Batteries • Supply voltage • Battery type • Maximum running time	9 V 6x type AA, 1.5 V approx. 4800 weighing operations
Power supply unit, optional • Supply voltage • Current consumption • Insulated device, protection class II (IEC 60601-1) • Mains voltage • Power supply frequency	12 V approx. 25 mA  100 V – 240 V 50 Hz – 60 Hz

^a Minimum service life when used as intended and serviced in a timely manner. seca products usually have a significantly longer service life.

11.2 Weight measurement

Verified model	
Verification in line with directive 2014/31/EU	Class III
Maximum capacity • Weighing range 1 • Weighing range 2	200 kg 300 kg
Minimum capacity • Weighing range 1 • Weighing range 2	2 kg 4 kg
Graduation • Weighing range 1 • Weighing range 2	100 g 200 g
Tare range	300 kg (subtractive)
Accuracy on initial verification • Weighing range 1: < 50 kg • Weighing range 1: 50 kg to 200 kg • Weighing range 2: < 100 kg • Weighing range 2: 100 kg to 300 kg	± 50 g ± 100 g ± 100 g ± 200 g

Non-verified model	
Maximum capacity	360 kg/800 lb/57 st
Minimum capacity	1 kg
Graduation	50 g/0.1 lb
Tare range	360 kg/800 lb/57 st (subtractive)
Accuracy • < 67 kg • 67 kg to 360 kg • < 147.7 lb • 147.7 lb to 800 lb • < 10.55 st • 10.55 st to 57 st At ambient temperature of 20° C and humidity of 45 %	± 100 g ± 0.15 % ± 0.2 lb ± 0.15 % ± 0.2 lb ± 0.15 %

17-10-07-675-100_2025-06S

12 OPTIONAL ACCESSORIES AND SPARE PARTS

Accessory/spare part	Article number
Power supply unit with Euro connector: 230 V~ /50 Hz/ 12 V= / 130 mA	68 32 10 252
Switch-mode power supply unit with adapters: • Input: 100 V – 240 V~, 50 Hz – 60 Hz • Output: 12 V= / 0.98 A	68 32 10 265 009
Switch-mode power supply unit with adapters: • Input: 100 V – 240 V~, 50 Hz – 60 Hz • Output: 12 V= / 0.6 A	68 32 10 270 009

13 DISPOSAL

13.1 Disposing of the device

Do not dispose of the device in your household waste. The device must be properly disposed of as electronic scrap. Follow your respective national regulations. For more information, please contact seca Service or an authorized service partner.

13.2 Disposing of batteries and rechargeable batteries

Do not dispose of used batteries and rechargeable batteries in household waste, regardless of whether they contain harmful substances or not. As a consumer, you are legally obliged to dispose of batteries and rechargeable batteries via local authority collection points or trade collection points. Only dispose of batteries and rechargeable batteries once they are completely discharged.

14 WARRANTY

There is a two-year warranty period from delivery for defects attributable to poor materials or workmanship. All movable parts, e.g. batteries, cables, power supply units, rechargeable batteries etc. are exempt. Defects which come under the warranty will be repaired for the customer free of charge against proof of purchase. Additional claims cannot be considered. Costs of transport to and from seca are the responsibility of the customer if the device is located somewhere other than the customer's headquarters. In the event of transport damage, claims under warranty can only be made if the complete original packaging was used for transport and the device was secured and fastened in it according to its originally packaged condition. You should therefore keep all packaging parts.

The warranty will be voided if the device is opened by persons not expressly authorized by seca to do so.

In the event of a warranty issue, please contact your local seca office or the dealer from whom you ordered the product.

15 DECLARATION OF CONFORMITY

seca gmbh & co. kg hereby declares that the product complies with the terms of the applicable European directives and regulations. The unabridged declaration of conformity can be found at www.seca.com.

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