
seca 757/727

Instructions for Use

17-10-07-703-100_2026-07S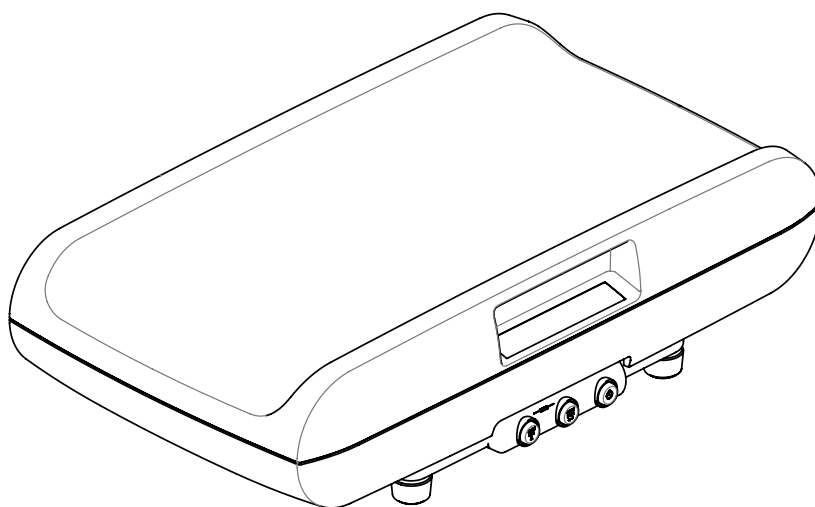

Deutsch – Gebrauchsanweisung	3
English – Instructions for Use.....	30
Français – Mode d’emploi.....	56
Italiano – Istruzioni per l’uso	82
Español – Instrucciones de uso	108
Português – Manual de instruções de utilização	135
Polski – Instrukcja obsługi.....	162

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CONTENTS

1 About this document.....	31	6 Operation.....	44
1.1 Representation in text.....	31	6.1 Switching the device on and off.....	44
1.2 Representation in diagrams	31	Switching on the device.....	44
2 Description of device	31	Switching off the device.....	44
2.1 Intended use.....	31	6.2 Measuring weight	44
2.2 Contraindications.....	31	Weighing	44
2.3 Clinical benefit	32	Taring additional weight (tare)	45
2.4 Patient target group.....	32	Displaying measuring result permanently (hold)	45
2.5 User qualification	32	Automatic switchover of weighing range (seca 757 only)	46
Operation	32	6.3 Other functions (menu)	46
Administration (devices with interface module only)	32	Activating/deactivating the Autohold function (AHOLd)	46
2.6 Functional description.....	32	Activating/deactivating acoustic signals (bEEP)	46
Measuring weight	32	Setting attenuation (FIL)	47
External interface module (optional accessory)	32	Setting display backlighting (LCd)	47
seca connect 103 software (optional accessory)	32	Switching the weight unit (seca 727 only)... ..	48
3 Safety precautions.....	32	Restoring factory settings (rESet).....	48
3.1 Safety information in these instructions for use	32	7 Hygiene treatment	49
3.2 Basic safety information.....	33	7.1 Cleaning.....	49
Handling device	33	7.2 Disinfecting.....	49
Preventing electric shock	34	7.3 Sterilizing	49
Preventing injuries and infections	34	8 Function check	50
Preventing damage to device.....	35	9 Troubleshooting.....	50
Handling measuring results.....	35	9.1 General faults	50
Handling packaging material	35	10 Servicing	51
Handling batteries and rechargeable batteries	36	10.1 Verified scales	51
4 Overview	36	10.2 Non-verified scales	52
4.1 Scale	36	11 Technical data.....	52
4.2 Symbols in the display	37	11.1 General technical data.....	52
4.3 Menu	38	11.2 Weight measurement	53
Navigating in the menu	38	11.3 Height measurement	53
Menu structure	38	12 Optional accessories and spare parts.....	54
4.4 Labels.....	38	13 Disposal	54
5 Starting up device.....	41	13.1 Disposing of the device	54
5.1 Scope of delivery	41	13.2 Disposing of batteries and rechargeable batteries	54
5.2 Transporting the device	41	14 Warranty	55
5.3 Removing the transport lock.....	42	15 Declaration of conformity	55
5.4 Attaching the height measuring tape.....	42		
5.5 Setting up and aligning the device	42		
5.6 Establishing power supply	43		
Inserting the battery pack	43		
Connecting the power supply unit.....	43		
5.7 Setting up the data connection (optional).....	43		

1 ABOUT THIS DOCUMENT

1.1 Representation in text

Symbol	Description
✓	Requirement for actions
►	Action
1. 2.	Actions with specified sequence
a) b)	Steps of an action with specified sequence
⇒	Result of an action
• •	First level of a list
– –	Second level of a list

1.2 Representation in diagrams

Symbol	Description
	Indicates relevant points on the device or on device components
	Indicates directions of movement of the device or of device components
	Navigation path in menu displays

2 DESCRIPTION OF DEVICE

2.1 Intended use

The electronic baby scale supports physicians in decision-making regarding diagnosis or therapy, based on the patient's weight or weight changes.

2.2 Contraindications

No contraindications are known.

2.3 Clinical benefit

The electronic baby scale supports physicians in decision-making regarding diagnosis or therapy, based on the measured weight. With that, the electronic baby scale only provides an indirect clinical benefit.

The physician must perform additional examinations for an accurate diagnosis.

2.4 Patient target group

The device is intended for babies, toddlers and children who do not exceed the maximum capacity of the scale and fit on the tray.

2.5 User qualification

Operation	The device may only be operated by persons with formal training in the healthcare sector or in medicine.
Administration (devices with interface module only)	The device may only be set up and incorporated in a network by experienced administrators or hospital technicians.

2.6 Functional description

Measuring weight	Weight calculation is performed by a single load cell.
External interface module (optional accessory)	Data can be sent to the seca connect 103 software via a WiFi or LAN connection with the aid of the seca 452 external interface module (optional).
seca connect 103 software (optional accessory)	The seca connect 103 software obtains measurement data from an internal or external interface module and sends them to compatible seca measuring devices or to third-party EMR systems.

3 SAFETY PRECAUTIONS

3.1 Safety information in these instructions for use

DANGER!

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries will occur.

WARNING!

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries may result.

CAUTION!

Used to identify a hazardous situation. If you fail to take note of this information, minor to moderate injuries may result.

NOTICE!

Used to identify possible incorrect usage of the device. If you fail to take note of this information, you may damage the device, or the measuring results may be incorrect.

NOTE

Includes additional information about use of the device.

3.2 Basic safety information

Handling device

- ▶ Please take note of the information in these instructions for use.
- ▶ Keep the instructions for use in a safe place. The instructions for use are a component of the device and must be available at all times.
- ▶ In the interest of patient safety, you and your patients are obliged to report serious events that occur in connection with this product to the manufacturer and to the authority responsible in your country.

**DANGER!****Risk of explosion**

- ▶ Do not use the device in an environment in which one of the following gases has accumulated:
 - oxygen
 - flammable anesthetics
 - other flammable substances/air mixtures

**CAUTION!****Patient hazard, damage to device**

- ▶ Additional devices connected to medical electrical devices must provide evidence of compliance with the relevant IEC or ISO standards (e.g. IEC 60950 for data-processing devices). Furthermore, all configurations must comply with the requirements of standards for medical systems (see IEC 60601-1-1 or Section 16 of edition 3.1 of IEC 60601-1 respectively). Anyone connecting additional devices to medical electrical devices is considered a system configurator and therefore responsible for ensuring that the system complies with the requirements of standards for systems. This also applies to additional devices recommended by seca. Your attention is drawn to the fact that local laws take precedence over the above-mentioned requirements of standards. In the event of any queries, please contact your local specialist dealer or Technical Service.

**CAUTION!****Patient hazard, damage to device**

- ▶ Have servicing carried out regularly as described in the relevant section of this document.
- ▶ Technical modifications may not be made to the device. The device does not contain any user-serviceable parts. Only have servicing and repairs performed by an authorized seca service partner. You can find your local service partner at www.seca.com.
- ▶ Use only seca original accessories and spare parts, otherwise seca will not grant any warranty.

**CAUTION!****Patient hazard, malfunction**

- ▶ Keep other medical electrical devices, e.g. high-frequency surgical devices, a minimum distance of approx. 1 meter away to prevent faulty measurements or wireless transmission interference.
- ▶ Keep HF devices such as cellphones at a minimum distance of approx. 1 meter to prevent faulty measurements or wireless transmission interference.
- ▶ The actual transmission output of HF devices may require minimum distances of more than 1 meter. For details, go to www.seca.com.

Preventing electric shock

WARNING! **Electric shock**

- ▶ Set up devices which can be operated with a power supply unit so that the power supply socket is within easy reach and the power supply can be disconnected quickly.
- ▶ Ensure that your local electricity supply matches the details on the power supply unit.
- ▶ Never touch the power supply unit with wet hands.
- ▶ Do not use extension cables and multiple outlets.
- ▶ Ensure that cables are not crushed or damaged by sharp edges.
- ▶ Ensure that cables do not come into contact with hot objects.
- ▶ Do not operate the device at an altitude of more than 3000 m above sea level.

Preventing injuries and infections

WARNING! **Injury from falling**

Baby scales are usually placed on elevated work surfaces. If the baby were to fall from the work surface, it could suffer serious irreversible or fatal injuries.

- ▶ Never leave a baby unsupervised.
- ▶ Route the connector cables (if present) to prevent users from tripping over them.
- ▶ Ensure that the device is steady and level.

WARNING! **Risk of infection**

- ▶ Before and after every measurement, wash your hands to reduce the risk of cross-contamination and nosocomial infections.
- ▶ Should the patient have contagious diseases, subject the device to a hygiene treatment as described in the relevant section of these instructions for use immediately afterwards.
- ▶ Ensure that the patient does not have any open wounds or infectious skin alterations which may come into contact with the device.
- ▶ Subject the device to a hygiene treatment at regular intervals.

Preventing damage to device

NOTICE!

Damage to device

- ▶ Ensure that fluids never get inside the device. These can destroy the electronics.
- ▶ Switch off the device before you disconnect the power supply unit from the power supply socket.
- ▶ If the device is not to be used for an extended period, disconnect the power supply unit from the power supply socket. Only then is the device de-energized.
- ▶ If you are not using the device for an extended period of time, remove batteries or battery packs. Only then is the device de-energized.
- ▶ Do not drop the device.
- ▶ Do not subject the device to shocks or vibrations.
- ▶ Perform a function check before each use as described in the corresponding section in this document. Do not operate the device if it is not working properly or is damaged.
- ▶ Do not place the device in direct sunlight and ensure that it is not placed in the direct proximity of a heat source. The excessive temperatures could damage the electronics.
- ▶ Avoid rapid temperature fluctuations. If the device is transported so that a temperature difference of over 20 °C occurs, the device must be left to stand for at least 2 hours before it is switched on, otherwise condensation may form; this may damage the electronics.
- ▶ Use the device only in the intended ambient conditions.
- ▶ Do not use the device over 3000 m above mean sea level. Electronic components might become damaged.
- ▶ Store the device only in the intended storage conditions.
- ▶ Use only cleaning agents and disinfectants that match the details in the section entitled "Hygiene treatment".
- ▶ Ensure that maximum capacity is not exceeded.

Handling measuring results

NOTICE!

Inconsistent measuring results

- ▶ Before you save and continue using measured values determined using this device (e.g. in seca software or in an EMR system), ensure that the measured values are plausible.
- ▶ If measured values have been sent to seca software or to an EMR system, ensure before continuing to use them that the measured values are plausible and assigned to the correct patient.

Handling packaging material

WARNING!

Risk of suffocation

Packaging materials made of plastic film (e.g. bags) present a risk of suffocation.

- ▶ Store packaging material out of the reach of children.
- ▶ If the original packaging material is no longer available, only use plastic bags with safety holes to reduce the risk of suffocation. Use recyclable materials if possible.

NOTE

Store the original packaging material for future use (e.g. returning for servicing).

Handling batteries and rechargeable batteries

WARNING!
Injury

Batteries and rechargeable batteries contain harmful substances which may explode and so escape if not handled properly.


- ▶ Do not try to recharge batteries.
- ▶ Do not expose batteries/rechargeable batteries to heat.
- ▶ Do not burn batteries/rechargeable batteries.
- ▶ If harmful substances are leaking out, avoid contact with the skin, eyes, and mucous membranes. Rinse affected areas of the body with plenty of clean water and seek medical help at once.

NOTICE!
Damage to device and malfunction due to incorrect handling

- ▶ Use only the type of battery/rechargeable battery quoted in this document.
- ▶ Always replace all batteries/rechargeable batteries at the same time.
- ▶ Do not short-circuit batteries/rechargeable batteries.
- ▶ If you do not use the device for an extended period of time, remove the batteries/rechargeable batteries. This prevents acid from leaking into the device.
- ▶ If harmful substances have penetrated the device, do not continue using it. Have the device checked by an authorized seca service partner and repaired if necessary.

4 OVERVIEW

4.1 Scale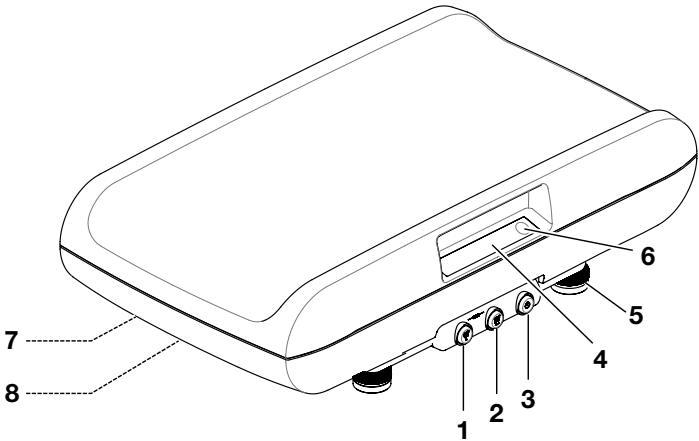


Item	Element	Function
1		hold/tare key ▲ • During weighing: – Press briefly: Activates Hold function – Press and hold: Activates Tare function • In the menu: – Select submenu, select menu item – Set value

17-10-07-703-100_2026-07S

Item	Element	Function
2		send/print key  - During weighing (if wireless network is set up): - Press briefly: To send measuring result to reception-ready devices (wireless printer, PC with USB wireless module) - Press and hold: Print out measuring result (wireless printer) - In the menu: - Confirm selected menu item - Save set value
3		OFF button Used for switching the device on and off
4	Display	Display element for measuring results and to configure the device
5	Foot screw	Used for precise alignment of the scale (4 pcs.) (seca 757 only)
6	Spirit level	Indicates whether the device is horizontal (seca 757 only)
7	Power supply connection	For connecting the power supply unit
8	Battery compartment	Takes batteries (type AA/1.5 V)

4.2 Symbols in the display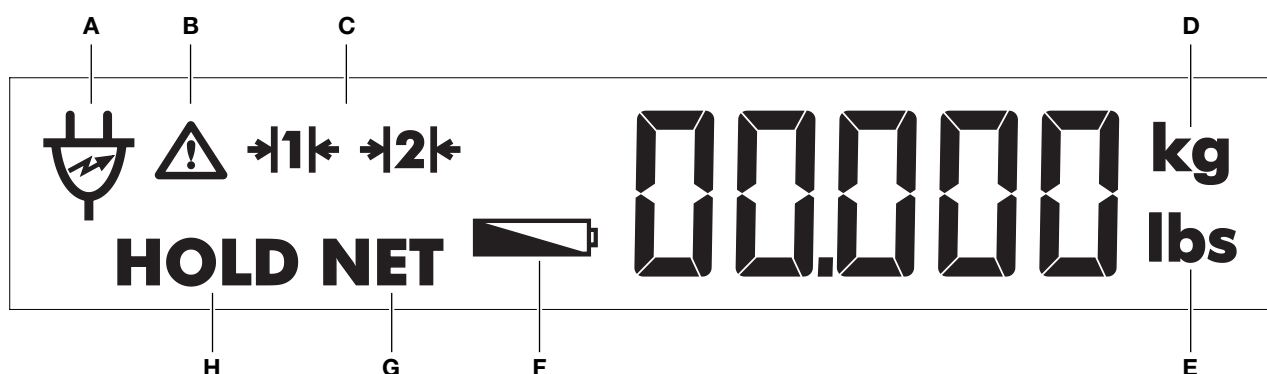

Item	Symbol	Meaning
A		Operation with power supply unit
B		Non-verifiable function active
C	→ 1 ← → 2 ←	Weighing range currently in use (seca 757 only)
D	kg	Weight in kilograms
E	lbs	Weight in pounds (seca 727 only)
F		Batteries are low
G	NET	Tare function active
H	HOLD	Hold function active

4.3 Menu

Navigating in the menu

hold
tare
▲

send
print
↵

menu

hold
tare
▲

send
print
↵

hold
tare
▲

send
print
↵

hold
tare
▲

send
print
↵

1. Press both the **hold/tare ▲** and **send/print ↵** keys.
⇒ The last menu item selected is displayed.

2. Press the **hold/tare ▲** key until the desired menu item is displayed.

3. Confirm your selection by pressing the **send/print ↵** key.
⇒ The current setting for the menu item or a submenu are displayed.

4. To change the setting or call up a different submenu, press the **hold/tare ▲** key until the desired setting is displayed.

5. Confirm your selection by pressing the **send/print ↵** key.
⇒ The desired setting is adopted.
⇒ The menu closes.

NOTE
If no key is pressed for a few seconds, the menu will automatically close.

Menu structure

```
graph TD
    rESET["rESET (Reset)"] <--> AHOLD["AHOLD (Autohold)"]
    AHOLD --> OnOff1["- On<br>- Off"]
    AHOLD <--> bEEP["bEEP (Beep)"]
    bEEP --> PrESS["PrESS (Press)"]
    bEEP --> HOLd["HOLd (Hold)"]
    PrESS --> OnOff2["- On<br>- Off"]
    HOLd --> OnOff3["- On<br>- Off"]
    bEEP <--> FIL["FIL (Filter)"]
    FIL --> ZeroOneTwo["- 0<br>- 1<br>- 2"]
    FIL <--> UnIt["UnIt (Unit)*"]
    UnIt --> grlbs["- gr<br>- lbs"]
    UnIt --> rESET
```

*nur seca 727


4.4 Labels

Markings on the device and on the type plate	
Text/symbol	Meaning
	Name and address of manufacturer, date of manufacture

38 • seca 757/727

17-10-07-703-100_2026-07S


Markings on the device and on the type plate	
Text/symbol	Meaning
UDI	Unique Device Identification (product identification number in accordance with Regulation (EU) 2017/745)
REF	Article number
SN	Serial number
GAL	Value in m/s^2 (model-dependent) • Gravitational acceleration on earth • Depends on the intended location
ProdID	seca product identification number
Approval Type	Type designation of design approval
Mat.No.	Variant number
	Follow instructions for use
	Medical electrical device, type B
MD	Medical device in accordance with Regulation (EU) 2017/745
IP20	Type of protection in accordance with IEC 60529: • Protection against ingress of solid foreign objects with a diameter of over 12.5 mm • Protection against access with fingers • Protection against dripping water
	Insulated device in accordance with IEC 60601-1: Protection class II
	Class III scale as per Directive 2014/31/EU
e	Value in units of mass Used to classify and verify a scale
d	Value in units of mass States the difference between two consecutive display values
	Active weighing range
	Device complies with EU directives • M: Conformity label in compliance with directive 2014/31/EU for non-automatic scales (verified models) • 24: (Example: 2024) Year in which declaration of conformity was implemented and the CE-label was applied (verified models) • 0102: Notified Body for Metrology (verified models) • 0123: Notified Body for Medical Devices
	Device meets the requirements of the USA and Canada. Certified and tested by a licensing laboratory (NRTL) of TÜV SÜD Product Services GmbH.

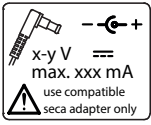Markings on the device and on the type plate	
Text/symbol	Meaning
	Device complies with United Kingdom directives • M: Conformity label in compliance with UK directive SI 2016 no. 1152 for non-automatic scales (NAWIR) (verified models) • 24: (Example: 2024) Year in which declaration of conformity was implemented and the UKCA label was applied (verified models) • xxxx: Notified Body for Medical Devices of the United Kingdom • yyyy: Notified Body for Metrology of the United Kingdom (verified models)
	Importer/representative in the United Kingdom: seca Ltd 40 Barn Street B5 5QB Birmingham United Kingdom
	Importer/representative in Switzerland: seca ag (schweiz) Medizinische Waagen und Messsysteme Schönmatt Str. 2 CH-4153 REINACH
	Type plate on the power supply connection socket • Power supply voltage required in V • Maximum current consumption in mA • : Note polarity of device connector • : Operate device with direct current • : Use compatible seca power supply units only
	Do not dispose of device in household waste

Markings on the packaging	
Symbol	Meaning
	Protect from moisture
	Arrows indicate top of product Transport and store in an upright position
	Fragile, do not throw or drop
	Permitted min. and max. temperature for transport and storage
	Permitted min. and max. humidity for transport and storage
	Permitted min. and max. air pressure for transport and storage
	Open packaging here
	Packaging material can be disposed of through recycling programs

5 **STARTING UP DEVICE**


5.1 Scope of delivery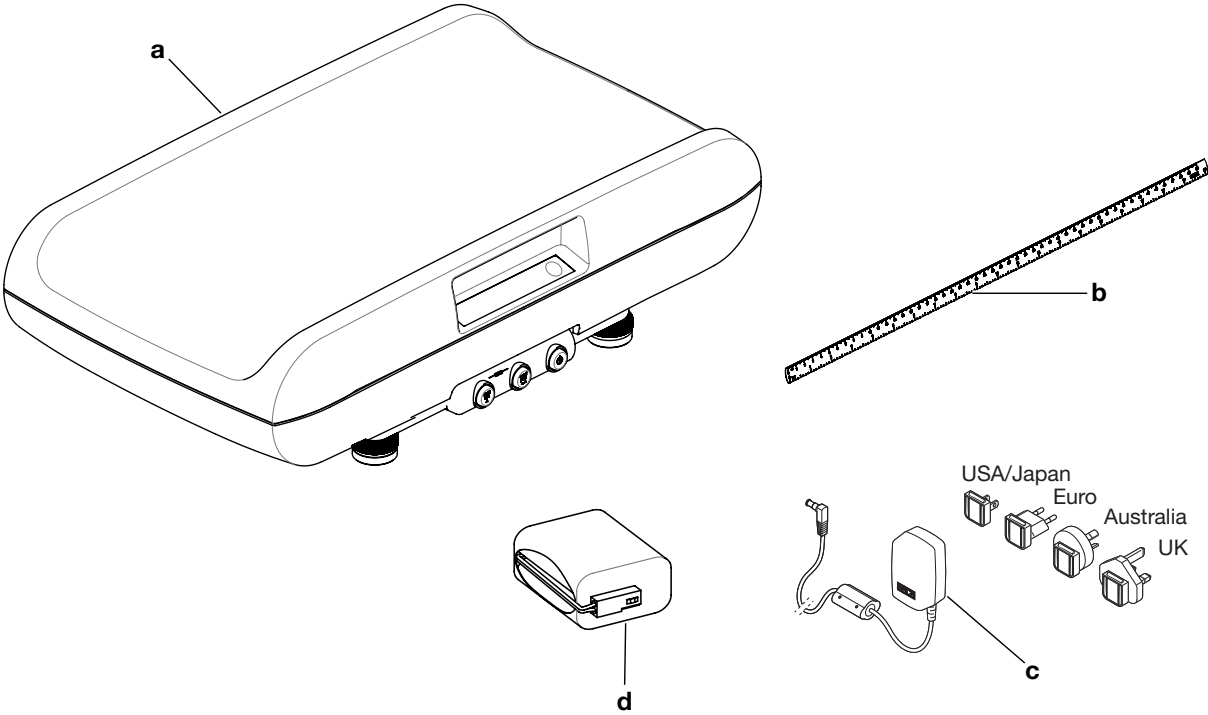

Item	Component	Pcs.
a	Baby scale	1
b	Height measuring tape, self-adhesive	1
c	Power supply unit with adapters	1
d	Battery block	1
Not shown	Instructions for use	1


5.2 Transporting the device

► Lift the scale by its housing as shown in the illustration.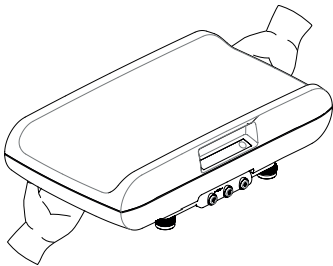

17-10-07-703-100_2026-07S

5.3 Removing the transport lock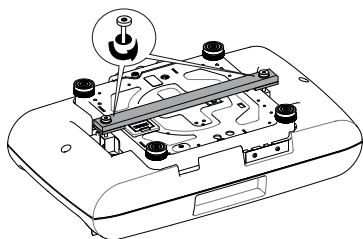


1. Carefully turn the scale over so that the base plate is on top.
2. Loosen the transport lock screws.
3. Remove the transport lock.

NOTE

Put the transport lock in a safe place for subsequent transport of the scale.

5.4 Attaching the height measuring tape

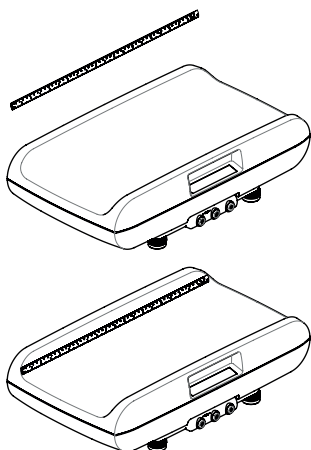The scale is supplied with a height measuring tape.

1. Clean and disinfect the tray thoroughly (→ [Hygiene treatment](#), page 49).

NOTE

The tray must be clean and dry before you attach the height measuring tape.

2. Remove the protective film from the back of the height measuring tape.
3. Attach the height measuring tape to the wall parallel to the upper edge of the tray, as shown in the adjacent figure.

5.5 Setting up and aligning the device

CAUTION!

Faulty measurement as a result of devices being set up incorrectly


- Place the device on a stable, level work surface.
- Align the device before using it and every time it changes location.
- Align the device only without a load on it.

NOTICE!

Faulty measurement as a result of force shunt

If the scale and its housing are in contact with something, e.g. a towel, the weight will not be measured correctly.

- Set the scale up so that only its foot screws are in contact with the ground.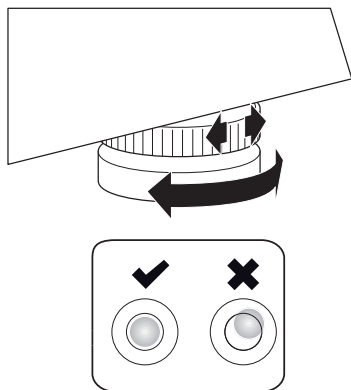

WARNING!

Faulty measurement as a result of force shunt


- Make sure that the scale does not touch the wall or other objects during measurement.

1. Place the scale on a firm, level surface.
2. Loosen the knurled wheels (**seca 757** only).
3. Align the scale by turning its foot screws so that the air bubble of the spirit level is in the exact center of the circle (**seca 757** only).
4. Tighten the knurled wheels (**seca 757** only).
⇒ The foot screws are secured against being maladjusted.

5.6 Establishing power supply


Power is supplied to the scale with a battery block or a power supply unit (both included in the scope of delivery).

Inserting the battery pack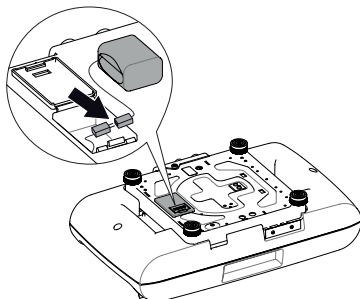

1. Press on the latch of the battery compartment.
2. Fold down the cover of the battery compartment.
3. Remove the connector cable from the battery compartment.
4. Connect the connector cable to the battery block.
5. Insert the battery block into the battery compartment.
6. Close the battery compartment.

Connecting the power supply unit

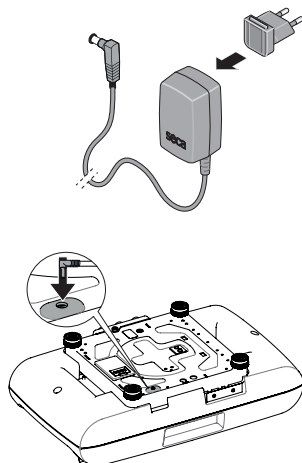WARNING!

Personal injury or damage to device as a result of incorrect power supply units

Conventional power supply units may deliver a higher voltage than is indicated on them. The measuring device may overheat, catch fire, melt or short-circuit.

- Use only original seca power supply units with a controlled 12 V output voltage.

1. Plug the adapter required for your region into the power supply unit.
2. Plug the power supply connector of the power supply unit into the connecting socket of the scale.
3. Plug the power supply unit into a power supply socket.
4. Carefully turn the scale over.

NOTE

Allow the scale to charge on the network for at least 24 hours when it is first charged so that the battery block is completely charged

5.7 Setting up the data connection (optional)

If you want to set up a data connection via **seca connect 103** and **seca 452**, contact your hospital technician, your IT administrator or seca Service.

6 OPERATION

WARNING!

Personal injury, faulty measurement

- Before using the device each time, perform a function check as described in the corresponding section of these instructions for use.

6.1 Switching the device on and off

Switching on the device

- ✓ The device has no load.

- Press the  key.
 - ⇒ **SECA** is displayed.
 - ⇒ All the elements of the display are shown briefly.
 - ⇒ The device is ready for operation when **0.000** is displayed.

NOTE

If the measuring device is operated with a power supply unit, the  symbol appears in the display.

Switching off the device

- Press the ON/OFF button .

NOTE

If there is no input during the standby time, the device switches off automatically.

6.2 Measuring weight

WARNING!

Injury from falling

Baby scales are usually placed on elevated work surfaces. If the baby were to fall from the work surface, it could suffer serious irreversible or fatal injuries.

- Never leave a baby unsupervised.
- Route the connector cables (if present) to prevent users from tripping over them.
- Ensure that the device is steady and level.

Weighing

- ✓ The device has no load.

1. Press the  key.
2. Place the baby on the scale.
3. Make sure that the baby is lying as still as possible.
 - ⇒ The display flashes until a stable weight is measured.
4. Read off the measuring result.

NOTE

When the Autohold function is activated, the weight value is automatically displayed permanently.

Taring additional weight (tare)

Using the **Tare** function, you can prevent any additional weight (e.g. a towel or a cushioned mat on the weighing surface) from affecting the measuring result.

WARNING!

Faulty measurement

When the **Tare** function is used incorrectly, the weight of the patient may not be determined correctly.

- Perform the steps as described to prevent faulty measurements.

WARNING!

Faulty measurement

If an additional weight is touching the surface on which the device is standing, the weight will not be measured correctly.

- Make sure that additional weights are placed only on the tray.

NOTE

The maximum weight which can be displayed is reduced by the weight of the objects already placed on the scale.

- ✓ The device has no load.

1. Press the  key.
2. Place the additional weight on the scale.
3. Press and hold the **hold/tare**  key until the **NET** message is displayed.
4. Wait until the display stops flashing and **0.000** is displayed instead.
5. Place the baby on the scale.
6. Make sure that the baby is lying as still as possible.
⇒ The display flashes until a stable weight is measured.
7. Read off the measuring result.
⇒ The additional weight is deducted automatically.

NOTE

You can perform as many measurements as you like with this additional weight.

8. To deactivate the **Tare** function, press and hold the **hold/tare**  key until the **NET** message is no longer displayed.

NOTE

When you switch the device off, the **Tare** function is deactivated automatically.

Displaying measuring result permanently (hold)

When you activate the **Hold** function, the weight value continues to be displayed after the load is removed from the scale. This allows you to take care of the baby before recording the weight value.


1. Place the baby on the scale.
2. Briefly press the **hold/tare** key .
⇒ The display flashes until a stable weight is measured.
⇒ The weight value is displayed permanently.
⇒ The message  **HOLD** is displayed.
3. Read off the measuring result.
4. To deactivate the **Hold** function, briefly press the **hold/tare** key .

NOTE

When the **Autohold** function is activated, the weight value is displayed automatically.

Automatic switchover of weighing range (seca 757 only)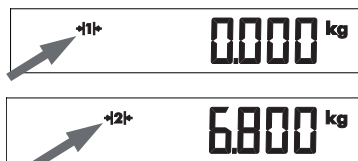

The scale has two weighing ranges. In weighing range 1 (→|1|←), graduations in the weight display are finer. In weighing range 2 (→|2|←), you can use the maximum capacity of the scale.

After the scale is switched on, weighing range 1 is active.

If a particular weight value is exceeded, the scale automatically switches to weighing range 2.

- To switch back to weighing range 1, completely remove the weight from the scale.
⇒ Weighing range 1 is active again.

6.3 Other functions (menu)

Activating/deactivating the Autohold function (AHOLD)

Other functions and settings are available in the menu. For a summary of the menu structure, see the section entitled "Overview".

When you activate the **Autohold** function, the measuring result for every weighing procedure continues to be displayed after the weight has been removed from the scale. It is then no longer necessary to activate the **Hold** function manually for each individual weighing procedure.

1. From the menu, select the **AHOLD** menu item.
2. Confirm your selection.
⇒ The current setting is displayed.
3. Select the desired setting:
 - **On**
 - **OFF**
4. Confirm your selection.
⇒ The menu closes.

Activating/deactivating acoustic signals (bEEP)

You can set whether an acoustic signal is heard:

- Whenever pressing the key
- When a stable weight value is reached (**Hold/Autohold**)

1. From the menu, select the **bEEP** menu item.
2. Confirm your selection.
3. Select a menu item:
 - **PrESS**: beep when key is pressed
 - **HOLD**: acoustic signal when weight value is stable.
4. Confirm your selection.
⇒ The current setting is displayed.
5. Select the desired setting:
 - **On**
 - **OFF**
6. Confirm your selection.
⇒ The menu closes.

Setting attenuation (FIL)

Attenuation (**FIL** = filter) allows you to reduce interference during weight determination. The selected setting affects the sensitivity with which the weight display reacts to patient movements and the time until the **Hold** function displays a weight value permanently (**Hold** value).

NOTE

With the **0** setting and patients who are not very steady on their feet, it is possible that no weight value is displayed permanently despite the **Hold** function being activated.

The greatest deviation between displayed and actual weight value occurs at setting **2**.

FIL 1

FIL 0

1. From the menu, select the **FIL** menu item.
2. Confirm your selection.
⇒ The current setting is displayed.
3. Select an attenuation level.

Attenuation level (FIL)	Sensitivity	Determination of Hold value
0	Sensitive	Slow
1	Medium	Medium
2	Slow	Fast

4. Confirm your selection.
⇒ The menu closes.

Setting display backlighting (LCd)

You can change the duration and brightness of the display backlighting.

Lcd

dUr

brl

1. From the menu, select the **Lcd** menu item.
2. Confirm your selection.
3. Select a menu item:
 - **dUr**: Duration
 - **brl**: Brightness
4. Confirm your selection.
⇒ The current setting is displayed.
5. Select the desired setting:

Function	Setting
dUr (duration)	• SHort (approx. 15 sec.) • Lon9 (approx. 45 sec.) • PErM (permanent)
brl (brightness)	• 50 % • 100 % • OFF

6. Confirm your selection.
⇒ The menu closes.

Switching the weight unit (seca 727 only)

WARNING! **Patient hazard**

To prevent misinterpretations, measuring results for medical purposes must only be displayed and used in SI units (kilograms/grams, meters/centimeters). Some devices have the option of displaying measuring results in different units. This is purely an additional function.

- ▶ Only use the measuring results in SI units.
- ▶ You as the user take sole responsibility for the use of measuring results in non-SI units.

During the measurement procedure, you can switch between kilograms/grams and pounds.

Unit

gr kg

Lbs lbs

1. Select the **Unit** menu item.
2. Confirm your selection.
⇒ The current setting is displayed.
3. Select the unit in which you want weight to be displayed:
 - Kilograms/grams (**gr**)
 - Pounds (**lbs**)
4. Confirm your selection.
⇒ The setting is adopted.
⇒ The menu closes.

Restoring factory settings (rESEt)

You can restore the factory settings for the functions below:

Function	Factory setting
Weight unit (Unit)	gr
Autohold (AHOLd)	model-dependent
Acoustic signal (PrESS)	OFF
Acoustic signal (HOLd)	On
Attenuation (FIL)	1

rESEt

1. From the menu, select the **rESEt** menu item.
2. Confirm your selection.
⇒ The menu closes.
3. Switch off the scale.
⇒ Factory settings are restored and are available when the scale is switched back on.

7 HYGIENE TREATMENT

WARNING! **Electric shock**

The device is not de-energized when the ON/OFF button is pressed and the display goes out. Use of fluids on the device may cause an electric shock.

- ▶ Before each hygiene treatment, ensure that the device is switched off.
- ▶ Disconnect the power supply connector before each hygiene treatment.
- ▶ Take the battery block out of the device before every hygiene treatment.
- ▶ Ensure that no fluids penetrate the device.

WARNING! **Risk of infection**

- ▶ Subject the device to a hygiene treatment at regular intervals as described in this section.

NOTICE!

Damage to device

Unsuitable cleaning agents and disinfectants may damage the sensitive surfaces of the device and impair its operability.

- ▶ Do not use aggressive or abrasive cleaning agents.
- ▶ Do not use organic solvents (e.g. white spirit or petroleum spirit).

7.1 Cleaning

- ▶ If required, moisten a soft cloth with a mild soap solution and wipe the device over with it.

7.2 Disinfecting

1. Disinfect the device at regular intervals using a disinfectant suitable for sensitive surfaces and acrylic glass. seca recommends a 70 % ethanol solution as a surface disinfectant.
2. Follow the instructions for use of the disinfectant.
3. Disinfect the device:
 - ▶ Moisten a soft cloth with disinfectant and wipe down the device with it.
 - ▶ Comply with the intervals, see table:

Interval	Component
Before and after each measurement	Tray
As required	• Display • Controls

7.3 Sterilizing

The device must not be sterilized.

8 FUNCTION CHECK

CAUTION! Personal injury

If you find faults or deviations during the function check which you are unable to remedy with the aid of the “Troubleshooting” section in this document, you must not use the device.

- Have the device repaired by seca Service or an authorized service partner.
- Follow the section entitled “Servicing” in this document.

- Perform a function check before each use.

A complete function check includes:

- A visual inspection for mechanical damage
- A test of the alignment of the device
- A visual and function check of the display elements
- A function check of all the controls shown in the section entitled “Overview”
- A function check of the optional accessories

If you find faults or deviations during the function check, initially try to remedy the fault with the aid of the “Troubleshooting” section in this document.

9 TROUBLESHOOTING

9.1 General faults

Fault	Cause	Remedy
No weight is displayed with a load on the scale.	The device has no power supply.	• Switch on device. • Insert battery pack.
 is displayed.	Voltage of the battery pack is dropping.	Insert a new battery pack as soon as possible
bAtt is displayed.	The battery pack is discharged.	Insert new battery pack
StOP is displayed.	Maximum capacity has been exceeded.	Remove the load from the device.
tEMP is displayed.	The ambient temperature of the scale is too high or too low.	• Operate the device only in the intended ambient conditions (General technical data). • After transport or storage of the device, wait approx. 15 minutes before switching it on.
The device no longer reacts when keys are pressed.	The keypad is defective.	Inform seca Service.
One display segment is displayed continuously or not at all.	The display is defective.	Inform seca Service.
0.000 is not displayed before weighing takes place	A load was already on the device before it was switched on.	• Remove the load from the device • Switch off the device, then switch it back on again
Implausible measuring results.	Patient moved too much during the measurement.	Take the load off the device, wait until 0.000 is displayed and repeat the measurement procedure.

Fault	Cause	Remedy
Er:[No.]:11 is displayed.	The device has too high a load or too high a load in one corner.	- Remove the load from the device or distribute the weight more evenly. - Switch off the device, then switch it back on again. - If the problem persists, inform seca Service.
Er:[No.]:12 is displayed.	The device has been switched on with too high a load.	- Remove the load from the device. - Switch off the device, then switch it back on again.
Er:[No.]:16 is displayed.	The device was caused to oscillate, the zero point could not be determined.	Switch off the device, then switch it back on again.

10 SERVICING

10.1 Verified scales

CAUTION!

Faulty measurements as a result of verification being performed poorly or not at all

- Have verification performed only by authorized persons.
- Always have verification performed if one or more quality seals are damaged.

seca recommends having your device serviced prior to verification.

CAUTION!

Faulty measurements as a result of poor servicing

- Have servicing and repairs carried out exclusively by seca Service or an authorized service partner.
- You can find your local service partner at www.seca.com.


Have authorized persons perform verification in line with national legal regulations.

Verification is necessary whenever one or more quality seals are damaged or the contents of the verification counter no longer match the number on the applicable verification sticker. If quality seals are damaged, contact seca Service directly.

Verifications may only be performed by authorized agencies. To guarantee this, the scale is equipped with a verification counter which records each change in verification-related data.

If you want to check whether the scale has been properly verified, proceed as follows:

- ✓ The device is switched off.

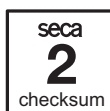- Press and hold any key and switch the scale on.
⇒ The current content of the verification counter flashes in the display for a few seconds.
- Compare the content of the verification counter displayed with the number on the verification counter seal.

Both numbers have to match for the verification to be valid. If the verification sticker and the verification counter do not match, the scale must be verified. Please contact your service partner or seca Service. Once the scale has been verified, a new, updated verification counter sticker is used to identify the verification counter reading. The person authorized to perform verification secures this verification with an additional seal. The verification counter sticker can be ordered from seca Service.

10.2 Non-verified scales

The product needs to be set up carefully and serviced regularly. Depending on how frequently the product is used, seca recommends servicing at intervals of 3 to 5 years.

CAUTION!

Faulty measurements as a result of poor servicing

- ▶ Have servicing and repairs carried out exclusively by seca Service or an authorized service partner.
- ▶ You can find your local service partner at www.seca.com.

11 TECHNICAL DATA

11.1 General technical data

General technical data	
Dimensions • Depth • Width • Height	320 mm 551 mm 151 mm
Net weight	approx. 7.1 kg
Height of digits	11 mm
Ambient conditions, operation • Temperature • Air pressure • Humidity	+10 °C to +40 °C / +50 °F to 104 °F 700 – 1060 hPa 30 % – 80 % no condensation
Ambient conditions, storage/transport • Temperature • Air pressure • Humidity	-10 °C to +65 °C / +14 °F to 149 °F 700 – 1060 hPa 0 % – 95 % no condensation
Battery pack • Supply voltage • Maximum running time	7.2 V approx. 48 hours
Power supply unit • Supply voltage • Current consumption (without permanent background lighting) • Insulated device, protection class II (IEC 60601-1) • Mains voltage • Power supply frequency	12 V approx. 47 mA  100 V – 240 V 50 Hz – 60 Hz
Application parts in accordance with IEC 60601-1	Tray
Medical device in accordance with regulation (EU) 2017/745	Class I with measuring function
IEC 60601-1: Medical electrical device, Type B	
Type of protection in accordance with IEC 60529	IP20

17-10-07-703-100_2026-07S

General technical data	
Duty cycle	Continuous duty
Service life, from commissioning ^a	at least 10 years

^a Minimum service life when used as intended and serviced in a timely manner. seca products usually have a significantly longer service life.

11.2 Weight measurement

seca 757	
Verification in compliance with Directive 2014/31/EU	Class III
Maximum capacity	
• Weighing range 1	5 kg
• Weighing range 2	15 kg
Minimum capacity	
• Weighing range 1	0.04 kg
• Weighing range 2	0.1 kg
Graduation	
• Weighing range 1, < 5 kg	2 g
• Weighing range 2, > 5 kg	5 g
Tare range	20 kg (subtractive)
Accuracy on initial verification	
• Weighing range 1: < 1 kg	± 1 g
• Weighing range 1: 1 kg to 4 kg	± 2 g
• Weighing range 1: 4 kg to 5 kg	± 3 g
• Weighing range 2: < 2.5 kg	± 2.5 g
• Weighing range 2: 2.5 kg to 10 kg	± 5 g
• Weighing range 2: 10 kg to 15 kg	± 7.5 g

seca 727	
Maximum capacity	20 kg / 44 lbs
Minimum capacity	0.02 kg / 0.04 lbs
Graduation	
• < 6 kg / 13 lbs	1 g / 0.1 oz
• 6 kg to 20 kg / 13 to 44 lbs	2 g / 0.1 oz
Tare range	20 kg / 44 lbs
Accuracy	
• < 6 kg / 13 lbs	± 2 g / ± 0.1 oz
• 6 kg to 20 kg / 13 to 44 lbs	± 0.15 %
at ambient temperature of 20° C and humidity of 45 %	

11.3 Height measurement

Metrology data, height measurement	
Height measuring tape	
• Measuring range	0 – 50 cm / 0 – 20"
• Graduation	1 mm / 1/8"

12 OPTIONAL ACCESSORIES AND SPARE PARTS

Accessory/spare part	Article number
Height measuring tape, self-adhesive	19 17 01 264
Carts for seca baby scales seca 402	402 0000 009
Carts with drawer for seca baby scales seca 403	403 0000 009
NiMH battery block: 7.2 V= / 1300 mAh	68 22 12 721 009
Switch-mode power supply unit	68 32 10 265
Power pack: seca 447 : • Input: 100 V – 240 V~, 50 Hz – 60 Hz • Output: 12 V= / 0.5 A	447 0000 009
Mobile power supply seca 454	454 0000 009
Accessories for integration via WiFi/LAN • External interface module seca 452 • Software seca connect 103 • Barcode scanner	452 0030 009 Will be provided in the course of integration projects Cannot be ordered through seca; for recommendations, see System instructions for use for seca 103/452

13 DISPOSAL

13.1 Disposing of the device

Do not dispose of the device in your household waste. The device must be properly disposed of as electronic scrap. Follow your respective national regulations. For more information, please contact seca Service or an authorized service partner.

13.2 Disposing of batteries and rechargeable batteries

Do not dispose of used batteries and rechargeable batteries in household waste, regardless of whether they contain harmful substances or not. As a consumer, you are legally obliged to dispose of batteries and rechargeable batteries via local authority collection points or trade collection points. Only dispose of batteries and rechargeable batteries once they are completely discharged.

14 WARRANTY

There is a two-year warranty period from delivery for defects attributable to poor materials or workmanship. All movable parts, e.g. batteries, cables, power supply units, rechargeable batteries etc. are exempt. Defects which come under the warranty will be repaired for the customer free of charge against proof of purchase. Additional claims cannot be considered. Costs of transport to and from seca are the responsibility of the customer if the device is located somewhere other than the customer's headquarters. In the event of transport damage, claims under warranty can only be made if the complete original packaging was used for transport and the device was secured and fastened in it according to its originally packaged condition. You should therefore keep all packaging parts.

The warranty will be voided if the device is opened by persons not expressly authorized by seca to do so.

In the event of a warranty issue, please contact your local seca office or the dealer from whom you ordered the product.

15 DECLARATION OF CONFORMITY

seca gmbh & co. kg hereby declares that the product complies with the terms of the applicable European directives and regulations. The unabridged declaration of conformity can be found at www.seca.com.

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