
seca 745/725

Instructions for Use

17-10-07-704-100_2026-09S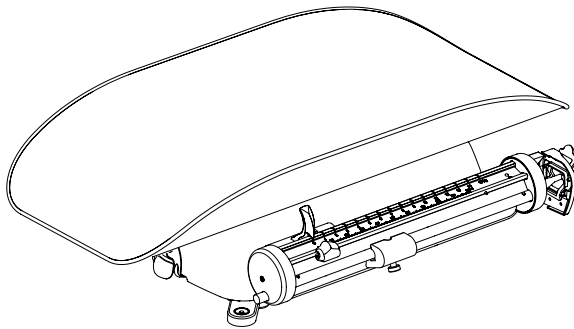

Deutsch – Gebrauchsanweisung	3
English – Instructions for Use.....	28
Français – Mode d’emploi.....	52
Italiano – Istruzioni per l’uso	76
Español – Instrucciones de uso	100
Português – Manual de instruções de utilização	124
Polski – Instrukcja obsługi.....	150

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CONTENTS

1 About this document	29		
1.1 Representation in text	29		
1.2 Representation in diagrams	29		
2 Description of device	30		
2.1 Intended use	30		
2.2 Contraindications	30		
2.3 Clinical benefit	30		
2.4 Patient target group	30		
2.5 User qualification	30		
2.6 Functional description	30		
3 Safety precautions	31		
3.1 Safety information in these instructions for use	31		
3.2 Basic safety information	31		
Handling device	31		
Preventing injuries and infections	32		
Preventing damage to device	33		
Handling measuring results	33		
Handling packaging material	33		
4 Overview	34		
4.1 Scale	34		
4.2 Labels	35		
5 Starting up device	37		
5.1 Scope of delivery	37		
5.2 Removing the transport lock	38		
5.3 Attaching the tray	38		
5.4 Transporting the device	38		
5.5 Setting up and aligning the device	39		
		5.6 Making the scale operational	40
		Releasing/locking the beam	40
		Checking the beam position	40
		Align the beam horizontally	41
6 Operation	42		
6.1 Measuring weight	42		
Weighing	43		
Taring off additional weight	44		
7 Hygiene treatment	45		
7.1 Cleaning	45		
Cleaning housing	45		
7.2 Disinfecting	45		
7.3 Sterilizing	46		
8 Function check	46		
9 Troubleshooting	47		
10 Servicing	48		
10.1 Verified scales	48		
10.2 Non-verified scales	48		
11 Technical data	49		
11.1 General technical data	49		
11.2 Weight measurement	49		
12 Disposal	50		
12.1 Disposing of the device	50		
13 Warranty	50		
14 Declaration of conformity	51		

1 ABOUT THIS DOCUMENT

1.1 Representation in text

Symbol	Description
✓	Requirement for actions
▶	Action
1. 2.	Actions with specified sequence
a) b)	Steps of an action with specified sequence
⇒	Result of an action
• •	First level of a list
– –	Second level of a list

1.2 Representation in diagrams

Symbol	Description
	Indicates relevant points on the device or on device components
	Indicates directions of movement of the device or of device components
	Navigation path in menu displays

2 DESCRIPTION OF DEVICE

2.1 Intended use

The mechanical baby scale supports physicians in decision-making regarding diagnosis or therapy, based on the patient's weight or weight changes.

2.2 Contraindications

No contraindications are known.

2.3 Clinical benefit

The mechanical baby scale supports physicians in decision-making regarding diagnosis or therapy, based on the measured weight. With that, the mechanical baby scale only provides an indirect clinical benefit.

The physician must perform additional examinations for an accurate diagnosis.

2.4 Patient target group

The device is intended for babies and toddlers who do not exceed the maximum capacity of the scale and fit on the tray.

2.5 User qualification

The device may only be operated by persons with formal training in the healthcare sector or in medicine.

2.6 Functional description

Weight calculation takes place using a load lever system.

3 SAFETY PRECAUTIONS

3.1 Safety information in these instructions for use

DANGER!

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries will occur.

WARNING!

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries may result.

CAUTION!

Used to identify a hazardous situation. If you fail to take note of this information, minor to moderate injuries may result.

NOTICE!

Used to identify possible incorrect usage of the device. If you fail to take note of this information, you may damage the device, or the measuring results may be incorrect.

NOTE

Includes additional information about use of the device.

3.2 Basic safety information

Handling device

- ▶ Please take note of the information in these instructions for use.
- ▶ Keep the instructions for use in a safe place. The instructions for use are a component of the device and must be available at all times.
- ▶ In the interest of patient safety, you and your patients are obliged to report serious events that occur in connection with this product to the manufacturer and to the authority responsible in your country.

**CAUTION!****Patient hazard, damage to device**

- ▶ Have servicing carried out regularly as described in the relevant section of this document.
- ▶ Technical modifications may not be made to the device. The device does not contain any user-serviceable parts. Only have servicing and repairs performed by an authorized seca service partner. You can find your local service partner at www.seca.com.
- ▶ Use only seca original accessories and spare parts, otherwise seca will not grant any warranty.

**Preventing injuries
and infections****WARNING!****Injury from falling**

Baby scales are usually placed on elevated work surfaces. If the baby were to fall from the work surface, it could suffer serious irreversible or fatal injuries.

- ▶ Never leave a baby unsupervised.
- ▶ Route the connector cables (if present) to prevent users from tripping over them.
- ▶ Ensure that the device is steady and level.

**WARNING!****Risk of infection**

- ▶ Before and after every measurement, wash your hands to reduce the risk of cross-contamination and nosocomial infections.
- ▶ Should the patient have contagious diseases, subject the device to a hygiene treatment as described in the relevant section of these instructions for use immediately afterwards.
- ▶ Ensure that the patient does not have any open wounds or infectious skin alterations which may come into contact with the device.
- ▶ Subject the device to a hygiene treatment at regular intervals.

Preventing damage to device

NOTICE!

Damage to device

- ▶ Do not drop the device.
- ▶ Do not subject the device to shocks or vibrations.
- ▶ Perform a function check before each use as described in the corresponding section in this document. Do not operate the device if it is not working properly or is damaged.
- ▶ Use only cleaning agents and disinfectants that match the details in the section entitled "Hygiene treatment".
- ▶ For scales: Ensure that maximum capacity is not exceeded.

Handling measuring results

NOTICE!

Inconsistent measuring results

- ▶ Before you save and continue using measured values determined using this device (e.g. in seca software or in an EMR system), ensure that the measured values are plausible.
- ▶ If measured values have been sent to seca software or to an EMR system, ensure before continuing to use them that the measured values are plausible and assigned to the correct patient.

Handling packaging material

WARNING!

Risk of suffocation

Packaging materials made of plastic film (e.g. bags) present a risk of suffocation.


- ▶ Store packaging material out of the reach of children.
- ▶ If the original packaging material is no longer available, only use plastic bags with safety holes to reduce the risk of suffocation. Use recyclable materials if possible.

NOTE

Store the original packaging material for future use (e.g. returning for servicing).

4 OVERVIEW

4.1 Scale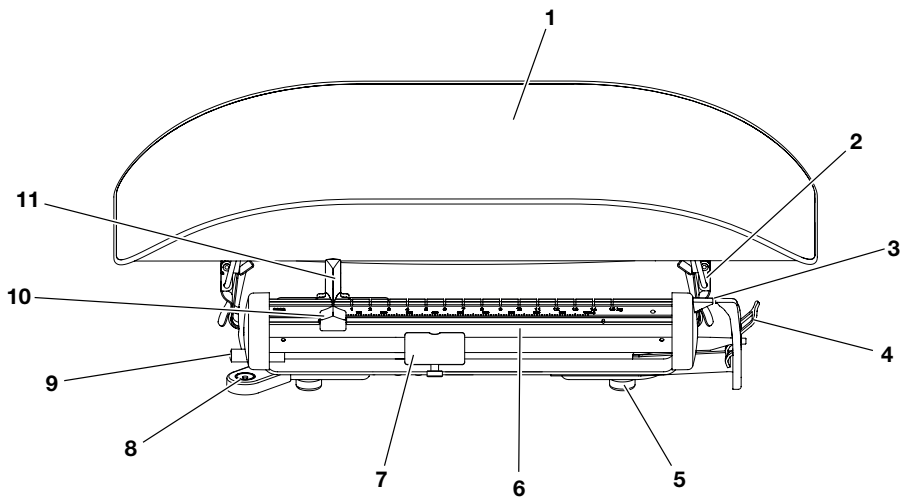

Item	Element	Function
1	Tray	Contact area for babies
2	Locking levers	2 pcs., are used to lock the tray onto the housing
3	Pointer and counter-pointer	Indicate whether the beam is aligned horizontally
4	Locking lever	Is intended for locking/releasing the beam • Press downward: Locking (for transport) • Push upward: Release (for weighing)
5	Foot screw	Used for precise alignment of the scale (4 pcs.) (seca 745 only)
6	Beam with scale	For reading off the weight
7	Tare weight with knurled screw	Can be moved to align the beam horizontally • On commissioning and before every weighing procedure • To tare off an additional weight
8	Spirit level	Indicates whether the device is horizontal (seca 745 only)

17-10-07-704-100_2026-09S

Item	Element	Function
9	Fine-tare screw	Can be adjusted to exactly align the beam horizontally - On commissioning and before every weighing procedure - To tare off an additional weight
10	Small sliding weight	Can be moved to set the weight display accurately
11	Large sliding weight	Can be moved to set the weight display roughly - Pull forward: Release the sliding weight - Release: Lock the sliding weight

4.2 Labels

Markings on the device and on the type plate	
Text/symbol	Meaning
	Name and address of manufacturer, date of manufacture
UDI	Unique Device Identification (product identification number in accordance with Regulation (EU) 2017/745)
	Article number
	Serial number
ProdID	seca product identification number
Approval Type	Type designation of design approval
Mat.No.	Variant number
	Follow instructions for use
	Medical device in accordance with Regulation (EU) 2017/745
e	Value in units of mass Used to classify and verify a scale
d	Value in units of mass States the difference between two consecutive display values
	Class III scale as per Directive 2014/31/EU


Markings on the device and on the type plate	
Text/symbol	Meaning
	Device complies with EU directives • M: Conformity label in compliance with directive 2014/31/EU for non-automatic scales (verified models) • 24: (Example: 2024) Year in which declaration of conformity was implemented and the CE-label was applied (verified models) • 0102: Notified Body for Metrology (verified models) • 0123: Notified Body for Medical Devices
	Device complies with United Kingdom directives • M: Conformity label in compliance with UK directive SI 2016 no. 1152 for non-automatic scales (NAWIR) (verified models) • 24: (Example: 2024) Year in which declaration of conformity was implemented and the UKCA label was applied (verified models) • xxxx: Notified Body for Medical Devices of the United Kingdom • yyyy: Notified Body for Metrology of the United Kingdom (verified models)
 	Importer/representative in the United Kingdom: seca Ltd 40 Barn Street B5 5QB Birmingham United Kingdom
 	Importer/representative in Switzerland: seca ag (schweiz) Medizinische Waagen und Messsysteme Schön matt Str. 2 CH-4153 REINACH

Markings on the packaging	
Symbol	Meaning
	Protect from moisture
	Arrows indicate top of product Transport and store in an upright position
	Fragile, do not throw or drop
	Permitted min. and max. temperature for transport and storage

Markings on the packaging	
Symbol	Meaning
	Permitted min. and max. humidity for transport and storage
	Permitted min. and max. air pressure for transport and storage
	Open packaging here
	Packaging material can be disposed of through recycling programs


5 STARTING UP DEVICE

5.1 Scope of delivery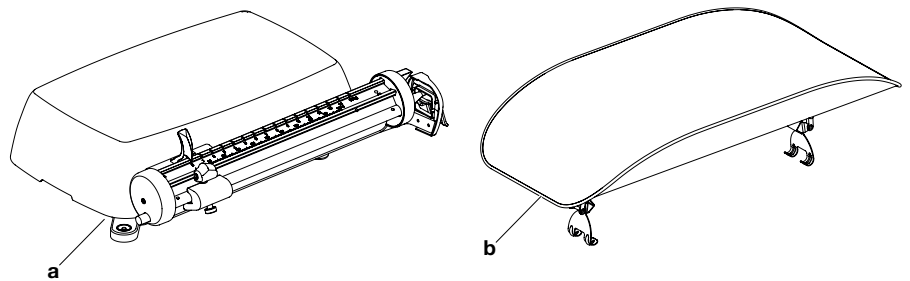

Item	Component	Pcs.
a	Baby scale	1
b	Tray	1
Not shown	Instructions for use	1

5.2 Removing the transport lock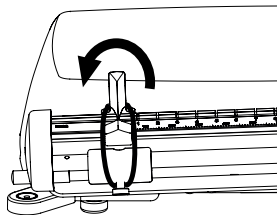


The sliding weights and the tare weight are secured by a transport lock (rubber ring).

- Remove the transport lock as shown in the adjacent diagram.


NOTE

- Put the transport lock in a safe place for subsequent transport of the scale.

5.3 Attaching the tray

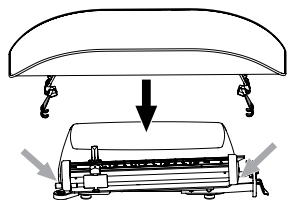
1. Place the tray in the center of the scale.
2. Make sure that the locking levers of the tray grip into the openings of the housing.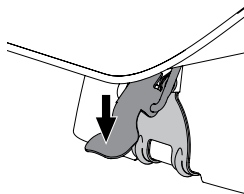

3. Press the locking levers downward.
- ⇒ The tray is fastened to the housing.

5.4 Transporting the device

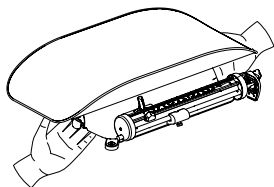NOTICE!

Damage to device from incorrect handling

The tray is not intended for transport of the scale. The tray can become deformed or come off the scale as a result of high strain.

- When transporting the scale, hold it only by the housing.
- Lift the scale by its housing as shown in the illustration.

5.5 Setting up and aligning the device

CAUTION!

Faulty measurement as a result of devices being set up incorrectly


- ▶ Place the device on a stable, level work surface.
- ▶ Align the device before using it and every time it changes location.
- ▶ Align the device only without a load on it.

WARNING!

Faulty measurement as a result of force shunt

- ▶ Make sure that the scale does not touch the wall or other objects during measurement.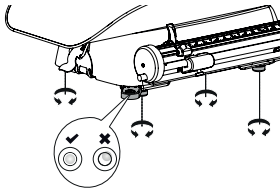


1. Place the scale on a firm, level surface.
2. Loosen the knurled wheels (**seca 745** only).
3. Align the scale by turning its foot screws so that the air bubble of the spirit level is in the exact center of the circle (**seca 745** only)
4. Ensure that only the feet of the scale are in contact with the ground.
5. Tighten the knurled wheels (**seca 745** only).
 - ⇒ The foot screws are secured against being maladjusted.

5.6 Making the scale operational


Releasing/locking the beam

The beam must be locked when the device is being transported and when the tray is to be placed on the housing. Before weighing, the beam must be released.

- To release the beam, push the locking lever upward.

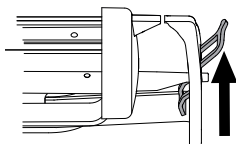
- To lock the beam, push the locking lever downward.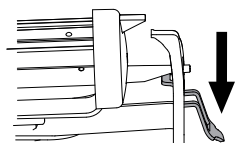

Checking the beam position

- ✓ The device has no load.
- ✓ Both sliding weights are in the 0 position.
- ✓ The locking lever of the beam is released.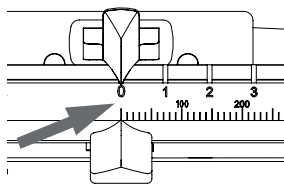


CAUTION!

Faulty measurement as a result of force shunt

If you touch the scale, weight will not be measured correctly.

- Do not touch the scale while the beam is rocking.



1. Wait until the beam has stopped rocking.
2. Check whether pointer and counterpointer are at exactly the same level.
3. If the pointer and the counterpointer are not at exactly the same level, align the beam horizontally as described in the section below.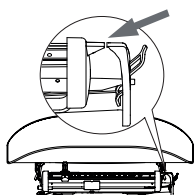

Align the beam horizontally

The beam is aligned horizontally when pointer and counterpointer are at exactly the same level.

- ✓ Both sliding weights are in the 0 position.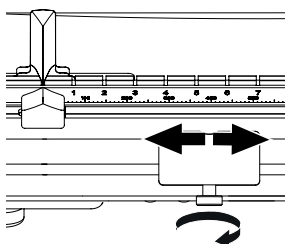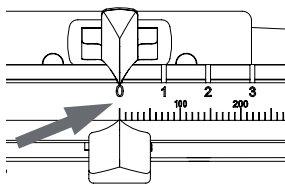

1. Release the knurled screw on the tare weight.
2. Move the tare weight to align the beam horizontally.
3. Wait until the beam has stopped rocking.


CAUTION!

Faulty measurement as a result of force shunt

If you touch the scale, weight will not be measured correctly.

- Do not touch the scale while the beam is rocking.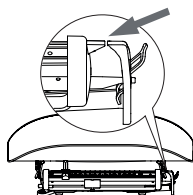

4. Check whether pointer and counterpointer are at exactly the same level.
5. If necessary, repeat step 2.
6. Fix the tare weight with the knurled screw.
7. Turn the fine-tare screw until pointer and counterpointer are at exactly the same level.

⇒ The beam is aligned horizontally.

⇒ The scale is ready for operation.

6 OPERATION

WARNING!

Personal injury, faulty measurement

- Before using the device each time, perform a function check as described in the corresponding section of these instructions for use.

6.1 Measuring weight

WARNING!

Injury from falling

Baby scales are usually placed on elevated work surfaces. If the baby were to fall from the work surface, it could suffer serious irreversible or fatal injuries.

- Ensure that the device is steady and level.
- Never leave a baby unsupervised.

CAUTION!

Risk of falling, incorrect operation

To prevent falls from the scale and ensure correct operation, you must check that the scale is ready for operation before every use.

- Make sure that the tray is secured correctly.
- Perform a function check.
- Check whether the beam is horizontal.

WARNING!


Personal injury, faulty measurement

- Before using the device each time, perform a function check as described in the corresponding section of these instructions for use.

Weighing

✓ The locking lever of the beam is released.

1. Place the baby on the scale.
2. Move the large sliding weight until pointer and counterpointer are at approximately the same level.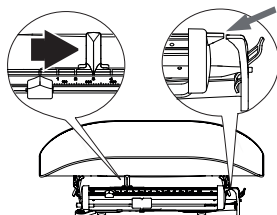



CAUTION!

Faulty measurement as a result of force shunt

If you touch the scale, weight will not be measured correctly.

- Do not touch the scale while the beam is rocking.

3. Wait until the beam has stopped rocking.
4. Move the small sliding weight until pointer and counterpointer are at exactly the same level.
5. Wait until the beam has stopped rocking.

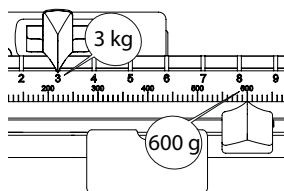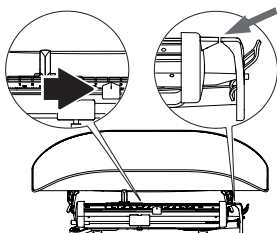6. Read off the measuring result.
 - Read off the kilogram value on the large sliding weight (here: 3 kg = 3,000 g)
 - Read off the gram number on the small sliding weight (here: 600 g)
 - Add the two values (here: 3,600 g)

Taring off additional weight

Using the tare function, you can prevent any additional weight (e.g. a towel or a cushioned mat on the weighing surface) from affecting the measuring result.

WARNING!

Faulty measurement

When the tare function is used incorrectly, the weight of the patient may not be determined correctly.

- Perform the steps as described to prevent faulty measurements.

WARNING!

Faulty measurement

If an additional weight is touching the surface on which the device is standing, the weight will not be measured correctly.

- Make sure that additional weights are placed only on the tray.

NOTE

The maximum weight which can be displayed is reduced by the weight of the objects already placed on the scale.

1. Place the additional weight on the scale.
2. Tare the beam (→ [Align the beam horizontally, page 41](#)).
3. Perform the weighing procedure (→ [Weighing, page 43](#)).

7 HYGIENE TREATMENT

WARNING!

Risk of infection

- ▶ Subject the device to a hygiene treatment at regular intervals as described in this section.

NOTICE!

Damage to device

Unsuitable cleaning agents and disinfectants may damage the sensitive surfaces of the device and impair its operability.

- ▶ Do not use aggressive or abrasive cleaning agents.
- ▶ Do not use organic solvents (e.g. white spirit or petroleum spirit).

7.1 Cleaning

Cleaning housing

- ▶ If required, moisten a soft cloth with a mild soapy solution and wipe the device over with it.

7.2 Disinfecting

1. Disinfect the device at regular intervals using a disinfectant suitable for sensitive surfaces and acrylic glass. seca recommends a 70 % ethanol solution as a surface disinfectant.
2. Follow the instructions for use of the disinfectant.
3. Disinfect the device:
 - ▶ Moisten a soft cloth with disinfectant and wipe down the device with it.
 - ▶ Comply with the intervals, see table:

Interval	Component
Before and after each measurement	Tray
As required	Controls

The device must not be sterilized.

8 FUNCTION CHECK

CAUTION!

Personal injury

If you find faults or deviations during the function check which you are unable to remedy with the aid of the “Troubleshooting” section in this document, you must not use the device.

- ▶ Have the device repaired by seca Service or an authorized service partner.
- ▶ Follow the section entitled “Servicing” in this document.

- ▶ Perform a function check before each use.

A complete function check includes:

- A visual inspection for mechanical damage
- A test of the alignment of the device
- A visual and function check of the display elements
- A function check of all the controls shown in the section entitled “Overview”
- A function check of the optional accessories

If you find faults or deviations during the function check, initially try to remedy the fault with the aid of the “Troubleshooting” section in this document.

9 TROUBLESHOOTING

Fault	Cause	Remedy
When a load is on the scale, the beam does not move	The scale is not ready for use	• Remove the transport lock (rubber ring) • Release the locking lever
Implausible measuring results	Scale is aligned incorrectly	Align the scale
	Beam was not aligned horizontally before the weighing procedure	Align the beam horizontally
	Force shunt, scale is touching a wall or the housing is in contact with something (e.g. a towel)	• Move scale away from wall • Set up the scale so that only its feet are in contact with the ground
	Force shunt, an additional weight (e.g. a towel) is touching the surface on which the scale is standing	Make sure that additional weights are placed only on the tray

10 SERVICING

10.1 Verified scales

CAUTION!

Faulty measurements as a result of verification being performed poorly or not at all

- ▶ Have verification performed only by authorized persons.
- ▶ Always have verification performed if one or more quality seals are damaged.

seca recommends having your device serviced prior to verification.

CAUTION!

Faulty measurements as a result of poor servicing

- ▶ Have servicing and repairs carried out exclusively by seca Service or an authorized service partner.
- ▶ You can find your local service partner at www.seca.com.

10.2 Non-verified scales

The product needs to be set up carefully and serviced regularly. Depending on how frequently the product is used, seca recommends servicing at intervals of 3 to 5 years.

CAUTION!

Faulty measurements as a result of poor servicing

- ▶ Have servicing and repairs carried out exclusively by seca Service or an authorized service partner.
- ▶ You can find your local service partner at www.seca.com.

11 TECHNICAL DATA

11.1 General technical data

General technical data	
Dimensions seca 725	
• Depth	312 mm
• Width	552 mm
• Height	174 mm
Dimensions seca 745	
• Depth	312 mm
• Width	552 mm
• Height	179 mm
Net weight	approx. 6 kg
Ambient conditions, operation	
• Temperature	+10 °C to +40 °C / +50 °F to 104 °F
• Air pressure	700 – 1060 hPa
• Humidity	30 % – 80 % no condensation
Ambient conditions, storage/transport	
• Temperature	-10 °C to +65 °C / +14 °F to 149 °F
• Air pressure	700 – 1060 hPa
• Humidity	0 % – 95 % no condensation
Medical device in accordance with Regulation (EU) 2017/745	Class I with measuring function

11.2 Weight measurement

seca 745	
Verification in compliance with Directive 2014/31/EU	Class III
Maximum capacity	16 kg
Minimum capacity	0.2 kg
Graduation	10 g
Tare range	2.5 kg (subtractive)
Accuracy on initial verification	
• < 5 kg	± 5 g
• > 5 kg	± 10 g

seca 725	
Maximum capacity	16 kg
Graduation	5 g
Tare range	2.5 kg (subtractive)
Accuracy	
• < 4 kg	± 10 g
• > 4 kg	± 0.25 %

12 DISPOSAL

12.1 Disposing of the device

This device consists of high-quality raw materials which can be recycled. Dispose of the device in accordance with the local laws and regulations applicable in your country.

13 WARRANTY

There is a two-year warranty period from delivery for defects attributable to poor materials or workmanship. All movable parts, e.g. batteries, cables, power supply units, rechargeable batteries etc. are exempt. Defects which come under the warranty will be repaired for the customer free of charge against proof of purchase. Additional claims cannot be considered. Costs of transport to and from seca are the responsibility of the customer if the device is located somewhere other than the customer's headquarters. In the event of transport damage, claims under warranty can only be made if the complete original packaging was used for transport and the device was secured and fastened in it according to its originally packaged condition. You should therefore keep all packaging parts.

The warranty will be voided if the device is opened by persons not expressly authorized by seca to do so.

In the event of a warranty issue, please contact your local seca office or the dealer from whom you ordered the product.

14 DECLARATION OF CONFORMITY

seca gmbh & co. kg hereby declares that the product complies with the terms of the applicable European directives and regulations. The unabridged declaration of conformity can be found at www.seca.com.

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